 **esperanza**<sup>®</sup>  
*creating space for your dreams*

# **ELECTRIC BREAST PUMP**

## **BEBITA**

**MANUAL**  
**ECM001B**

# ELECTRIC BREAST PUMP

## TECHNICAL PARAMETERS

3 types of power supply: USB charging/ mains charging/ 4 x 1.5 V AA batteries (alkaline batteries NOT included)

Voltage (input): 100-240 V (AC~) 50/60 Hz

Voltage (output): 5 V (DC) 1.0 A

Maximum suction pressure: 0.32 bar

Interval usage: 30 minutes ON / 30 minutes OFF

2 working modes: stimulation and suction

9 suction levels

5 stimulation levels

Big LCD display

Super quiet in working

150ml milk bottle

skin-friendly massage cushion

High efficiency suction funnel

Bisphenol A - BPA Free

natural baby sucking effect

The breast pump set allows nursing mothers to express their breast milk easily and comfortably. If not enough breast milk is taken from mothers who have recently given birth, this can lead to the painful build-up of milk, which in the worst case scenario could cause inflammation of the nipples (mastitis). The stagnation of milk can be prevented through targeted expressing of breast milk using the breast pump.

In addition, breast pumps help to alleviate cracked or sore nipples. You can use the integrated stimulation function to draw out flat or inverted nipples, making it easier to pump the breast milk. The breast pump also allows you to store breast milk in the bottle, for example if you will be travelling or away from your baby for an extended period of time. Babies who cannot be breast-fed directly for health reasons (due to a premature birth or cleft palate) can still be given breast milk by using the breast pump.

## INTENDED USE

The breast pump must be used exclusively for pumping breast milk from nursing mothers. Do not use the breast pump on animals. The breast pump is only intended for domestic/private use, not for commercial use. This device may only be used for the purpose for which it is designed and in the manner specified in the instructions for use. Improper use can be dangerous. The manufacturer is not liable for damage resulting from improper or careless use.


## **IMPORTANT SAFETY INSTRUCTIONS**

- Keep packaging material away from children to avoid danger of suffocation.
- Keep small parts out of reach of children.
- The breast pump must not come into contact with hot surfaces or sharp-edged objects.
- Do not use any additional parts that are not recommended by the manufacturer or offered as accessories.
- For hygiene reasons, the breast pump may only be used on one person. Do not share the breast pump with other nursing mothers.
- The milk pump must only be used on the female breast.
- Never use the breast pump if feeling drowsy or when asleep.
- Never use the breast pump in the bath or the shower.
- Never heat up breast milk in the microwave as it may be heated unevenly and could burn your child when they drink it.
- When transporting breast milk in the bottle (e.g. when on the move or travelling), make sure that the bottle cap is always screwed on tightly.
- Make sure that cables are not accessible to children.
- Only use the charger included in package.
- Children must not play with the device.
- Cleaning and user maintenance must not be performed by children unless supervised.
- Do not pull, twist or bend the mains cable.
- Before using the breast pump for the first time, ensure that the technical specifications of the charger match those of the motor unit.
- Pull the charger out of the socket in the case of operational faults, before cleaning and when the device is not in use.
- In the event of a fault, do not open or shake the breast pump under any circumstances. Please contact Customer Services or your retailer.
- Repairs must only be carried out by Customer Services or authorized retailers. Under no circumstances should you open or repair the breast pump yourself, as faultless functionality can no longer be guaranteed thereafter. Failure to comply will result in voiding of the warranty.
- Never submerge the breast pump in water.
- Never touch the breast pump with wet or damp hands.
- Never reach for the breast pump if it has fallen into water. Remove the charger from the socket immediately if the breast pump has fallen into water.
- When using the breast pump, place it on an even and stable surface.
- Unwind the mains cable fully in order to avoid overheating.
- This device must not be used by people with reduced physical, sensory or mental skills or a lack of experience or knowledge.
- Protect the breast pump and its accessories against impacts, humidity, dirt, marked temperature fluctuations and direct sunlight.
- Do not pull on the breast pump tubes.

## **WARNING FOR CHILD SAFETY**

- This device may only be used under adult supervision.
- Feeding from the teat continuously and for long periods can lead to tooth decay.
- Use the teat for feeding, not as a pacifier.
- Always check the temperature of the milk before feeding.
- Inspect the device before each use and pull the teat in all directions.
- Dispose of the teat at the first sign of damage or wear and tear.
- Do not expose the teat to direct sunlight or sources of heat and do not leave in disinfectant ("sterilizing solution") for longer than recommended as this may damage the teat.
- Keep the teat away from sweet substances as these may cause tooth decay.
- Do not use any abrasives or anti-bacterial cleaning agents.
- Sterilize all parts that come into contact with the breast milk for five minutes in boiling water.
- Clean the device before each use.
- Make sure that the temperature of the liquid in the bottle never exceeds 50°C.

## DEVICE DESCRIPTION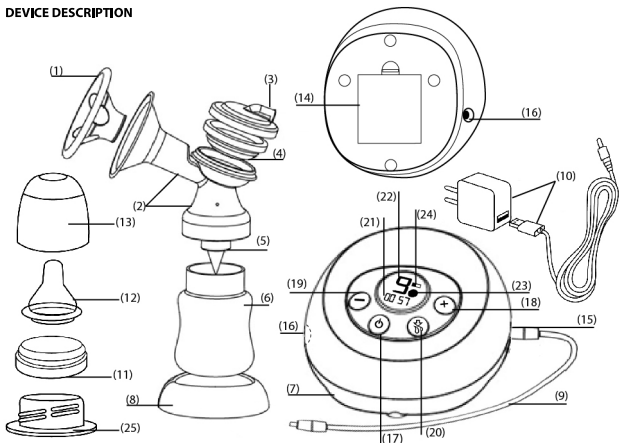


- (1) Petal massage cushion
- (2) Main unit of pump
- (3) Pump cover with tube inlet
- (4) Silicone membrane
- (5) Silicone valve
- (6) Milk bottle
- (7) Motor unit with display
- (8) Bottle holder
- (9) Silicone tube
- (10) 2in1 charger (USB charger and mains charger)
- (11) Screw ring
- (12) Teat
- (13) Teat cap
- (14) Battery compartment
- (15) Tube inlet on motor unit
- (16) Charging hole
- (17) "⏻" On/off button
- (18) "+" button
- (19) "-" button
- (20) "⚙️" Working mode switch button
- (21) Time display
- (22) Suction level display
- (23) Working mode display ("⚙️" stimulation mode / "⏻" suction mode)
- (24) Battery status display
- (25) Adapter for NUK (the breast pump can also be used with this bottle if desired)

## INITIAL USE

Check that the packaging of the breast pump has not been tampered with and make sure that all components are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed.

### Assembling the breast pump

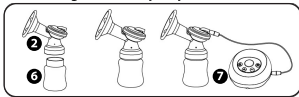- Screw the main unit of pump onto the bottle. Ensure the pump is firmly secured so that no liquid can escape.
- Insert the silicone tube into the tube inlet on pump cover and the tube inlet on motor unit.
- Connect the breast pump to the mains cable and insert the mains plug into the socket.
- If you choose to operate the breast pump using batteries, open the battery compartment cover on the rear side of the motor unit. Insert 4 AA batteries. Make sure that the batteries are inserted correctly by following the marks of battery polarity inside the battery compartment.


## CLEANING AND STERILIZING THE BREAST PUMP

- Clean and sterilize the separate parts of the breast pump that come into direct contact with the breast milk (except for the motor unit) individually before first use and any further use.


Note: Never sterilize the motor unit.

To sterilize the breast pump correctly, proceed as follows:

- a. Unscrew the main unit of pump from the bottle and remove the silicone tube from the pump and the motor unit.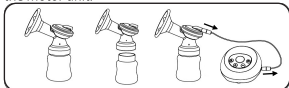

- b. Remove the massage cushion, silicone membrane and silicone valve from the pump set.

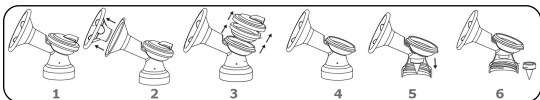- c. Sterilize the bottle and pump set (with massage cushion, silicone membrane and silicone valve) for 5 minutes in boiling water or using a steam sterilizer.

Note:

- Do not use the breast pump until all individual parts are completely dry.
  - Do not use any antibacterial cleaning products, since these may damage the plastic on the breast pump.
  - Be careful when you remove the white valve and when you clean it. If it gets damaged, your breast pump does not function properly.
  - To clean the valve, rub it gently between your fingers in warm water with some washing-up liquid. Do not insert objects into the valve, as this may cause damage.
- d. Clean the motor unit carefully using a slightly damp cloth only. Do not use any cleaning agents or solvents. Under no circumstances should you hold the motor unit under water, as this can cause liquid to enter and damage the breast pump. If you do not intend to use the breast pump for a prolonged period of time, remove the batteries.

## OPERATION

### Tips to ensure a better milk flow

- Choose a location to use the pump where you can relax.
- Choose a time to use the pump when you can relax.
- Having your child nearby or looking at a picture of your child can have a positive effect on the milk flow.
- If feeding your child using one breast, use the breast pump on the other breast at the same time.
- Warmth and relaxation encourage milk flow. Use the breast pump after having a shower or bath.
- If your breasts are swollen or sore, place a warm towel on them for a few minutes. This will improve the milk flow.
- If the milk does not start to flow from your breasts immediately, try to relax and then try again a few minutes later.
- If the breast pump still does not extract any milk from your breasts after five minutes, stop using it and try again later.
- If you experience any sharp pains when using the breast pump, consult a doctor.

### Operating the breast pump

1. Before using the breast pump, wash your hands and breasts thoroughly.
2. Sit with your upper body bent forwards slightly. You can support your back better by using a pillow.
3. Place the breast shield with silicone cushion gently on the breast. Make sure that the nipple is completely enclosed and there are no air pockets. To ensure proper function, the breast shield must be drawn firmly onto the breast.
4. To switch on the breast pump, press the on/off button. The breast pump will begin automatically in stimulation mode. Your breast nipple will be stimulated in order to prepare it for pumping the breast milk. Use the "+/-" buttons to set the desired stimulation intensity.

5. To begin pumping the breast milk, press the working mode button. Use the "+" / "-" buttons to select a suction intensity that is comfortable for you. Stop using the pump once there is 150 ml of milk in the bottle.

6. If you want to switch back to stimulation mode, press the working mode button.

Note: The breast pump has a memory function, which recalls only the last programmed stimulation level when it is switched on.

7. To switch off the breast pump, press the on/off button.

### STORING THE BREAST MILK

Expressed milk can be stored in the fridge or freezer for a certain period of time.

- When storing breast milk, ensure that the breast pump and the bottles are disinfected before usage. Only store breast milk in disinfected containers.

- Store breast milk in the fridge or freezer immediately after expressing. Do not leave breast milk out at room temperature.

- If storing breast milk in the freezer, it is recommended that you mark the bottle with the date of expressing.

### WARNING

Observe the storage periods in the following table:

	ROOM TEMPERATURE	REFRIGERATOR(approx. 3 - 5°C)	FREEZER(approx. -16°C)
Recently expressed breast milk	Can be kept for 6 hours	Can be kept for 24 hours (Do not store in the fridge door, as it is not sufficiently cool there)	Can be kept for 3 months
Defrosted breast milk (that had been	Use immediately	10 hours	Do not refreeze

### FEEDING WITH THE BREAST MILK

You can use the supplied bottle with the silicone teat when feeding with the breast milk.

- Prior to feeding, ensure the bottle has been disinfected before filling it with the breast milk that has been stored.

- To defrost breast milk, store it overnight in the fridge, so as to retain the nutrients. Once the breast milk is defrosted, it must be used within 24 hours. Breast milk should only be defrosted in warm water in especially urgent cases.

- If the breast milk smells unpleasant, it should be disposed off immediately.

- Never heat up breast milk in the microwave as it may be heated unevenly and could burn your child when they drink it.

- If heating up the breast milk, check its temperature before feeding it to your baby.

### TROUBLESHOOTING

Problem	Potential solutions
There is no suction from the breast pump.	<ul style="list-style-type: none"><li>• Check all parts of the breast pump are correctly assembled.</li><li>• Check whether the pump set is correctly placed on the breast.</li></ul>
No breast milk is being pumped.	<ul style="list-style-type: none"><li>• Check that the silicone tube is correctly inserted, the suction level is not set to the minimum and the breast pump is switched on.</li><li>• Ensure that there is no milk remaining in the silicone valve and obstructing the milk flow.</li></ul>
Pumping the breast milk is causing pain.	<ul style="list-style-type: none"><li>• Try to relax and then attempt it again a few minutes later. As you use the breast pump more and more, you will find it easier to pump milk.</li><li>• Reduce the suction level.</li><li>• Consult your doctor.</li></ul>
The breast milk is leaking out underneath the silicone cushion.	<ul style="list-style-type: none"><li>• Remove the silicone cushion and re-attach it. Ensure the silicone cushion is correctly positioned and firmly secured to the pump unit.</li><li>• Lean your upper body forwards slightly.</li></ul>
One of the breast pump components has become altered or damaged.	<ul style="list-style-type: none"><li>• Do not use any corrosive, gaseous cleaning agents or solvents.</li><li>• In some cases, marked fluctuations in temperature can damage the plastic on the breast pump.</li><li>• If any of the parts becomes damaged, stop using the breast pump.</li></ul>
The breast pump will not switch on.	<ul style="list-style-type: none"><li>• Ensure that the breast pump is correctly assembled and switched on.</li><li>• Ensure that the mains plug is correctly connected to the breast pump.</li></ul>

## WARRANTY

1. This warranty is issued by EDC Poterek Sp. Jawna with address:  
ul. Poznańska 129/133, 05-850 Ożarów Mazowiecki, Phone: 022-7213571 w.10 – called hereinafter Esperanza.
2. This warranty confers upon you specific rights and remedies against Esperanza. Please read this document carefully, by making a claim under this warranty, you will be deemed to have understood and accept its terms.
3. Esperanza warrants that any Esperanza product together with attached accessories is free from defects in materials and workmanship under conditions of its normal use.
4. If you make a claim under this warranty, Esperanza will, at its option, repair or replace any defective product into new one by using new or use factory-refurbished parts. In case of using factory-refurbished parts they will be equal to new products.
5. Defective product should be returned together with receipt, commercial invoice or any other proof of purchase to the place of purchase. It should include all accessories attached in original packing. Esperanza has right to reject warranty if any accessories will be missed.
6. Esperanza is committed to respond to the complaint within 14 calendar days of its notification in the point of sale. Esperanza will repair the defective product in the shortest possible time. If will be need to import from abroad of any parts which are necessary to repair this time may be extended.
7. In case of replacing defective product onto new one, Esperanza has right to replace it with newer model with the same technical parameters if the old one model was replaced by new one during warranty period.
8. If you make claim under this warranty, you are deemed to have agreed that repair and (upon availability) replacement, as applicable, under the warranty services described herein, is your sole and exclusive remedy vis-a-vis Esperanza.

## EXCLUSIONS:

### This warranty does not extend to:

1. Any product not manufactured by Esperanza
2. Any product that has been damaged or rendered defective as a result of:
  - use it in other way than normal intended use
  - modification or internal interferes or changes of product
  - service or repair by anyone other than Esperanza service
  - improper store, transportation or packing
  - improper installation of product or software
  - mechanical, physical, chemical etc. damages
3. In no event shall Esperanza be liable for any damages, loss of business, profits, data or loss based on use Esperanza products with other devices.
4. Products with removed warranty and other labels and products spent or have hard use marks.

All products are covered by Esperanza's 2-year manufacturer's warranty while product is purchased by the consumer (the consumer is a person who buys product for the purpose of not professional or commercial activity). In the case of purchase for the company (for purposes relating to the activities of professional or commercial) warranty is 1 year from date of purchase. Esperanza products have all the necessary safety certificates required in the EU.

According to Directive 2012/19/EU, disposal of waste electrical and electronic equipment together with household waste is prohibited. Their components need to be recycled or disposed separately. Otherwise, they can cause the release of hazardous and noxious substances which pose a threat to the environment and human health. The law requires each consumer to return unnecessary or waste electrical and electronic equipment to the producers, sales agents or waste treatment facilities. Detailed conditions are governed by national legislation. The above symbol placed on the product, in the user manual or on the packaging refers to this requirement. Thanks to proper segregation and compliance with the provisions relating to waste disposal each user contributes significantly to environmental protection.

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