nenio®

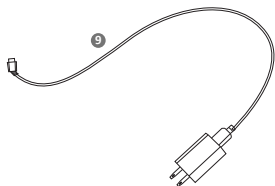
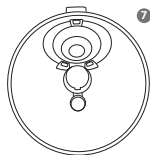
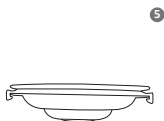
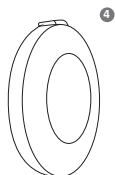
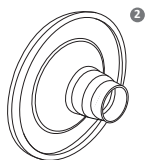
Ovio Twin

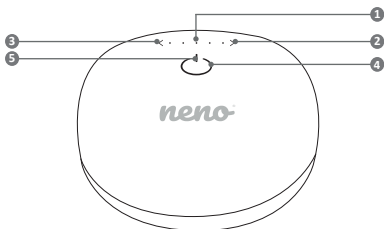
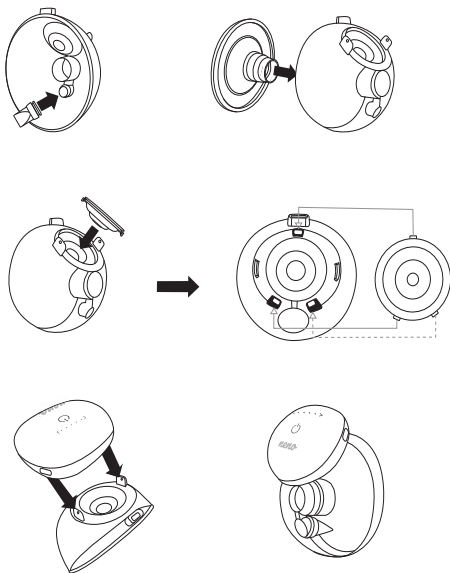
Table of contents



Spis treści

6	Instrukcja obsługi	PL
11	User manual	EN
17	Bedienungsanleitung	DE
23	Návod k použití	CZ
29	Használati utasítás	HU
35	Používateľská príručka	SK
40	Användarhandbok	SE
46	Käyttäjän käsikirja	FI
52	Brukerhåndbok	NO
57	Brugermanual	DK
63	Gebruikershandleiding	NL
69	Manual del usuario	ES
75	Manuale utente	IT
81	Manuel de l'utilisateur	FR
87	Manual de utilizare	RO
93	Korisnički priručnik	HR/BA
98	Uputstvo za upotrebu	RS/ME
104	Uporabniški pravilnik	SI
110	Εγχειρίδιο χρήστη	GR
116	Lietotāja rokasgrāmata	LV
122	Vartotojo vadovas	LT
127	Kasutusjuhend	ET
133	Посібник користувача	UA
139	Ръководство за потребителя	BG
145	Упатство за употреба	MK
151	Manual do utilizador	PT

A

B**C**

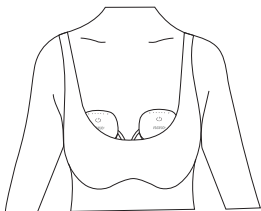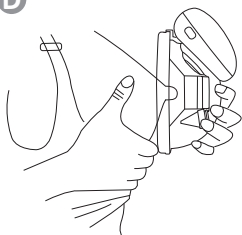D

E

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USER MANUAL

Dear Customer,

The Neno OVIO TWIN device you have purchased is an electronic, three-phase shell breast pump with a dedicated mobile app. The device is used to express milk from a woman's breast after childbirth and has been designed with comfort, effectiveness and discretion in mind. The mobile app allows you to remo-

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EN

The crossed-out wheeled bin symbol indicates that unwanted electrical or electronic equipment, including accessories (such as power supplies or cables) and components (e.g., batteries, if included), must not be disposed of with household waste. Instead, these items should be taken to an appropriate collection point where they will be accepted free of charge for proper disposal or recycling. This requirement is in accordance with the revised WEEE Directive (2012/19/EU) and Regulation (EU) 2023/1542, which amends Directive 2008/98/EC and Regulation (EU) 2019/1020, and repeals Directive 2006/66/EC. Proper disposal of such devices helps prevent environmental damage. Information on local collection points is available from the relevant municipal authorities. Improper disposal of waste may be subject to penalties under applicable local laws.

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Producer:

KGK TREND Sp. z o.o.
Ujastek 5b, 31-752 Cracow, Poland.
Made in PRC