# B.FIT

## BLUETOOTH BATHROOM SCALE 8IN1

### MANUAL

**EBS016K    EBS016W**

# BLUETOOTH BATHROOM SCALE 8IN1

## TECHNICAL PARAMETERS

- weight measuring range: 5kg - 180kg
- 3 units: kilograms "kg", pounds "lb" and stones "st"
- automatic user recognition
- 8 users available
- automatic shutoff function
- low battery indication
- overload indication
- weight and bone mass measurement precision: 100g
- body fat, body water, muscle percentage precision: 0.1 %
- caloric requirements precision: 1 kcal
- 20 measurement storage for registered user when data can not be transferred to app
- big LCD display with blue backlight : 75\*31 mm
- 6mm high-tempered safety glass platform: 30\*30cm
- foot-tap switch on
- Power supply: 2\* AAA Battery (not included)
- working temperature: 0°C - 40°C
- Bluetooth 4.0 or higher
- free "OKOK international" app
- 8 diagnostic parameters of each measurement

## IMPORTANT SAFETY INSTRUCTIONS

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

### WARNING

- The scales must not be used by persons with medical implants (e.g. heart pacemakers). Otherwise their function could be impaired.
- Do not use during pregnancy.
- Do not stand on the outer edge to one side of the scale otherwise it may tip!
- Batteries are highly dangerous if swallowed. Keep batteries and scale out of reach of small children. If batteries are swallowed, get medical help immediately.
- Keep children away from packaging materials (risk of suffocation).
- Caution! Do not step on the scale with wet feet or when the scale's surface is damp – danger of slipping!
- This device is for personal use only and is not intended for medical or commercial applications.
- Note that technical tolerances mean that results may vary, because the scale is not calibrated for professional medical use.
- Place the scale on a firm level floor; a firm floor covering is required for correct measurement.
- Protect the unit against hard knocks, moisture, dust, chemicals, major temperature fluctuations and heat sources which are too close (stoves, heating radiators).

## CHILDREN AND FRAIL INDIVIDUALS

- In order to ensure your children's safety, please keep all packaging (plastic bags, boxes, polystyrene etc.) out of their reach. **WARNING!** Do not allow small children to play with the foil as there is a danger of suffocation!
- This device is not intended to be used by individuals (including children) who have restricted physical, sensory or mental abilities and/or insufficient knowledge and/or experience, unless they are supervised by an individual who is responsible for their safety or have received instructions on how to use the device.
- Children should be supervised at all times in order to ensure that they do not play with the device.

## LIMITATIONS

When measuring body fat and other values, deviating and implausible results may occur in:

- children under approx. 10 years,
- competitive athletes and body builders,
- pregnant women,

- persons with fever, undergoing dialysis treatment or with symptoms of edema or osteoporosis,
- persons taking cardiovascular medication (affecting the heart and vascular system),
- persons taking vascular dilating or vascular constricting medication,
- persons with considerable anatomic deviations of the legs in relation to total body size (length of the legs considerably shortened or lengthened).

## GENERAL TIPS

- If possible, always weigh yourself at the same time of day (preferably in the morning), after using the toilet, on an empty stomach and unclothed, in order to achieve comparable results.
- Important for the measurement: only measure body fat when barefoot and with the soles of the feet slightly moist. Completely dry soles can result in unsatisfactory measurements due to inadequate conductivity.
- Stand still and upright during the measurement.
- Wait a few hours after unaccustomed physical activity.
- Wait approx. 15 minutes after getting up in the morning to allow the water in the body to distribute.
- Remember that only long-term trends are important. Short-term changes in weight within a few days are usually caused by a loss of fluid. Body water plays an important role towards our general wellbeing.

## SYSTEM REQUIREMENTS

Apple system requirements: IOS 6.0 or higher + Bluetooth 4.0 or higher

Android system requirements: Android 4.3 or higher + Bluetooth 4.0 or higher

## THE MEASURING PRINCIPLE

This scale operates according to the B.I.A. principle (bioelectric impedance analysis). This enables the measurement of physical relationships within seconds by means of an undetectable, completely harmless electric current. The body fat percentage and other physical relationships in the body can be determined by measuring the electrical resistance (impedance) and calculating constants and individual parameters such as age, height, gender and degree of activity. Muscle tissue and water have good electrical conductivity, and therefore lower resistance. Bones and fatty tissue, on the other hand, have low conductivity as fat cells and bones conduct hardly any current as a result of their very high resistance.

Please be aware that values obtained from the diagnostic scale represent only an approximation of actual analytical medical data. Only a specialist physician can accurately determine body fat, body water, muscle percentage and bone structure using medical procedures (e.g. computed tomography).

## INITIAL USE OF THE SCALE

### Inserting batteries

On the back of the scale you can find the battery compartment. Insert the batteries into the scale ensuring that the battery polarity is correct. If the scale does not function, remove the batteries completely and reinsert them.

### Change the unit

Press the "UNIT" button which is on the back of the scale, you can switch between kilograms "kg", pounds "lb" and stones "st".

### Setting up the scale

Place the scale on a firm level floor; a firm floor covering is required for correct measurement.

## INITIAL USE WITH THE APP

Scan below QR code to download OKOK app or install "OKOK.International" from App store.

Start the app and follow the instructions.

Note: The displaying language of this app will keep the same as the language of your smartphone. You can change your smartphone language to change the app language.

You MUST enter personal user parameters in order to measure your body fat percentage and

other physical data. The scale has 8 user memories which allows, for example, you and your family members to save personal settings. It is also possible to activate users via other mobile end devices on which the OKOK app has been installed or by changing the user profile in the app (see the app settings).

### **ACTIVATE BLUETOOTH® IN THE SMARTPHONE SETTINGS.**

Note: Bluetooth Smart devices, such as this scale, are not visible in the general settings of the Bluetooth device list. Bluetooth Smart devices are visible only in special apps or in device manufacturer apps.

Keep the smartphone close to the scale in order to maintain an active Bluetooth® connection during initial use.

### **USER REGISTRATION**

The following settings must be set or entered in the "okok international" app:

Email address/password /Initials (or abbreviation) of user name /Body height (100 to 220 cm )/ Age ( 10 to 100 Years ) / Sex ( man or woman)/ Target weight

After all the settings are done, you will have an profile of the registered user. You can add and register other family members in the same way.

### **TAKING MEASUREMENTS**

- Place the scale on a firm level floor; a firm floor covering is required for correct measurement.
- Switch on Bluetooth: When your Bluetooth is not switched on, app will ask you to switch on Bluetooth first.
- Click "OKOK" icon to run the app. Follow the app instructions to operate.
- Step on the scale with bare feet and ensure that you are standing still with equal weight distribution and with both legs on the electrodes.

Note: The measurement result will be incorrect if the measurement is carried out with socks on.

- Bind scale to your smart phone: In the first use, App will show that a Bluetooth scale is found and ask you to bind the scale, then click button "bind".
- The scale begins weighing immediately. When the weight flicks 3 times, the measurement result is stable and shows on LCD. Then another sign "0000" appears. After that, other parameters show in turn on LCD, such as, weight, BMI, fat water, bone, BMR...
- After binding with your scale, the measurement results will show on app automatically.

NOTE: If the user is not recognized, only the weight can be shown.

### **EVALUATION OF RESULTS**

Every measurement has following 8 essential parameters and every parameter is analyzed with interpretations and suggestions.

Weight in kg / BMI / Body fat in % / Organ fat in % / Body water in % / Muscle percentage in % / Bone mass in kg / Basal metabolic rate in kcal (BMR)

#### **bone mass**

Like the rest of our body, our bones are subject to the natural development, degeneration and ageing processes. Bone mass increases rapidly in childhood and reaches its maximum between 30 and 40 years of age. Bone mass reduces slightly with increasing age. You can reduce this degeneration somewhat with healthy nutrition (particularly calcium and vitamin D) and regular exercise. With appropriate muscle building, you can also strengthen your bone structure. Note that this scale will not show you the calcium content of your bones, but will measure the weight of all bone constituents (organic substances, inorganic substances and water). Little influence can be exerted on bone mass, but it will vary slightly within the influencing factors (weight, height, age, gender), no recognized guidelines or recommendations relating to bone mass measurement.

#### **ATTENTION:**

Please do not confuse bone mass with bone density. Bone density can be determined only by means of a medical examination (e.g. computer tomography, ultrasound). It is therefore not possible to draw conclusions concerning changes to the bones and bone hardness (e.g. osteoporosis) using this scale.

#### **BMR**

The basal metabolic rate (BMR) is the amount of energy required by the body at complete rest to maintain its basic functions (e.g. while lying in bed for 24 hours). This value largely depends on weight, height and age. It is displayed on the diagnostic scale in kcal/day units using the scientifically recognized Harris-Benedict formula. Your body requires this amount of energy in any case and it must be reintroduced into your body in the form of nutrition. If you take on less energy over the longer term, this can be harmful to your health.

## **RESULTS IN RELATION TO TIME**

Remember that only long-term trends are important. Short-term fluctuations in weight over a few days are usually the result of a loss of fluid. The interpretation of the results will depend on changes in your overall weight and body fat, body water and muscle percentages, as well as on the period during which these changes take place. Sudden changes within days must be distinguished from medium term changes (over weeks) and long term changes (months). A basic rule is that short term changes in weight almost exclusively represent changes in water content, whereas medium and long term changes may also involve the fat and muscle percentages.

- If your weight reduces over the short term, but your body fat percentage increases or remains the same, you have merely lost water – e.g. after a training session, sauna session or a diet restricted only to rapid weight loss.
- If your weight increases over the medium term and the body fat percentage falls or stays the same, then you could have built up valuable muscle mass.
- If your weight and body fat percentage fall simultaneously then your diet is working – you are losing fat mass.
- Ideally you should support your diet with physical activity, fitness or power training. This way you can increase your muscle percentage over the medium term.
- Body fat, body water or muscle percentages should not be added (certain elements of muscle tissue also contain body water).

## **OTHER FUNCTIONS**

### **User assignment**

It is possible to assign up to 8 users for a weight-only measurement (with shoes) and for a diagnosis measurement (bare feet).

For a new measurement, the scale assigns the measurement to the user whose most recently saved measurement falls within +/- 3 kg .

When the measurement is over the recognizable weight, it shows as unknown measurement. You can use the app to assign the weight.

### **Saving the assigned measurements on the scale**

If the app is open and there is an active Bluetooth® connection to the scale, newly assigned measurements are sent immediately to the app. In this case, the values are not saved on the scale.

If the app is not open, the newly assigned measurements are saved on the scale. A total of 20 measurements can be saved on the scale. The saved measurements are transferred automatically to the app if you open the app within the Bluetooth® range.

Data synchronization takes place within 10 seconds. It is possible to automatically transfer data when the scale is switched off.

## **CLEANING AND MAINTENANCE**

The accuracy of the measurements and service life of the device depend on its careful handling:  
**IMPORTANT**

- The unit should be cleaned occasionally. Do not use abrasive detergents and never immerse the unit in water.
- Ensure that no liquid gets on the scale. Never dip the scale into water. Never wash it under running water.
- Do not place any objects on the scale when it is not being used.
- Protect the unit against hard knocks, moisture, dust, chemicals, major temperature fluctuations and heat sources which are too close (stoves, heating radiators).
- Do not expose the scale to high temperatures or strong electromagnetic fields (e.g. mobile phones).

## **ERROR SIGNS**

"Lo" : When "Lo" shows on LCD, it means the batteries should be replaced with new ones.  
"Err": over-load. The maximum load capacity of 180kg is exceeded.

## WARRANTY

1. This warranty is issued by EDC Poterek Sp. Jawna, with address:  
ul. Poznańska 129/133, 05-850 Ożarów Mazowiecki, Phone: 022-7213571 w.10 – called hereinafter Esperanza.
2. This warranty confers upon you specific rights and remedies against Esperanza. Please read this document carefully, by making a claim under this warranty, you will be deemed to have understood and accept its terms.
3. Esperanza warrants that any Esperanza product together with attached accessories is free from defects in materials and workmanship under conditions of its normal use.
4. If you make a claim under this warranty, Esperanza will, at its option, repair or replace any defective product into new one by using new or use factory-refurbished parts. In case of using factory-refurbished parts they will be equal to new products.
5. Defective product should be returned together with receipt, commercial invoice or any other proof of purchase to the place of purchase. It should include all accessories attached in original packing. Esperanza has right to reject warranty if any accessories will be missed.
6. Esperanza is committed to respond to the complaint within 14 calendar days of its notification in the point of sale. Esperanza will repair the defective product in the shortest possible time. If will be need to import from abroad of any parts which are necessary to repair this time may be extended.
7. In case of replacing defective product onto new one, Esperanza has right to replace it with newer model with the same technical parameters if the old one model was replaced by new one during warranty period.
8. If you make claim under this warranty, you are deemed to have agreed that repair and (upon availability) replacement, as applicable, under the warranty services described herein, is your sole and exclusive remedy vis-a-vis Esperanza.

## EXCLUSIONS:

### This warranty does not extend to:

1. Any product not manufactured by Esperanza
2. Any product that has been damaged or rendered defective as a result of:
  - use it in other way than normal intended use
  - modification or internal interferes or changes of product
  - service or repair by anyone other than Esperanza service
  - improper store, transportation or packing
  - improper installation of product or software
  - mechanical, physical, chemical etc. damages
3. In no event shall Esperanza be liable for any damages, loss of business, profits, data or loss based on use Esperanza products with other devices.
4. Products with removed warranty and other labels and products spent or have hard use marks.

All products are covered by Esperanza's 2-year manufacturer's warranty while product is purchased by the consumer (the consumer is a person who buys product for the purpose of not professional or commercial activity). In the case of purchase for the company (for purposes relating to the activities of professional or commercial) warranty is 1 year from date of purchase. Esperanza products have all the necessary safety certificates required in the EU.

According to Directive 2012/19/EU, disposal of waste electrical and electronic equipment together with household waste is prohibited. Their components need to be recycled or disposed separately. Otherwise, they can cause the release of hazardous and noxious substances which pose a threat to the environment and human health. The law requires each consumer to return unnecessary or waste electrical and electronic equipment to the producers, sales agents or waste treatment facilities. Detailed conditions are governed by national legislation. The above symbol placed on the product, in the user manual or on the packaging refers to this requirement. Thanks to proper segregation and compliance with the provisions relating to waste disposal each user contributes significantly to environmental protection.

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