medivon®

User manual

EYE MASSAGER

PL | EN | DE | FR | ES | PT | IT | CZ | SK
SL | EE | LT | LV | HU | RO | RU | NB | FI
EL | HR | SV | DA | NL | TR | BG | AR

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Weight: around 750 g

Normal working conditions:

Operating environmental temperature: +5°C~+40°C

Relative humidity: ≤80%

Atmospheric pressure: 86kPa~106kPa

Transportation and storage conditions:

Storage environmental temperature: -20°C~+55°C

Relative humidity: ≤93%

Atmospheric pressure: 50kPa~106kPa

2. Device operation

A short press of the POWER(3) button turns on the device. If the battery LED flashes after startup, please charge the device using the included USB cable. It is recommended to fully charge the battery before using the device for the first time. After putting the massager on your head, make sure it is positioned comfortably, you can adjust the settings using the elastic band (1)(2).

Device modes are controlled by repeatedly pressing the power button (3). Subsequent functionalities that are currently active will be highlighted on the screen using LEDs (6). The device has 5 individually prepared cyclical operating modes.

The eye massager has a Bluetooth function that is enabled by default when the device is turned on. Just connect with your mobile device and search for "Horizon Pro" from the list.

A short press of the music button (4) starts the next song from the list, and a longer press (4) turns off the music mode.

To turn off the massager after relaxation, hold down the POWER(3) button again.

3. Warnings and contraindications

Before using the product, please read and understand the operating instructions carefully.

Please clean the product's eye shield before use.

Children and people unable to express their needs should not use the product.

People with sensitive skin or insensitive to heat should not use the product.

Pregnant women, people suffering from heart disease, circulatory disorders, malignant tumors, cerebrovascular diseases and other acute diseases, as well as people under the care of a doctor, should not use the product.

If you experience any discomfort or abnormal skin reaction while using the product, stop using it immediately and consult a doctor.

People with eye problems should avoid using the product or use it under medical supervision. People who have undergone eye surgery and patients suffering from glaucoma, cataracts, retinal detachment and other eye diseases should not use the product.

People wearing contact lenses should remove them before using the product. Do not use the product in places with high temperature, flammable, humid or places with electromagnetic radiation.

Do not disassemble the product without permission; Please make sure the product is free from impact and water to avoid failure.

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Used electrical and electronic equipment and batteries and accumulators must not be mixed with other waste. This is indicated by the crossed-out bin symbol on the product packaging. Electrical and electronic equipment and batteries and accumulators may contain hazardous substances, mixtures and components that are harmful to the environment and human health. Improper handling or damage to them may result in damage to health or environmental pollution. Proper handling of used electrical and electronic equipment and batteries and accumulators in the household contributes positively to the reuse and recovery, including recycling, of used equipment. Used electrical and electronic equipment and batteries and accumulators can be returned to local selective waste collection points. The distributor is obliged to collect used equipment from households free of charge at the point of sale, provided that the used equipment is of the same type and has performed the same functions as the equipment sold. A distributor, when delivering household equipment to a purchaser, is obliged to take back used household equipment free of charge at the place of delivery of such equipment, provided that the used equipment is of the same type and has performed the same functions as the equipment delivered. A distributor running a retail outlet with a sales area within the meaning of art. 2 item 19 of the Act of 27 March 2003 on spatial planning and development of at least 400 m² devoted to the sale of household equipment, is obliged to accept, free of charge, in that outlet or in its immediate vicinity, used household equipment, none of the external dimensions of which exceeds 25 cm, without the need to purchase new household equipment. The product was introduced to the market after 13 August 2005

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The manufacturer declares under his sole responsibility that the Horizon Pro device complies with Directive 2014/30/EU of the European Parliament and of the Council of February 26, 2014 on the harmonization of the laws of the Member States relating to electromagnetic compatibility, as well as Directive 2011 of the European Parliament and of the Council /65/EU of 8 June 2011 on the restriction of the use of certain hazardous substances in electrical and electronic equipment, together with Commission Delegated Directive (EU) 2015/863 of 31 March 2015 amending Annex II to the Directive of the European Parliament and 2011/65/EU as regards the list of restricted substances. The full text of the declaration of conformity is available at the internet address: www.medivon.pl/deklaracje

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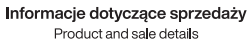
This Guarantee is provided by Armare S.A., a company governed by Polish law, with its registered office in Warsaw, ul. Grzybowska 87, 00-844, Poland. You can find us in the official Polish register of companies under the KRS number: 0000730590, NIP: 5213833614 and REGON: 380721860. We guarantee that the equipment you have purchased from us (hereinafter referred to as the "Product") will work properly. If something goes wrong for a reason inherent in the Product, this warranty protects you. The warranty starts from the day you receive the Product and lasts: 24 months, if you are a consumer or a natural person running a business and the purchase of the Product is not of a professional nature for you. Otherwise, the warranty period is 12 months. If the law in your area requires a longer warranty, the warranty period will be extended to meet local law requirements.

In addition, if the Product does not comply with the contract, you have the right to legal protection from and at the expense of the seller. This warranty does not affect these rights - it does not limit, suspend or exclude them.

Detailed warranty conditions, how to submit complaints, what happens when you find a defect, what are exceptions to the warranty, when the warranty expires or is extended, and what are the rules regarding the processing of personal data, you can find on our website:
<http://medivon.pl/warranty>

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Importer: Armare S.A. | ul Grzybowska 87, Warszawa, 00-844, Polska