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INSTRUCTION MANUAL

Electric Breast Pump LOVI Prolactis 3D Soft

Please read this instruction manual carefully before using the device.

Important safety information:

Failure to observe this instruction manual and the safety information may result in danger and the device failure. Use the device only as intended and described in this manual. The information contained in this manual is subject to appropriate technical changes.

INTRODUCTION

Congratulations on your purchase of the Electric Breast Pump **LOVI Prolactis 3D Soft**! It is a revolutionary breast pump, created for the mother's highest comfort.

It has a 3D funnel that mirrors the baby's suckling process. This is a breakthrough solution – the 3D funnel - unlike to classic one, not only sucks, but also squeezes the nipple. The funnel surrounds and squeezes the breast evenly, as baby's mouth and tongue do, working simultaneously in multiple dimensions. Even, gentle pressure stimulates the release of prolactin - a hormone responsible for milk production, causing the quick flow of milk, without discomfort.

The LOVI Prolactis 3D Soft breast pump is:

- **fast**, because it can express up to 100 ml of milk in just 10 minutes,
- **delicate**, as the soft silicone funnel adjusts to the breast,
- **two-phased** - reflects the two phases of expressing milk like a baby: stimulation imitates fast suckling movements of baby stimulating milk production; and deep suckling phase - reflects slower, deeper, efficient suckling of milk.

It is ideal for situations requiring frequent milk expressing. It also provides support for active and working mums who need alternative ways to feed their babies with breast milk.

LOVI Prolactis 3D Soft is the support in comfortable expression of milk, without losing the precious time.

NOTE: LOVI brand recommends breastfeeding as the healthiest feeding method. It is recommended to start using an alternative feeding method only after consulting your medical practitioner, midwife or pharmacist if breastfeeding is impossible or insufficient.

Breast milk is the healthiest and most valuable food for newborns and babies. It is recommended as sole food given to your baby in the first six months of his or her life. After six months, it should be combined with solid foods.

NOTE! The breast pump is not intended for expressing milk when a hungry child cannot empty the breast himself or herself, i.e. in all cases when your milk ducts are blocked.

USEFUL TIPS

When should I use a breast pump?

- If possible, wait until your lactation stabilize, what usually occurs within 2-4 weeks after birth. Exceptions to this rule:
- Your breasts produce too much milk, and they become swollen and hard (breast fullness). Expressing a small amount of milk before or during feeding can give you a relief and help your baby to grasp the nipple.
- Your nipple is inverted because of the long interval between feedings.
- You have concave, flat or damaged nipples (using a breast pump can improve nipple shape).
- You feel overflow between feeding and do not want to wake the baby.
- The baby is sick and does not want to suckle.
- Your breasts produce too little milk, using the breast pump between feedings helps increase milk production.
- You are working, or you are away from home and want to continue breastfeeding, you should regularly express milk using a breast pump to sustain the natural process of lactation.

How often should I express milk?

If you temporarily cannot breastfeed, it is recommended that you express milk using a breast pump at least eight times a day. That will keep your milk production at the appropriate level. In the beginning, it may be difficult for you to express more milk, but after a few days, you will be able to express more. Even with stable lactation, the possibility of expressing milk in one session is slighter (30-60 ml), therefore to collect one portion of milk, you may require several expressing sessions.

BASIC PRODUCT INFORMATION

Breast pump No. XN/MD-2253A

The device, which is an eclectic breast pump, hereinafter referred to as the breast pump, is a product designed for breastfeeding mothers for expressing breast milk. It enables collection of milk for later use, i.e. in the case of separation from the child. The breast pump helps maintain lactation and prepares for your planned absence or random events such as treatment, cracked and sore nipples.

- The device is intended for personal use.
- The device warranty is 24 months.
- Before using, read the instruction manual and follow the rules of safe use.
- Keep the instruction manual together with the device.
- The expected life of the product is two years.
- Silicone parts may change their colour permanently. These elements must be replaced after finding discolouration or after the estimated time of use.

Follow the recommendations on the correct use of the device and its accessories. Never use a damaged or incomplete/broken device.

Damaged or worn parts should be replaced immediately. The manufacturer is not liable for damages resulting from inconsistent with this instruction manual and improper use. Any damages arising from misuse cannot be the basis for complaint or repairs under warranty. Making any repairs or modifications to the product by unauthorized persons results in loss of warranty and may result in risks to health and life.

GENERAL SAFETY RULES

- If you experience problems with breastfeeding or feeding your baby causes you pain, you shall seek medical advice before using the breast pump.
- It is best to use the breast pump for the first time under specialist supervision.
- We recommend to read specialist literature and get to know the principles of baby feeding.

NOTE! Insufficient suction may result in a smaller amount of expressed milk, but too high suction can cause soreness during expressing and reduce the production of milk. If you experience any of these problems, adjust the suction power (see: SUCTION POWER).

- Keep the device clean. Prevent deposition of dust and other contaminants that could affect the performance of the product.
- Taking care of hygiene is essential for the safety of the mother and her baby.
- Parts of the device which cannot be washed should be protected against water and moisture.

This marking indicates that the product should not be disposed of with other household wastes. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this from other types of wastes and recycle it responsibly to promote the sustainable reuse of material resources. For further information on the recycling of this product, please contact the municipal authorities in your area of waste disposal service or the supplier of the product.

The manufacturer declares that the product complies with European Union directives and harmonized standards. The product has been marked by conformity assessment procedures.

The product meets the requirements of the directive concerning electrical equipment designed for use within certain voltage limits (LVD) 2014/35/EU, the Electromagnetic Compatibility Directive (EMC) 2014/30/EU, and RoHS Directive 2011/65/EU.

It is the Class II appliance.

WARNING! This product is not a toy. Keep all parts away from children. Keep the instructions of use as it contains important information. Batch code is on the packaging.

PRODUCT MISUSE

- Never use a product with any signs of damage.
- Never use the device during the bath.
- Never use the product in locations exposed to contact with water, for example near a bathtub, washbasin, kitchen sink or shower unit.
- Never use the device if there is possible that you can fall asleep.
- Never connect the device to an AC adapter outdoors.
- Never use an AC adapter with a damaged power cord. The damaged AC adapter can be repaired by the manufacturer's authorized service centre only.
- Never use the product contrary to its purpose and conditions of use.
- Do not resell a used device.

DEVICE SPECIFICATION

Battery supply	Li-ion battery 3,7V , 1100mAh
AC power supply	100-240 V / 50/60Hz
Maximum power	5 W

PRODUCT COMPONENTS

Fig 1. Breast pump parts

1. Funnel cover
2. Silicon massage cushion with anti-backflow valve 2 pcs
3. Breast pump body with funnel
4. Silicone diaphragm
5. Cover
6. Tube connecting pump body with the unit 2 pcs
7. AC power supply (USB)
8. Tube connectors 4 pcs
9. Pump unit
10. Base
11. Bottle 1 pc
12. Screw ring
13. Teat
14. Bottle adapter
15. Hood/cup with scale
16. Sealing disc
17. Storing bag

Fig 2. Display

1. LCD display
2. Battery level
3. Working time
4. Vacuum reduction button
5. Vacuum increase button
6. Phase indicator - stimulation or expression
7. On/off button
8. Suction power level
9. Mode indicator - stimulation or expression

CLEANING AND DISINFECTION INSTRUCTIONS

Before first and each next use disassemble the breast pump. Wash all components of breast pump and bottle in warm water with mild detergent, then rinse thoroughly and disinfect in accordance to rules below.

Before first use, disinfect all components which come in contact with mother's milk (nr.2,3,4,11,12,13,14,15,16), by placing them for 5 minutes in boiling water. This is to ensure hygiene. NOTE! Do not leave components on the bottom of the pot. Stir the water during disinfecting to prevent damage to components.

- Be sure that all components are completely submerged in boiling water during disinfection.
- Do not boil elements that don't have direct contact with mothers milk and electric elements.
- Remove all components from water immediately after boiling.

Before subsequent use, disinfect all above mentioned elements of breast pump and bottle by pouring with boiling water or placing in electric or microwave steam sterilizer, to ensure hygiene.

WARNING! There are no other elements that can be disinfected in microwave sterilizer.

- Do not pour water on bladder cap (nr.5). If you will notice water on or in the cap, pull out the cup plug and leave both elements to dry, so that water won't enter the tubing and the pump unit.
- Warning! Do not disinfect silicone tubing with connectors, plastic brace, bladder cap and electric elements. Do not submerge these elements in water. Wipe with disinfectant.

After each disinfection completely air- or towel-dry before reassembly and next use. Dried parts should be stored in an airtight container or a plastic bag. You can also use a clean towel.

Warning! Partially dry components may affect the operation of your Electronic Breast Pump. Make sure that all parts are completely dry before use.

Washing instructions:

- Always wash the product immediately after use to remove any residual milk.
- Disconnect the tubing with connectors. These elements don't need cleaning, because the silicone bladder protects them from contact with feed.
- Clean all components of the breast pump in accordance with the instructions mentioned above.
- DO NOT use any solvents or chemicals to clean the product.
- Rinse with clean, hot water.
- If you use electric sterilizer for disinfecting elements please follow the instructions given by its manufacturer.

Silicon massage cushion with anti-backflow valve should be washed very gently and the food should not be allowed to dry out on its surface.

Warning! Before each use, check the condition of all elements for cracks, tears, discoloration or deterioration, as they may affect the effectiveness of the device.

How to disinfect the teat?


Before first and each use wash all parts of the product and disinfect. Place for 5 min in boiling water. This is to ensure hygiene. Clean before each use. Boiling and hard water may create lime scale effect. Wash all parts thoroughly with clean water, including areas that may be invisible but where food has passed, to remove any food residues. After using, remove the remaining food from the bottle and the teat, rinse, wash and disinfect in a steam/electric sterilizer or in boiling water. Bottle can be washed in dishwasher (max. temp. 60°C). It is not recommended to place the teat in a dishwasher. Hard chemicals / detergents may damage the product. Do not use any antibacterial agents. Inspect the feeding teat before each use and pull the feeding teat in all directions. Especially when the baby has teeth. Throw away at the first signs of damage or weakness. Do not tighten up the bottle locking ring, as it may disturb the teat's ventilation. Always check for leakage. Never enlarge the teat flow holes by any means, as it can damage the teat. Take particular care when heating in a microwave. Always stir heated food to ensure even distribution of temperature and check liquid temperature before giving it to the child. When warming food in a microwave, remember to leave the bottle open. Do not place product in hot oven or heat in a gas/electric cooker. Do not leave a feeding teat in direct sunlight or heat, or leave in disinfectant ("sterilising solution") for longer than recommended, as this may weaken the teat. It is recommended to replace a teat max. after 2 months of use. Retain the instruction for later use. Bottle made from Polypropylene. Batch code is placed on the packaging.

For your child's safety and health


WARNING!

Continuous and prolonged sucking of fluids will cause tooth decay.
Always check food temperature before feeding.
Throw away at the first signs of damage or weakness.
Keep components not in use out of the reach of children.
Never attach to cords, ribbons, laces or loose parts of clothing. The child can be strangled.
Never use feeding teats as a soother.
Always use this product with adult supervision.

ASSEMBLING LOVI ELECTRIC BREAST PUMP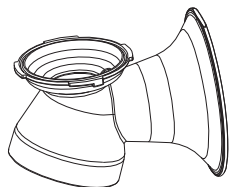


Hold the breast pump body with the funnel (3) as shown in the picture.
Make sure the hole with thread is facing up.

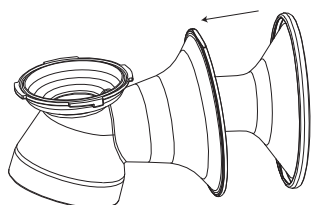
Place the silicone massage cushion with the valve (2) in the funnel pointing the valve towards the funnel. Then place the valve gently into the breast pump body by lightly pushing it down. Place the massage cushion on the funnel and make sure that the edges of cushion fit evenly to the edges of the funnel. Take a note that both elements are profiled and should be properly matched to each other.
NOTE! The cushion with the anti-backflow valve should be assembled very gently.


Screw the bottle adapter to the bottle (14). Screw the whole set into the breast pump body.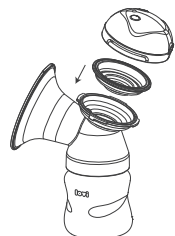


Place silicone diaphragm in the goblet of breast pump body. Then place the cover on the breast pump body and rotate in a clockwise direction in such a way that the hole for the tube connector will be located on the opposite side of the funnel.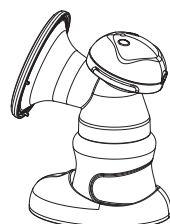

Place a complete set on the base (10).

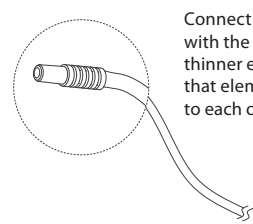
Connect the tube connectors (8) with the tube (6) by placing their thinner ends in the tubing. Note that elements should tightly fit to each other.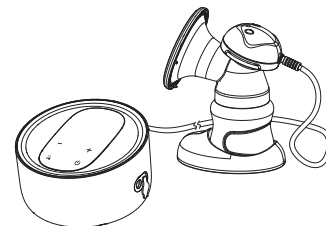

Connect tube with connectors with the breast pump and pump unit. Put one end in the opening of the cover and the second in the port placed on the control panel. Press connectors to ensure that they fit in the openings tightly.

POWER OPTIONS

Using AC adapter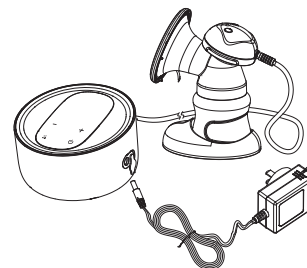

Place AC adapter ending in the port placed on the pump unit and then plug the AC adapter into an electrical outlet. The AC adapter for the breast pump is 5V 1A.

AC adapter pins may vary depending on the country. It is recommended to use the original AC adapter which was provided with the breast pump. However, the most of adapters of mobile phones are also 5V 1A or 5V 2A. They can be connected with the USB cable of the breast pump. It can damage the pump unit if the other adapter is more than 5V.

Important! Before using other adapter, you should pay attention to the voltage of it first. The above concerns countries where the EU adapter (which is included in the set) does not operate.

Using Li-ion battery

Charging the battery is done by using the AC adapter. Charging takes approx. 3 hours.


NOTE! Before first use, fully charge the battery. One full charging of the battery allows the pump to operate for approx. 1 hour (about 2/3 sessions). To extend the life of the breast pump, charge the battery before it is completely discharged. When the battery powers the pump unit, the display shows the current charge status. The current battery charge status is displayed in the dynamic form. When the battery runs low, you will hear a beep and the icon will flash. A used battery can only be replaced by an authorized manufacturer's service centre.

INSTRUCTION MANUAL

Before each use:


- Wash your hands thoroughly before using the breast pump.
- Ensure that the pump is pre-washed and dried, if not, follow the instructions below.
- Ensure that there is no liquid or moisture in the tube. Never switch on the pump if you notice any water in the tube. Note! Do not mouth blow the tube to remove water.

NOTE! Before expressing check the charge level of the battery.
If the battery is running low, connect the AC adapter to the pump unit and let the battery recharge.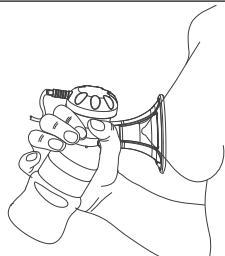

1. Place the breast pump on your breast so that the nipple is deep inside the funnel (exactly in the middle).

NOTE! The silicone cushion must cling tightly to the breast. Otherwise, the suction may be reduced.

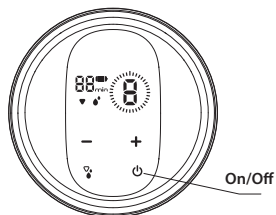2. The breast pump is equipped with a touch screen. Touch the on/off icon to start expressing. You will hear a beep sound. You will hear it during every changing of settings on the control panel.

3. At the same time, the pump unit starts 30 min countdown. When the timer reaches 30 minutes, the engine switches off. If you touch on/off icon during expressing, the pump will turn off. The second touch of the on/off icon will start counting down again. The breast pump will switch on and immediately start working in the stimulation phase on the first power level.

4. To adjust the suction power in the stimulation phase to your needs, regulate it's level by touching icons of (+) or (-).

5. There are nine levels of suction powers available in the stimulation phase.

6. If you want to go to the deep suction phase, press the button with drops.

7. By touching the (+) or (-) icons on the control panel, you can adjust the suction power in the deep suckling phase to suit your needs. You can return from deep suckling to stimulation phase by touching the same button on the control panel.

8. If you prefer to skip the stimulation phase or when the milk begins to flow earlier, touch the icon with a drop to change the phase.

9. When you will end expressing, touch on/off icon and turn off the breast pump.

WARNING! Before putting the breast pump away from the breast, make sure it is turned off. If necessary, you can put the breast pump away during pumping by placing your finger between the pad and the breast (just as you do it when putting the child away).

WARNING! The breast pump switches off automatically after 30 min. of work. The breast pump will switch off irrespective of whether or not it was adjusted during pumping.

SUCTION POWER

Whenever you select a suction level (in both stimulation and deep suckling phase), the breast pump remembers it until you switch it off or change it. When you finish expressing after switching the pump on again, it will start to work at 1 level of the stimulation phase. If milk gets under the silicone massaging cushion, reduce the suction level so that milk keeps up flowing into the bottle.

DECLARATION OF CONFORMITY EU No. 2/2021

In scope of directives: 2014/30/EU, 2014/35/EU, 2009/125/EC and 2011/65/EU

Manufacturer: Canpol Sp. z o.o.
Puławska Street 430, 02-884 Warsaw, Poland

We hereby declare under our responsibility that our breast pump (cat. no. 50/050)

brand: Lovi, breast pump - model: XN/MD-2253A,

is in accordance with the following directives:

- directive relating to electrical equipment designed for use within certain voltage limits
2014/35/EU

- directive relating to electromagnetic compatibility 2014/30/EU

- directive ErP 2009/125/EC

- directive RoHS 2011/65/EU

and is in accordance with the following harmonized standards:

EN 60335-1:2012 (PN-EN 60335-1:2012)

EN 55014-1:2006 (PN-EN 55014-1:2017-06)

EN 55014-2:2015 (PN-EN 55014-2:2015-06)

EN 61000-3-2:2014 (PN-EN 61000-3-2:2019-04)

EN 61000-3-3:2013 (PN-EN 61000-3-3:2013-10)

EN 62233:2008 (PN-EN 62233:2008)

as well as other technician specifications:

The Commission Regulation (EC) No. 278/2009

The last two digits of the year in which the CE marking was affixed: 20

Warsaw, 13.01.2021

(place and date of issue)

Chairman of the Board
Canpol Spółka z o.o.

Tomasa Biedrzycki

(name, surname and signature of authorized person, position)

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