Cuscinetti Thermogel per il seno

Thermogel breast pads
Coussinets d'allaitement Thermogel
Thermogel-Kissen für die Brust
Almohadillas Thermogel para el pecho
Discos Thermogel para os seios
Borstcompressen Thermogel
Kompresy żelowe Thermogel na piersi
Μαξιλαράκια Thermogel για το στήθος
Thermogel göğüs pedleri
Подушечки Thermogel для груди
Thermogel ضمادات الثدي

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EN Thermogel breast pads

PLEASE READ THESE INSTRUCTIONS CAREFULLY BEFORE USE AND KEEP THEM FOR FUTURE REFERENCE.

Thermogel breast pads offer you valid support during the delicate period of breastfeeding. Safe and easy to use, they provide immediately relief in the case of breast engorgement, breast tension, or before feeding to accommodate milk flow, favouring calm breastfeeding.

Used cold, they reduce inflammation and soothe pain. When used as a hot treatment, they permit the milk to flow more freely, helping to resolve problems related to milk blockage or stagnation and make breastfeeding easier. Discrete, soft and effective, the pads are easy to insert in a nursing bra.

WHEN TO USE THE THERMOGEL PADS

During breastfeeding, problems can arise that could cause some discomfort for the mother:

LACTATION ONSET

This is a phenomenon, which is often annoying, that takes place 24 - 48 hours after birth. It is characterised by significant swelling of the mammary glands, accompanied by congestion, an increase in the local temperature and a sense of tension, which can be considerable. The use of specific aids in combination with the regular suckling of the newborn and a delicate breast massage can accommodate the milk flow and decongestion of the breast (hot compresses) and provide relief (cold compresses).

BREAST ENGORGE MENT

This is a significant increase in mammary size and tension, accompanied by swelling, redness, pain, an increase in the local temperature, followed by stagnation of milk in the mammary ducts and glands. Breast

engorgement must be kept under control because it could lead to the occurrence of mastitis (infection of the breast tissue) with fever that can also be high. The main cause of the problem is insufficient suckling of the nursing infant, which is not enough to empty the breast. This condition can be prevented by frequent breastfeeding or by emptying the breast using a breast pump after every feeding, and by applying hot local compresses that help the ducts to dilate and, when accompanied by a light breast massage, help the milk come out. Cooling the area after feeding helps soothe breast pain.

BREAST TENSION DUE TO EXCESSIVE MILK

When the frequency of feedings is reduced when starting to wean the baby or when the mother returns to work and must be away for a part of the day, too much milk may be produced that will not be consumed by the baby. The breast can become full and hard, with the risk of engorgement. Also in this case, in addition to emptying the breast using a breast pump, it is useful to apply hot compresses before expression, and cold compresses afterwards.

HOW TO USE THE THERMOGEL PADS

There are 2 ways to prepare the Thermogel pads for use.

1. For an effective cooling effect to soothe breast pain, if the breast is swollen, place the pads in the refrigerator (one or both pads, as needed) and leave them in the refrigerator at +4°C for at least 2 hours.
- They can always be left in the refrigerator so that they are always ready to be used.
 - Remove the pads from the refrigerator and dry them with a cloth or kitchen paper towels.
 - Insert them in the cotton covers and place them inside the bra cups for 20 minutes or as needed.

BEFORE APPLYING THEM ON THE BREASTS, TEST THE PRODUCT ON THE BACK OF YOUR HAND TO MAKE SURE THE TEMPERATURE IS CORRECT.

It is recommended to position the larger part of the pad on the most painful part of the breast.

2. Like a delicate warm treatment.

- Take the pads from the packaging, heat them in the microwave, placing them in a bowl of water at max 300 watts for 5 minutes.
- Remove the pads from the water and dry them with a cloth or kitchen paper towels.
- Use the pads when they are slightly warmer than your body.
- Insert the pads in the cotton covers.

BEFORE APPLYING THEM ON THE BREASTS, TEST THE PRODUCT ON THE BACK OF YOUR HAND TO MAKE SURE THE TEMPERATURE IS CORRECT.

It is recommended to position the larger part of the pad on the most painful part of the breast, and to keep them applied for 10-15 minutes for a delicate heat effect.

When used with a cover moistened with lukewarm water, it provides greater relief and helps to stimulate the production or let-down of milk.

COOLING/HEATING TEMPERATURES:

HOT:

- In the microwave: at max 300 watts for 5 minutes in a bowl with water.

COLD:

- In the refrigerator: +4 °C (there is no max time).

NB: The Thermogel pads are a very effective method both hot or cold. After using the hot pad, it is a good idea to wait a few minutes before breastfeeding your baby, because the skin of your breast could be too hot for him. By carefully following the instructions provided above, you can carry out the hot or cold treatment also multiple times during the day as needed. It is recommended to not keep the pads constantly on your breast to avoid compression of the ducts.

WARNING 

- It is important to follow the instructions for the breastfeeding hygiene and safety.
- Conserve the casing and the user instructions for future reference.
- Do not use the pads without their cover.
- Always store the pads and the covers in

their packaging or in a clean and dry location, at room temperature (max 40 °C), away from heat sources and sunlight.

- After each use, the pads must be washed with warm water and normal soap, dried carefully with a cloth or kitchen paper towels and put away in a clean and dry location.
- Wash the cover after every use. Please refer to the care label for instructions on cleaning the cover.
- Do not exceed the indicated power for the microwave. Do not boil or excessively heat the pads because an excessive temperature could burn the skin or damage the product.
- Do not cool the product in the freezer because a temperature that is too low could cause problems to the skin or damage the product.
- Do not place them on hot or sharp surfaces.
- Do not damage the casing and keep the gel from coming out. If broken, dispose of them as normal household waste.
- If the content comes into contact with eyes or skin, rinse thoroughly with water.
- Do not use on the skin if there are open wounds.
- If this product is used for a purpose other than what is intended, or is not used in compliance with the instructions for use, only the user will be considered responsible for any consequences. Therefore, any damage to the product or other consequences are not covered by the warranty.
- The pads must not be used by people who are insensitive to heat and/or cold and must not be used on babies.
- Do not swallow the content. If the gel is accidentally swallowed, seek immediate medical attention.
- Do not use the product after the expiration date printed on the casing.
- To avoid any possibility of infection, it is recommended not to share the pads with other mothers.
- Keep out of reach of children.
- The pads must not be considered a toy.
- Report any serious accident to the manufacturer and to the competent authorities for the given country.

 CE	CE marking
	Consult the instruction manual
	Warning
 REF	Product catalogue reference code
	Manufacturer
 LOT	Production Batch
 MD	Medical Equipment
	Expiring date

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Artsana S.p.A.