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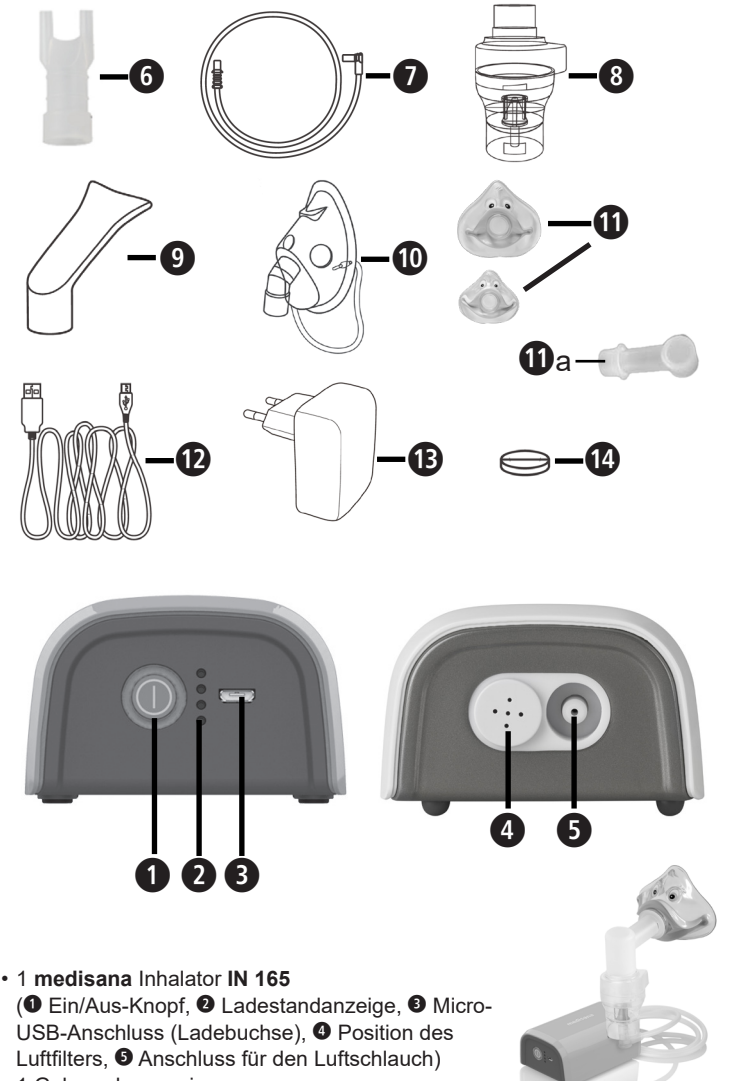
**medisana GmbH,**  
Carl-Schurz-Str. 2, 41460 NEUSS,  
GERMANY

Globalcare Medical Technology Co., Ltd  
7th Building, 39 Middle Industrial Main Road,  
European Industrial Zone, Xiaolan Town  
528415 Zhongshan City, Guangdong Province  
PEOPLE'S REPUBLIC OF CHINA  
Phone : +86 760 22589901

Donawa, Lifescience  
Piazza Albania, 10  
00153 Rome / Italy  
<https://www.donawa.com/wli/main/contatti/index>

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GB Instruction Manual  
Inhaler IN 165

Device and controls / Scope of supply

• 1 **medisana Inhaler IN 165**  
(**1** On/Off button, **2** Charging indicator, **3** Micro-USB port (charging socket), **4** Position of the air filter, **5** Connection for the air tube)

• 1 Instruction manual

• **Applied parts:** **6** Nosepiece, **7** 2 Air tubes, **8** Nebuliser, **9** Mouthpiece, **10** Adult face mask, **11** Child face mask (with adapter **11a**), **12** Micro-USB cable (charging cable), **13** USB charging adapter, **14** 5 × Air filter (replacement at position **4**), Storage bag (without illustration)

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use it and contact your supplier or your service centre.

If you notice any transport damage during unpacking, please contact your supplier without delay.

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required.

**WARNING**  
Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

Technical Specifications

|                                                                                                                       |                                                                                                                                                                                                                                                                                                                                                                                               |
|-----------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Name and model:                                                                                                       | medisana Inhaler IN 165 (type GCE845)                                                                                                                                                                                                                                                                                                                                                         |
| External power supply:                                                                                                | Input: 100–240V~ 50/60Hz, Output: 5V== 2A                                                                                                                                                                                                                                                                                                                                                     |
| Internal battery:                                                                                                     | 3.7V== 2570mAh<br>Charging time: approx. 4 hours<br>Battery working time: approx. 60 minutes<br>Expected battery charging life: 500 cycles                                                                                                                                                                                                                                                    |
| Power consumption:                                                                                                    | 10W                                                                                                                                                                                                                                                                                                                                                                                           |
| Nebulizing quantity:                                                                                                  | average 0.25 ml/minute (typ. 0.2 to 0.4 ml/minute)                                                                                                                                                                                                                                                                                                                                            |
| Aerosol performance according to EN ISO 27427:2019, based on an adult ventilatory pattern with Sodium Fluoride (NaF): | Aerosol output 0.51 ml<br>Aerosol output rate 0.16 ml/minute<br>Percentage of fill volume emitted per minute: 8 %<br>Residual volume: 1.47 ml<br>Particle size (MMAD): 2.9 µm<br>GSD (Geometric Standard Deviation): 2.8<br>RF (respirable fraction <5 µm): 71 %<br>Large particle range (>5 µm): 29 %<br>Medium size particle range (2 bis 5 µm): 26 %<br>Small particle range (<2 µm): 45 % |
| Max. Pressure:                                                                                                        | 70–120 kPa                                                                                                                                                                                                                                                                                                                                                                                    |
| Operating pressure:                                                                                                   | 35–60 kPa (with calibrated nozzle ø0.50)                                                                                                                                                                                                                                                                                                                                                      |
| Max. noise level:                                                                                                     | <45 dBA (1 m distance in front direction)                                                                                                                                                                                                                                                                                                                                                     |
| Nebuliser filling volume:                                                                                             | min. 2 ml, max. 6 ml                                                                                                                                                                                                                                                                                                                                                                          |
| Operating limits:                                                                                                     | 30 minutes On / 30 minutes Off                                                                                                                                                                                                                                                                                                                                                                |
| Expected service life:                                                                                                | 400 hours                                                                                                                                                                                                                                                                                                                                                                                     |
| Operating conditions:                                                                                                 | 10 to 40 °C / 50 to 104 °F<br>10 to 95 % relative humidity<br>Air pressure: 700 hPa to 1060 hPa                                                                                                                                                                                                                                                                                               |
| Storage and transportation conditions:                                                                                | –20 to +60 °C / –4 to +140 °F<br>10 to 95 % relative humidity<br>Air pressure: 700 hPa to 1060 hPa                                                                                                                                                                                                                                                                                            |
| Dimensions (L×W×H):                                                                                                   | approx. 109 × 73 × 45 mm                                                                                                                                                                                                                                                                                                                                                                      |
| Weight:                                                                                                               | approx. 230 g ±50 g                                                                                                                                                                                                                                                                                                                                                                           |
| IP Class                                                                                                              | IP 21                                                                                                                                                                                                                                                                                                                                                                                         |
| Article number:                                                                                                       | 54556                                                                                                                                                                                                                                                                                                                                                                                         |
| EAN number:                                                                                                           | 4015588 54556 6                                                                                                                                                                                                                                                                                                                                                                               |

GB IMPORTANT INFORMATION!  
KEEP IN A SAFE PLACE!

Please read this instruction manual carefully, in particular the safety instructions, before using the device. Keep the instruction manual in a safe place for later reference.  
If you pass the device on to a third party, this instruction manual must remain with the device.

- 
- Use the device only as described in this manual and therefore as an aerosoltherapy system, following the indications of your doctor. Any use different from the intended one is to be considered improper and hence dangerous; the manufacturer cannot be held liable for any damage caused by improper, incorrect and/or unreasonable use, or if the equipment is connected to electrical installations which do not comply with current safety regulations.
  - Keep the user manual for future reference.
  - Do not operate the unit in presence of any anaesthetic mixture inflammable with oxygen or nitrogen protoxide.
  - The correct functioning of the equipment can be affected by electromagnetic interferences which exceed the limits indicated by the European standards in force. In case this device interferes with other electrical devices, move it and plug it to a different power socket.
  - In case of failure and malfunction, read the chapter “Troubleshooting” in the instruction manual. Do not handle or open the compressor housing.
  - For repair operations address only to a technical service centre authorized by the manufacturer and require the use of original spare parts. The non-observation of the above mentioned indications can compromise the device safety.
  - Comply with the safety regulations concerning the electrical devices and in particular:
    - use only original components.
    - never submerge the unit in water.
    - never wet the device, it is not protected against water penetration.
    - never touch the unit with wet or moist hands.
    - do not leave the unit exposed to the weather elements.
    - place the unit on a stable and horizontal surface during its operation.
    - the use of this device by children and disabled requires always the close supervision by an adult with full mental faculties.
    - do not pull the power cord or the device itself to unplug it from the power socket.
    - the power plug is the separation element from the grid power; keep the plug accessible when the device is in use.
  - Before plugging in the device, make sure that the electrical rating, shown on the rating plate on the bottom of the unit, corresponds to the mains rating.
  - In case the power plug provided with the device does not fit your wall socket, address to qualified personnel for the replacement of the plug with a suitable one. In general, the use of adapters, simple or multiple, and/or extension cables is not recommended. If their use is indispensable, it is necessary to use types complying with safety regulations, paying however attention that they do not exceed the maximum power limits, indicated on adapters and extension cables.
  - Do not leave the unit plugged in when not in use; unplug the device from the wall socket when it is not operated or has fully charged.
  - The installation must be carried out according to the instructions of the manufacturer. An improper installation can cause damage to persons, animals or things, for which the manufacturer cannot be held responsible.
  - The power supply adapter cannot be replaced by the user. In case of damage, address to a technical service center authorized by the manufacturer for its replacement.
  - The power supply cord should always be fully unwound in order to prevent dangerous overheating.
  - Before performing any maintenance or cleaning operation, turn off the device and disconnect the plug from the main supply.
  - Some parts of the unit are so small that they may be swallowed by children; keep the equipment out of children’s reach.
  - If you decide not to use the device any longer, you have to dispose of it according to the current regulations.
  - Make sure to:
    - use this device only with medicines prescribed by your doctor.
    - make the treatment using only the applied part recommended by your doctor depending on the pathology.
    - use the nosepiece only if expressly indicated by your doctor and paying attention to NEVER introduce the bifurcations in the nose, but only bringing them as close as possible.
  - Check in the medicine package leaflet for possible contraindications for use with common aerosoltherapy systems.
  - To avoid strangulation and entanglement, keep cables and air tubes out of reach of young children.
  - Do not position the equipment so that it is difficult to operate the disconnection device.
  - Nebuliser and applied parts are single patient reuse. Device is multi-patient reuse.
  - Nebulizing system is not suitable for use in anaesthetic breathing system or a ventilator breathing system. The patient cannot undergo MRI scan while using this device.
  - This device is designed to nebulize solution and suspension liquids.
  - Do not use the device, if you spot damage or you notice something unusual.
  - This device consists of delicate components and must be handled with care. Observe the storage and operating conditions in the chapter “Technical Specifications”
  - The use of this device is not a replacement for visiting the doctor. In case of no health improvement after treatment consult again your doctor.
  - The device operates on battery, you can also use the device plugged to the mains.

Intended Use

The **IN 165** is an aerosoltherapy system suitable for domestic use. The device is designed for the production of compressed air to operate a small volume nebuliser for the production of medical aerosol for lung respiratory disorders.

Reference to standards

- Aerosol characteristics in accordance with regulation EN ISO 27427.
- Standards applied:
- EN 60601-1 (Medical electrical equipment, part 1: General requirements for basic safety and essential performance)
  - EN 60601-1-2 (Collateral Standard: Electromagnetic disturbances – Requirements and tests)
  - The device is a Class IIa medical device.

This device meets the requirements of the European Regulation on Medical Devices EU MDR 2017/745.  
Nebuliser, mouthpiece and masks are **type BF** applied parts.

Patient population:

The device is intended for use with children from 2 years old, adolescent and adult patients.

Intended users:

The user is an adult person without specific knowledge or professional ability. The patient is the intended operator except in case of children and patients that requires special assistance.

Indications:

Lung acute or chronic disorders or diseases of the respiratory disorders organs, or inflammation of the upper respiratory system.

Contraindications:

The device is not indicated to be used with quick-relief medications during a life-threatening asthma attack.

No contraindications exist to the administration of aerosols by inhalation. Contraindications related to the medicine used must be checked on the medicine package leaflet. Consult your physician in case of doubts.

Preparing the Device

Before using the device, carry out the cleaning operations as described in the chapter “Cleaning and Disinfection”.

Charging the Battery

Charge the battery in accordance with the following guidelines:

- Charge the battery only with the supplied charging adapter **13** and USB cable **12**.
- Before first use, remove the device from its packaging and verify that it is fully charged.
- To charge the battery, connect the larger of the two connectors of the USB cable **12** to the charging adapter **13** and plug the other end of the cable into the charging socket **3** of the inhaler. Plug the charging adapter **13** into a wall outlet. The charge level is displayed with the 4 LEDs **2**.

Application

You have two options for the power supply of your inhaler:  
Charge the device as described in chapter “Charging the Battery” (for example for use while travelling), or use the cable and adapter as a direct power supply.  
To start the treatment, briefly press the On/Off button **1**.  
While inhaling, sit upright and in a relaxed position at a table (not in an armchair), so as not to compress the airways and therefore not to impair the effectiveness of the treatment. Do not lay down during the inhalation. Stop the inhalation if you feel unwell.  
After you have finished inhaling for the amount of time recommended by your doctor, press the On/Off button **1** in order to switch off the device and remove the plug from the electrical socket.  
Empty the remaining inhalation solution from the nebulizer and clean the device as described in the chapter “Cleaning and disinfection”.

- This device was developed for operation in 30 minutes On / 30 minutes Off mode. Please switch off the device after 30 minutes and wait a further 30 minutes, before you continue the treatment.
- The device does not require calibration. Modification of the device is not permitted.

Cleaning and Disinfection

**WARNING**  
Before the first use and after each treatment: Carefully follow the cleaning and disinfection instructions of the applied parts, as they are very important for the device performances and the therapy success.  
Use original parts only.  
The applied parts cannot be cleaned and disinfected by any automated method.  
Do not boil nor autoclave the masks.

Limitations

The nebuliser **8** must be replaced after a long period of inactivity, especially if it shows deformations or breakings, or when the nebuliser nozzle is obstructed by dry medicine, dust, etc. We recommend replacing the nebuliser after a period between 6 months and 1 year, depending on the usage. The maximum times of cleaning and disinfection of the nebuliser are 360 times. The maximum times of cleaning and disinfection of the mask, the nosepiece and the mouthpiece are 360 times.

Preparation before cleaning

- Detach the air tube **7**, mouthpiece **9**, nosepiece **6**, mask **10/11/10a** from the nebuliser **8**.
- Make sure that all residual volume is removed from the nebuliser.
- Disassemble the nebuliser by turning counterclockwise the top and remove the medicine conduction cone.

Note: To avoid microbial growth and residual drug drying, clean and disinfect the applied parts immediately after each use.

Cleaning

- Briefly rinse all parts in running tap water for at least 10 seconds beforehand.
- Mix little dish detergent and warm tap water (i.e. dish washing liquid for hand washing in a ratio of 2 ml on 1 L) in a clean container. (We recommend the dish washing liquid of the brand **Fairy**, as it has been successfully tested with this mixing ratio.)
- Immerse the components of disassembled nebuliser, the mask, the mouthpiece and the nosepiece into the mixed water for about 5 minutes. Scrub all the surface of all the components with clean and small brush for at least 8 times.
- Afterwards, rinse all parts thoroughly in running tap water for at least 30 seconds, to fully remove any dish washing liquid residue.

Disinfection

- After cleaning, disinfect all disassembled parts. (Only parts that have been cleaned can be disinfected effectively).
- Boil the disassembled nebuliser, the mouthpiece and the nosepiece for 5 minutes in boiling tap water.
- Immerse the mask in a 2% sodium hypochlorite (NaOCl) solution for 15 minutes, (i.e. a solution made from the disinfecting agent Amuchina® or a 2% sodium hypochlorite solution mixed by your pharmacist). Afterwards, immerse the mask with sterile water for 3 minutes, then rinse all the surface of the mask with clean sterile water twice, to fully remove any possible residue of the disinfectant solution.

Drying

- Let all parts dry completely before reassembling and reuse to avoid risk of germ grow.
- Reassemble the nebuliser components and connect it to the air-outlet, switch the device on and let it work for 10–15 minutes.
- Care needs to be taken not to contaminate the parts after they have been cleaned and disinfected. Contamination can be avoided by good hand washing and not touching the inside sections of the device when laying them out to dry or when reassembling.

Inspection

Inspect all product components after each cleaning and disinfection. Replace any broken, misshaped or seriously discolored parts.

Packaging

Pack dry parts in a clean and sealed container when not in use. Do NOT pack wet or damp parts.

Storage

For the storage conditions, refer to chapter “Technical Specifications”.  
Note: Re-clean and re-disinfect the parts before next use if they have been stored for more than one day.

Transport

After cleaning and disinfection, always transport the parts with clean and sealed container. Contamination can be avoided by good hand washing and not touching the inside sections of any parts when you take out and re-assemble the parts for use.

Replacement of the air filter

Under normal conditions of use, the air filter **14/15** must be replaced approximately after 100 operating hours or after every year. We recommend to periodically check the air filter (after 10 – 12 treatments), and if the filter shows a grey or brown colour or is wet, replace it. Extract the filter and replace it with a new one. Do not try to clean the filter for reusing it. The air filter shall not be serviced or maintained while being in use with a patient. Only use original filters! Do not use the device without filter!

The instructions provided above have been validated by the manufacturer of the medical device as being capable of preparing a medical device for reuse. It remains the responsibility of the processor to ensure that the processing, as actually performed using equipment, materials and personnel in the processing facility, achieves the desired result. This requires verification and/or validation and routine monitoring of the process.

Troubleshooting

The device cannot be switched on

- Make sure the charging cable is properly plugged in. / Charge the battery.
- Make sure that the device has been used within the range of using time mentioned in this manual (30 min. on / 30 min. off).

The nebuliser functions poorly or not at all

- Make sure that the air tube **7** is properly attached on both ends.
- Make sure that the air tube **7** is not compressed, bent, dirty or blocked. If necessary, replace it with a new one.
- Make sure that the nebuliser **8** is completely assembled and the atomiser head has been correctly positioned and is not blocked.
- Make sure that the required inhalation solution is filled in the correct amount (up to 6 ml).

The device is noisy.

- Make sure that the filter is mounted properly.

Please report any serious incident that has occurred in relation to the device –injury or adverse event – to the local competent authority and to the Manufacturer or to the European Authorised Representative (EC REP).  
Vigilance contact points: <https://ec.europa.eu/growth/sectors/medical-devices/contacts/>

Disposal — Waste Electrical and Electronic Equipment (WEEE)

This product is subject to European Directive 2012/19/EU on waste electrical and electronic equipment and is marked accordingly. Never dispose of electronic devices with household waste. When disposing of batteries, keep cells or batteries of different electrochemical systems separate from each other. Please seek out information about the local regulations with regard to the correct disposal of electrical and electronic products. Correct disposal helps to protect the environment and human health.

All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Consult your local authority or your supplier for information about disposal.

**Device incl. 10/11:** These parts consist mainly of plastic, electronic components and a battery, and comply with RoHS und REACH.

**Applied parts (nebuliser, nosepiece, mouthpiece, masks) 6/7/8/9/10/11/10a:** These parts consist of plastic and comply with REACH. Before disposal, all these parts should be cleaned (according to the instructions in section “Cleaning”) and then disinfected by boiling for 5 minutes.

Warranty and repair terms

Your statutory warranty rights are not restricted by our guarantee below. Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

- The warranty period for **medisana** products is 3 years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
- Defects in material or workmanship will be removed free of charge within the warranty period.
- Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
- The following is excluded under the warranty:
  - All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
  - All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - Applied parts which are subject to normal wear and tear.
- Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

Globalcare Medical Technology Co., Ltd  
7th Building, 39 Middle Industrial Main Road,  
European Industrial Zone, Xiaolan Town  
528415 Zhongshan City, Guangdong Province  
PEOPLE’S REPUBLIC OF CHINA  
Phone : +86 760 22589901

Donawa, Lifescience  
Piazza Albania, 10  
00153 Rome / Italy  
<https://www.donawa.com/wli/main/contatti/index>

In accordance with our policy of continual product improvement, we reserve the right to make technical and optical changes without notice.

The current version of this instruction manual can be found under <https://docs.medisana.com/54556>.

Information about service can be found here: <https://www.medisana.com/servicepartners>

Explanation of Symbols

This instruction manual belongs to this device. The instruction manual includes important information on the initial start-up and handling. Read this instruction manual completely. Failure to follow these instructions may result in serious injury or damage to the device.

**WARNING**  
These warnings must be followed to prevent possible injury to the user.

**CAUTION**  
These instructions must be followed to prevent possible injury to the device.

**NOTE**  
These instructions provide you with useful additional information regarding installation or operation.

Information about protection type against foreign objects and against harmful effects due to the ingress of water.

Disposal only according to the “WEEE” Waste Electrical and Electronic Equipment EC Directive.

Recycling symbols/codes:  
These are used to provide information about the material and its proper use and recycling.

|        |                                                                             |       |                                                                                      |
|--------|-----------------------------------------------------------------------------|-------|--------------------------------------------------------------------------------------|
| REF    | Reference/ Article Number                                                   | I / O | On / Off                                                                             |
| SN     | Serial Number                                                               | ~     | Alternating voltage / Alternating Current                                            |
| LOT    | LOT number                                                                  | ==    | Direct voltage / Direct current                                                      |
| UDI    | Unique Device Identifier                                                    |       | Protection Class II                                                                  |
| #      | Type Number                                                                 |       | Medical Device                                                                       |
| EC REP | Authorised Representative                                                   |       | Applied part – Type BF degree of protection against electric shock (leakage current) |
|        | Manufacturer                                                                |       | Single patient multiple use (for applied parts only)                                 |
|        | Date of manufacture                                                         |       | Humidity range                                                                       |
|        | Importer                                                                    |       | Ambient pressure limitation                                                          |
| CE0123 | Device complies with European Regulation on Medical Device EU MDR 2017/745. |       | Temperature range                                                                    |

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