VITAMMY®

tape 1

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The instructions for use should be read carefully, as they contain important information for the patient on the use of the product.

Please keep these instructions so that you can read them again, if necessary.

KINESIOLOGY TAPE

Disposable medical device, intended for analgesic, orthopedic, neurological, postoperative and sports physiotherapy. It relieves pain, reduces muscle tension as well as stagnation and lymphoedema. Latex free. Non-sterile product.

PRODUCT APPLICATION:

The tape can be used to:

- pain relief,
- muscle support and regeneration,
- reduce muscle tension,
- improving circulation,
- reduction of swelling and lymphatic stagnation,
- loosening of soft tissues,
- improving posture,
- correction of incorrect position of joints and their stabilization.

COMPOSITION:

Cotton 97%, spandex, acrylic adhesive.

HOW TO USE:

1. Remove the product from the packaging.
2. Unwind the tape and use scissors to cut the desired length and shape. In order to prevent the corners of the tape from peeling off, its ends should be rounded (trimmed).
3. Carefully peel off the protective foil from the adhesive layer so that the layer is in contact with the smallest possible surface of the hand.
4. Before sticking to the skin, make sure that the adhesive side of the tape does not come into contact with any other surface.
5. Stick the tape to the right place on the body. **IMPORTANT:** The product should be applied to a clean and dry skin surface, and it is recommended to remove any abundant hair.
6. Removing the tape - peel off the tape in the opposite direction to the hair growth until the product is completely removed. Gently moisten the surface of the tape for easier removal.

Product for self-use, however, we advise using the tape in accordance with the recommendations of the doctor or physiotherapist.

TIME OF A SINGLE APPLICATION OF TAPE ON SKIN: up to 7 days and it depends on the method and place of application. In addition, when you observe detachment and rolling of the tape ends, remove the product from the body surface. In order to extend the effect of one application, after removing the tape, apply a new piece of tape in the same place.

WARNINGS:

1. Single-use product.
2. In order to apply the tape, it is necessary to use cutting tools, therefore people who do not have the skills to use them should not use the tape on their own.
3. Do not use on open wounds and damaged or irritated skin.
4. If the tape is contaminated, replace it with a new one.
5. Use by children only under the supervision of an adult.
6. Tape for external use. It cannot be put in the mouth or other places not intended.
7. A user allergic to acrylic glue should be especially careful when using the tape.
8. If you feel irritation or discomfort, remove the tape immediately.
9. Used tape should be treated as potentially infectious and disposed of properly.
10. If you notice any disturbing symptoms, seek medical attention immediately.
11. In case of a serious medical incident, the Manufacturer and the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products should be informed about this fact.

STORAGE:

The product should be stored in a closed package at a temperature of 3-25 ° C, in a dry place. Protect against direct sunlight. Keep the product out of the reach of small children.

SIZE: width 5 cm, length 5 m

SYMBOLS USED ON THE PACKAGING:

	Manufacturer		Non-sterile
	Production date		Does not contain latex
	Lot number		Single-use only
	Catalogue number		Keep dry
	Medical device		Storage temperature range
	Refer to user manual		Use by date
	Keep away from direct sunlight		Do not use if packaging is damaged
		Authorized Representative in European Union	

Date of issue of the Instructions for Use: 04.2022

Novamed.pl S.A.

ul. Traktorowa 143 91-203 Łódź POLAND

NOVAMEDPL

Changzhou DL Medical & Health Equipment Co., Ltd

No. 528, Building A6, 88 Changhong Rd. Hutang Town
Wujin District, Changzhou, Jiangsu, China

Riomavix S.L.

Calle de Almansa 55, 1D, Madrid 28039 Spain

DL0302