medivon.

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AC3003 | AC2001 | AC1001
AC5001 | AC4001 | AC3002 | AC3001

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Be sure to read this instruction manual before operate and keep it where safe.

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Product description

The accessory is used in a precise spot treatment of a given area. To facilitate EMS / TENS therapy, the accessories have been adjusted so that it is easier to perform the procedure and make sure that the electrodes are placed in the right place for the therapy to bring the best results.

Use where you need

The specially tailored design allows you to use the accessory wherever you want, which gives more room for maneuver for physically active people and those who are often on the go. It is also ideal for athletes.

Instructions for use

1. The skin must always be clean, moist and free of lotion/oil.
2. Insert one end of the cable into the metal clips to attach the accessory, and the other end into the self-adhesive electrode. Make sure they are properly connected to ensure good performance.
3. Make sure the accessory is optimally positioned at the targeted therapy site. The adhesive electrode should be within 40 mm of the area covered by the accessory.
4. Make sure the TENS device is OFF. Connect the cable plug to the TENS device. Once connected, turn on the TENS device.
5. After treatment, turn off the TENS device first. Remove the connector and only then disconnect the accessory.

Caution 

- During the procedure, if you feel discomfort or unusual symptoms, stop the procedure immediately and contact your doctor
- Do not use on broken skin. If skin burn occurs, stop using and consult a doctor immediately.
- If the electrodes are damaged, do not use them.
- Use the stimulator in accordance with the instructions of the device and/or physician. For your own safety and convenience, turn off the stimulator before attempting to remove the accessory
- Unsupervised use of the appliance by children or infirm persons may be dangerous.
- If some people experience skin irritation due to the conductive silver fiber in the accessory, stop using it.

Cleaning and maintenance

To clean the product, wash it in warm water with a neutral detergent. Do not use hot water.

Thoroughly remove all traces of the washing solution in warm water, do not wring out. Wrap the accessory in a terry towel. Use a towel to remove excess water. Put the towel aside and let the accessory dry at room temperature. Store away from heat sources and sunlight.

Shenzhen Roundwhale Technology Co., Ltd.

202, 2/F, Building 27, Dafa Industrial Park, Longxi Community, Longgang District, Shenzhen, China

Declaration of Conformity

Manufacturer:

Shenzhen Roundwhale Technology Co., Ltd.

202, 2/F, Building 27, Dafa Industrial Park, Longxi Community, Longgang District, Shenzhen 518000, China

SRN:CN-MF-000021504

EU Representative:

Shanghai International Holding Corp. GmbH(Europe)

Address: Eifflstrasse 80, 20537 Hamburg, Germany

SRN:DE-AR-000000001

Product Name: NEUROLOGICAL PHYSIOTHERAPY DEVICES - ACCESSORIES

Product code / Catalogue number:

Gloves AC4001, Socks AC5001, Knee Belt AC 3002, Elbow Belt AC 3003,

Joint Belt AC3001, Back Belt AC2001, Abdomen belt AC1001,P50100 Electrodes.

Basic UDI-DI:697176360PAD0001HT

EMDN code: N010280- NEUROLOGICAL PHYSIOTHERAPY DEVICES - ACCESSORIES

Classification:Class I, rule 1 of Term 4 of Annex VIII of MDR 2017/745

Conformity assessment procedure: Annex II + Annex III of MDR 2017/745

Notified Body name	N/A
Notified Body Identification number	N/A
(EC) Certificate(s)	N/A
Start of CE-marking:	2023-04-26

We, the manufacturer, hereby declare that the above-mentioned products meet the provisions of the Regulation (EU) 2017/745 on Medical Devices (MDR). All supporting documentations are retained under the premises of the manufacturer.

Authorized Signature & Stamp (Vice President) Feb 17,2025

Application

This manual applies to the devices and accessories listed below:

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Shenzhen Roundwhale Technology Co., Ltd. | Add: 202, 2/F, Building 27, Dafa Industrial Park, Longxi Community, Longgang Street, Longgang District, Shenzhen, China.

Tel: +86-755-23212776 | Fax: +86-755-23212776 | info@roovjoy.com

Shanghai International Holding Corp.
GmbH (Europe) Address: Eiffestrasse
80, 20537 Hamburg, Germany
E-mail: shholding@hotmail.com

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Importer: Armare S.A. | ul Grzybowska 87, Warszawa, 00-844, Polska