Momini

Sniffley

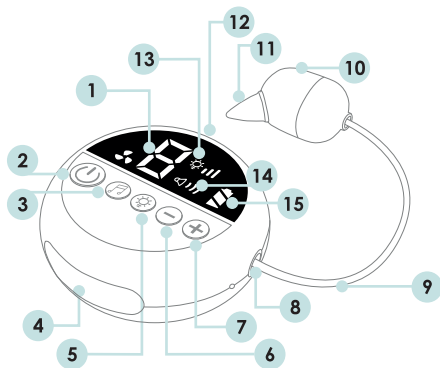
Automatic nasal aspirator
Automatyczny aspirator do nosa

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
Create happy beginnings

A

B

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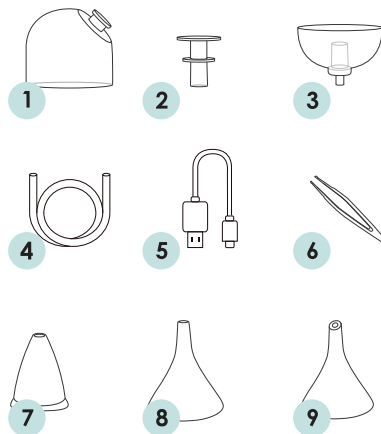01. SECURITY:

Before using the product, read this user manual and follow the recommendations contained therein when using the device.

02. DEVICE DESCRIPTION [PAGE 2 - A]:

1. Mode light / 2. Power button / 3. Music button / 4. Lamp / 5. Light adjustment button / 6. Suction power decrease button / 7. Suction power increase button / 8. Suction / 9. Tube / 10. Collection cup / 11. Silicone tip / 12. Charging port / 13. Brightness indicator / 14. Music/volume indicator / 15. Battery level indicator.

03. ACCESSORY DESCRIPTION [PAGE 2 - B]:

1. Lid of the slime container / 2. Diversion column / 3. Lower part of the slime container / 4. Tube / 5. USB charging cable / 6. Tweezers / 7. Conical tip / 8. Flat tip / 9. Angled tip.

Note: Before using the nasal aspirator, make sure you understand the features of each tip and choose the appropriate one based on your child's age to ensure comfortable and effective use.

Conical tip – shallow, most suitable for newborns
Flat tip – deep, for cleaning shallower areas of secretions
Angled tip – very deep: for children over 2 years of age.

Power button [page 2 - A2]: Long press turns the device on/off, A short press starts/stops suction.
Music button [page 2 - A3]: Short press changes the volume from 1 to 3. Adjustment sequence: 1 → 2 → 3 → off → on. Pressing it again changes the music.
Light button [page 2 - A5]: in off mode: long press turns the light on/off. A short press changes the brightness levels from 1 to 3.
Suction power reduction button [page 2 - A6]: Pressing "-" reduces the suction power in the range: 1 - 9.
Suction power increase button [page 2 - A7]: Pressing "+" increases the suction power in the range: 1 - 9.

04. DESCRIPTION OF USE OF THE DEVICE:

1. Open the collection cup and install the diversion column at the bottom.
2. Close the collection cup securely.
3. Select the suction tip and attach it to the collection cup.
4. Connect the device and collection cup using the tube.
5. Gently place the suction tip inside the child's nostril to begin mucus extraction.
6. Long press the "power" button to turn on the device (long press

again to turn it off).

7. Use the "-" or "+" button to adjust suction power as needed.
8. If desired, activate soothing music using the "melody" button.
9. After clearing the child's nose, turn off the device and detach the collection cup and tube.
10. Empty the collection cup, wash all components except the main unit (which is not waterproof).
11. After cleaning, dry the device with a soft cloth and store it in a cool, dry place away from direct sunlight.

05. TIPS:

For better results, if the mucus in the child's nose is dry, it is recommended to use saline solution to moisten it before using the electric nasal aspirator.

06. CLEANING & STORAGE:

1. Always turn off the device after use.
2. Disconnect and wash the suction tip, collection cup, and tube in water.
3. Keep all components and the device clean and dry.
4. The suction tips and collection cup can be sterilized in boiling water or steam but not in a microwave.
5. The main unit is not waterproof – do not wash it with water or expose it to high temperatures. Clean only with a damp cloth.
6. Avoid storing the product in direct sunlight or humid environments.

07. SAFETY MEASURES:

1. WARNING: During use, the child's mouth should remain open to allow proper airflow and prevent discomfort from negative pressure.
2. Avoid blocking the child's other nostril while using the aspirator to prevent discomfort from negative pressure in the head.
3. If the collection cup is half full, it is recommended to empty it before continuing use to prevent mucus from being sucked into the pump and potentially damaging the device.

08. PRECAUTIONS:

1. Do not turn on the nasal aspirator while charging.
2. The device is intended for use by adults only.
3. Do not expose the device to high temperatures, high humidity or direct sunlight.
4. The device is specially designed to remove mucus from the nose. It should not be used for any other purposes!
5. After each use, wash the elements intended for this purpose with water and then dry them thoroughly.
6. The main unit of the device is not waterproof - clean it with a dry or damp cloth, without direct contact with water.
7. Do not use corrosive agents to clean the device.

8. Charge the battery to 100% every 2 months if you don't use it for a long time.
9. Do not use excessive force when placing the suction tip into the nasal cavity.
10. During use, the child/infant should not hold their breath, close their mouth or swallow anything. Excessive vacuum pressure may cause an imbalance in ear pressure.
11. If your child/infant experiences any discomfort while using the device, stop using it immediately and consult a doctor.
12. If your nose has sores, inflammation or signs of bleeding, do not use the nasal aspirator.
13. Do not use the product as a toy for children.
14. Do not use the device if it is damaged.
15. Do not repair, disassemble or modify the device yourself. Any irregularities should be assessed by the service.

Technical specifications:

Charging voltage: DC 5V/1A
Maximum power: max. 5.5W
Nominal voltage: 3.7V/2500mAh
Charging time: about 3 hours
Intensity levels: 9 levels
Suction force: 65kPa ± 3kPa
Product net weight: 235g
Product size: 95mm × 45mm

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[EN] Rules for handling electrical and electronic equipment: Waste electrical and electronic equipment as well as batteries and accumulators cannot be mixed with other waste. This is indicated by the crossed out bin symbol on the product packaging. Electrical and electronic equipment as well as batteries and accumulators may contain hazardous substances, mixtures and components harmful to the environment and human health. Improper handling or damage may result in damage to health or environmental pollution. Proper household handling of waste electrical and electronic equipment and batteries and accumulators positively contributes to the reuse and recovery, including recycling, of waste equipment. Waste electrical and electronic equipment as well as batteries and accumulators can be returned to local selective waste collection points. The distributor is obliged to collect used equipment from households free of charge at the point of sale, provided that the used equipment is of the same type and performed the same functions as the equipment sold. When delivering equipment intended for households to the buyer, the distributor is obliged to collect used equipment

from households free of charge at the place of delivery of the equipment, provided that the used equipment is of the same type and performed the same functions as the delivered equipment. A distributor running a retail unit with a sales area within the meaning of Art. 2 point 19 of the Act of 27 March 2003 on spatial planning and development of at least 400 m² devoted to the sale of equipment intended for households, is obliged to accept waste equipment from households, free of charge, in this unit or in its immediate vicinity, none of the external dimensions exceeds 25 cm, without the need to purchase new household equipment. The product was introduced to the market on August 13, 2005 r.

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[EN] Summary declaration of conformity: The manufacturer declares, under its sole responsibility, that the device complies with Directive 2014/30/EU of the European Parliament and of the Council of 26 February 2014 on the harmonization of the laws of the Member States relating to electromagnetic compatibility and Directive 2011/65/EU of the European Parliament and of the Council of 8 June 2011 on the restriction of the use of certain hazardous substances in electrical and electronic equipment, together with Directive Commission delegated act (EU) 2015/863 of 31 March 2015 amending Annex II to Directive 2011/65/EU of the European Parliament and of the Council as regards the list of restricted substances. The full content of the declaration of conformity is available at www.momini.eu/deklaracje

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EN | More information about the product is available on the manufacturer's website at:

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