# BRAUN

## ExactFit™ 1E

BUA4000

# Braun ExactFit™ 1E

BUA4000

Upper arm blood pressure monitor

GB	English.....	4
CZ	Česky .....	11
DE	Deutsch .....	18
DK	Dansk.....	26
ES	Español .....	33
FI	Suomi .....	41
FR	Français.....	49
HR	Hrvatski.....	57
IT	Italiano .....	64
NL	Nederlands.....	72
NO	Norsk.....	80
PL	Polski.....	87
RO	Română.....	95
SA	اللغة العربية.....	109
SE	Svenska.....	110
SI	Slovenčina .....	118
TR	Türkçe.....	125

	1	2	3	4	5	6	7
EN	LCD display	Power (start/stop) button	Hose port	Battery compartment	Connector	Arm cuff	Air hose
CZ	LCD displej	Tlačítko vypínače (zapnutí/vypnutí)	Port hadičky	Prostor na baterie	Konektor	Manžeta	Vzduchová hadička
DE	LCD-Display	Einschalttaste (Start/Stopp)	Schlauch-Port	Batteriefach	Anschluss	Armmanschette	Luftschlauch
DK	LCD-display	Tænd/sluk-knapn	Sлангетилслутнинг	Batterirum	Konnektor	Manchet	Luftslange
ES	Pantalla LCD	Botón de encendido/apagado	Conexión de tubo	Compartimento para las pilas	Conector	Brazalete	Tubo de aire
FI	Nestekidenäyttö	Virtapainike (käynnistys/pysäytys)	Letkuliitintä	Paristolokero	Liitin	Käsivarsimansetti	Ilmaletku
FR	Écran LCD	Bouton d'alimentation (marche/arrêt)	Branchemet du tuyau	Compartiment à piles	Raccord	Brassard	Tuyau d'arrivée d'air
HR	LCD zaslon	Gumb za uključivanje/isključivanje	Priklučak za crijevo	Pretinac za baterije	Priklučak	Manžeta za ruku	Cijev za zrak
IT	Display LCD	Pulsante di alimentazione (avvio/arresto)	Porta tubo flessibile	Vano batteria	Connettore	Bracciale	Tubo dell'aria
NL	Lcd-display	Aan-uiknop (start/stop)	Slangaansluiting	Batterijvakje	Connector	Manchet	Luchtslang
NO	LCD-skjerm	Strøm (start/stopp) knapp	Slangeåpning	Batterirom	Uttak	Armmansjett	Luftslange
PL	Wyświetlacz LCD	Przycisk zasilania (start/stop)	Port przewodu	Komora baterii	Złącze	Mankiet	Przewód powietrzny
RO	Afișaj LCD	Butonul de alimentare (pornire/oprire)	Orificiu pentru furtun	Compartimentul pentru baterii	Conector	Manșetă pentru braț	Furtun de aer
SA	LCD شاشة	(زر الطاقة) بدء/إيقاف	منفذ الخرطوم	حاجبية البطارية	الموصل	كفة النزاع	خرطوم الهواء
SE	LCD-skärm	Strömknapp (start/stopp)	Slangport	Batterifack	Anslutning	Armmanschett	Luftslang
SI	Prikazovalnik LCD	Gumb za vklop (start/stop)	Vhod za cev	Predal za baterije	Prikluček	Manšeta	Cev za zrak
TR	LCD ekran	Güç (baslat/durdur) düğmesi	Hortum bağlantı noktası	Pil bölmesi	Konektör	Kol manşeti	Hava hortumu

# GB English

## Intended use and contraindications

This device is intended to be used by adults (+18 years old) to measure people's blood pressure, that includes systolic, diastolic and heartbeat pulse rate. This device is to be used for intermittent measurements only. For home usage only.

This device has not been tested for and hence is not intended for neonatal and infant subjects, pregnant, including pre-eclamptic users.

This blood pressure monitor is not intended for clinical use in a professional environment and is for home usage only. The use of this blood pressure monitor is not intended as a substitute for a consultation with your doctor.

## What you should know about blood pressure

Blood pressure constantly changes throughout the day. It rises sharply in the early morning and declines during the late morning. Blood pressure rises again in the afternoon and finally drops to a low level at night. Also, it may vary in a short period of time. Therefore, readings from successive measurements can fluctuate.

This device will display your blood pressure reading referring to two values: systolic and diastolic, through the principle of pressure oscillation. The systolic blood pressure (the upper number) indicates how much pressure your blood is exerting against your artery walls when the heart beats. The diastolic blood pressure (the lower number) indicates how much pressure your blood is exerting against your artery walls while the heart is resting between beats. The pulse value is also displayed after each measurement.

Blood pressure measured in a doctor's office only provides a momentary value. Repeated measurements at home better reflect one's actual blood pressure values under everyday conditions.

Moreover, many people have a different blood pressure when they measure at home, because they tend to be more relaxed than when in the doctor's office. Regular blood pressure measurements taken at home can provide your doctor with valuable information on your normal blood pressure values under actual "everyday" conditions.

The ESH (European Society of Hypertension) and WHO (World Health Organisation) have set up the following standard blood pressure values when measured with a resting pulse at home:

Blood pressure (mmHg)	Normal values	Grade 1 mild hypertension	Grade 2 moderate hypertension	Grade 3 severe hypertension
SYS = systole (upper value)	up to 134	135-159	160-179	$\geq 180$
DIA = diastole (lower value)	up to 84	85-99	100-109	$\geq 110$

<sup>1</sup> European Society of Hypertension

## Safety information and important precautions

To ensure accurate measuring results, read the complete instructions for use carefully. Measurement errors may occur if the original cuff is not used.

People suffering from cardiac arrhythmia, vascular constriction, arteriosclerosis in the extremities, diabetes or users of cardiac pacemakers should consult their doctor before measuring their blood pressure themselves. Deviations in blood pressure values may occur in such cases.

If your blood pressure reading is in the severe hypertension range with values

of 180 mmHg systolic and above, and/or 110 mmHg diastolic and above; and you find you have one or two readings in this range - please see your doctor straight away.

If you are under medical treatment or taking any medication, please consult your doctor first.

Avoid compression or restriction of the connection tubing. Continuous cuff pressure due to the connection tubing kinking can affect blood flow and cause possible injury to the user.

It is not recommended to use this device on the arm of the same side of the body on which the user had a mastectomy.

Measuring blood pressure too frequently can cause injury to the user due to blood flow interference.


It is not recommended to apply the cuff to a limb on which there is intravascular access or on which an arterio-venous (A-V) shunt is present. This can cause temporary interference of blood flow and potential injury to the user.

DO NOT apply the cuff over a wound as this can cause further injury.

It is not recommended to use the device while simultaneously using other medical electrical (ME) equipment on the same limb.

## Inserting batteries

- Use alkaline batteries, type AA 1.5V as supplied with this product.
- Remove the battery compartment cover and insert four batteries with correct polarity (see symbol in the battery compartment).
-  Only discard empty batteries. They should not be disposed of in the household waste, but at appropriate collection sites or at your retailer.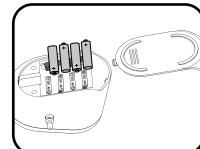

## Key rules for accurate blood pressure measurement

Always take readings at the same time of day, under the same conditions.

Do not take a measurement within 30 minutes of smoking, drinking coffee or tea or any form of exertion. These factors will influence the measurement.

Ensure at least 5 minutes of quiet rest before measurements. If possible, it is recommended to rest 10 minutes before taking the measurement. Try to always use the same resting time before taking a measurement.

Always measure on the same arm. The left arm is recommended. Do not move or speak. Sit comfortably, keep your legs uncrossed and feet flat on the floor, back and arm supported.

Place the cuff in direct contact with the skin. Do not wrap the cuff over a jacket, sweater or shirt.

If you want to take multiple measurements, wait approximately 5 minutes before repeating a measurement.

## Applying the arm cuff

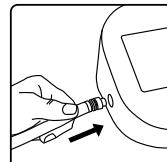
Fig. 1

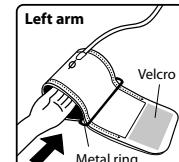
Fig. 2

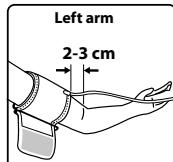
Fig. 3

1. Plug the air hose into the connector (Fig. 1).
2. The cuff is located correctly when the velcro is on the outside of the cuff and the metal ring does not touch the skin (Fig. 2).
3. Put your arm through the cuff loop. The bottom of the cuff should be approximately 2-3 cm above elbow (Fig. 3). The tube should lie over the brachial artery on the inside of the arm (Fig. 4).

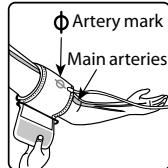
Fig. 4

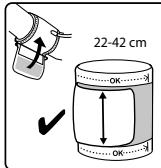
Fig. 5

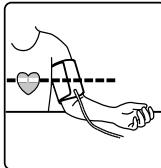
Fig. 6

4. Pull the cuff so that the top and bottom edges are tightened around your arm. Make sure not to wrap the cuff very tightly. Allow space for one finger to fit between the cuff and the arm.
5. When the cuff is positioned properly, press the Velcro firmly against the pile side of the cuff.
6. This cuff is suitable for use if the arrow heads fall within the OK range on the other side of the cuff when the cuff is tightened around your arm (Fig. 5).
7. Sit on a chair and place your arm on the table so that the cuff is at the same level as your heart (Fig. 6).

## Taking a measurement

8. Sit down with straight back with your feet flat on the floor.
  9. Wrap the cuff snugly around your arm (see "Applying the arm cuff" section above, Fig. 3-6).
  10. Do not move the unit during measurement, or the proper measurement will not be achieved.
  11. Press power button to begin. The last results will be shown on the screen (M reading). The measurement starts automatically. While measurement is in progress, do not move or speak.
- Note: If necessary to interrupt a measurement, press the power button at any time. The device immediately cancels the measurement, lowers cuff pressure and automatically turns off.
12. The cuff will first inflate and then deflate. At the end of the measurement the reading is displayed:

Upper (systolic) value  
Lower (diastolic) value  
Pulse

13. Press the power button to turn the product off. Otherwise the product will turn off automatically after 1 minute.
14. Test results will be saved into the memory automatically (see memory function below).

This device is not intended as a substitute for regular check-ups by your doctor, please continue to visit your doctor on a regular basis for a professional reading.


## Memory function

Your ExactFit™ 1E blood pressure monitor will automatically store the latest measurement. Storing is done automatically after the completed measurement.

## Battery charge indicator

### Batteries discharged—replacements required

- 1 When the batteries are fully discharged, the empty battery symbol is displayed. You cannot take any further measurement and must replace the batteries. Refer to the section called "Inserting batteries".

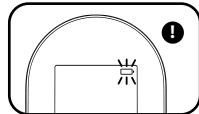## Storage and cleaning

- Do not expose device to either extreme temperatures, humidity, dust or direct sunlight.
- The cuff contains a sensitive air-tight inflatable bladder. Handle this carefully and avoid all types of straining through twisting or buckling.
- Clean the device with a soft dry cloth. Stains on the cuff can be carefully removed with a damp cloth. Do not use gasoline, thinners or similar solvents. Do not wash or dry clean the cuff. Do not store the product in a place exposed to direct sunlight or high humidity.
- Do not drop the device or treat it roughly in any way. Avoid strong vibrations.
- NEVER open the device! This will make the manufacturer's warranty invalid and could adversely affect the device's accuracy!

## Calibration

This product has been calibrated at the time of manufacture. If used according to the instructions for use, periodic recalibration is not required. If at any time you question the measuring accuracy, please contact our service representative (see last page for contact information).

## What to do if ...

Error	Description	Potential cause and remedy
	Batteries are low	Get ready to replace batteries soon with new ones.
	Batteries are fully discharged	Insert new batteries or check that batteries are correctly inserted.
«Err0»	Measuring error	The pulse signals on the cuff are too weak. Re-position the cuff and repeat the measurement.
«Err1»	Pumping error	The cuff is not fastened properly. Re-position the cuff and repeat the measurement.
«Err2»	Out of range results error	Relax for a while and repeat the measurement. Make sure the cuff is correctly positioned on your arm.
«Err3»	Pump time error	The cuff is not fastened properly. Re-fit the cuff and take a measurement again.

Error	Description	Potential cause and remedy
«Err254»	Device internal error	Remove and then reinsert the batteries. If the problem persists, contact customer support.
«Err255»	Calibration error	Remove and then reinsert the batteries. If the problem persists, contact customer support.
Readings from repeated measurements differ considerably.	Blood pressure is a fluctuating value. For healthy adults, deviations of 10 to 20 mmHg are possible.	Reminder: always use the same arm for measurements!
	Cuff is not properly fitted.	Ensure that the cuff is fitted according to the "Applying the arm cuff" section.
	Readings were not consistently taken at heart level.	For each measurement, make sure that the cuff is at heart level.
	Talking, coughing, laughing, moving etc. when measuring will influence the reading.	While measuring, relax, keep still, do not move or speak.
Blood pressure values measured at your doctor's office differ from your measurements.	Doctor visits can frequently cause anxiety.	Record the daily development of the measured values and consult your doctor.
Display remains blank, or is unusual, when instrument is switched on.	Batteries are not installed properly.	Check batteries for correct polarity.
Device frequently fails to measure blood pressure values, or values measured are too low (or too high).	Cuff may not be properly positioned.	Ensure that the cuff is fitted according to the "Applying the arm cuff" section.

## Specifications

Model:	BUA4000
Operation principle:	Oscillometric method
Display:	Liquid crystal display
Range of measurement:	Rated Cuff pressure : 0 mmHg~280mmHg SYS: 60~240mmHg; DIA:40~190mmHg
Measurement pressure:	Pulse 40~199 beats/minute pressure: +/-3 mmHg (cuff pressure)
Laboratory accuracy:	+/- 5% max
Pulse rate :	200 measurements
Battery life:	Approximately 1 minute of inactivity
Auto power off:	4 batteries, 1 arm cuff with tube, instruction manual
Accessories:	Automatic 4 batteries, type AA 1.5V 5 °C to 40 °C (41 °F to 104 °F) 15% to 90% relative humidity, non-condensing
Inflation:	-25 °C to 55 °C (-13 °F to 131 °F)
Power supply:	15% to 95% relative humidity, non-condensing
Operating temperature:	70~106 kPa
Operating humidity:	Universal size (22-42 cm)
Storage/transport temperature:	5 years
Storage/transport humidity:	2 years
Operate and store atmospheric pressure:	If device is not used within specified temperature and humidity ranges the technical accuracy cannot be ensured.
Cuff:	
Service life:	
Warranty:	

If device is not used within specified temperature and humidity ranges the technical accuracy cannot be ensured.  
**IP21** Protected against solid foreign objects of 12.5 mm diameter and greater. Protected against vertically falling water drops.

Type BF applied part

Operating temperature

See instructions for use

Storage temperature

Caution

Manufacturer

Operating humidity

Storage humidity

Batch code

Serial number

Medical Device

Authorized representative in the European Community

Date of manufacture

Catalogue number

Artery mark

Subject to change without notice.  
 Supply connection: internally powered equipment.  
 Mode of operation: continuous operation.

0297 This medical device carries the CE mark and is manufactured in conformity with RoHS Directive 2011/65/EU and other applicable directive and/or regulations as specified in the EU Declaration of Conformity.

MEDICAL ELECTRICAL EQUIPMENT needs special precautions regarding EMC. NOTE: Do not use this device in the presence of electromagnetic or other interference outside the normal range specified in EN 60601-1-2. For detailed description of EMC requirements please visit [www.HelenofTroy.com/emea-en/support](http://www.HelenofTroy.com/emea-en/support) or contact service center. Portable and mobile RF communications equipment can affect MEDICAL ELECTRICAL EQUIPMENT.

## Inquiry

In Europe, any serious incident (e.g. death, life-threatening injury, surgical intervention, ...) that has occurred in relation to this blood pressure monitor should be reported to Kaz Europe (see Helen of Troy Consumer Service Numbers) and to the competent authority of your member state.

The LOT and SN of your device are printed on the rating label on the back of the product. The first 3 numeric digits of the LOT No. represents the day of the year of manufacture. The next 2 numeric digits represent the last two numbers of the calendar year of manufacture and the letters at the end designate the manufacturer of the product. E.g. LOT 15614onb, this product was made on day 156, year 2014, at manufacturer identifier onb.

Please do not dispose of the product in the household waste at the end of its useful life. Disposal can take place at your local retailer or at appropriate collection points provided in your country.

## Warranty

Please read all instructions before attempting to use this device. Please retain the receipt as proof of and date of purchase. The receipt must be presented when making any claim within the relevant warranty period. Any claim under the warranty will not be valid without a proof of purchase.

Your device is warrantied for two years (2 years) from date of purchase.

This warranty covers defects in materials or workmanship that occur under normal use; defective devices meeting these criteria will be replaced free of charge.

The warranty DOES NOT cover defects or damage resulting from abuse or failure to follow the user instructions. The warranty becomes void if the device is opened, tampered with, or used with non-Braun branded parts or accessories, or if repairs are undertaken by unauthorised persons.

Accessories and consumables are excluded from any warranty.

For support requests, please visit [www.HelenofTroy.com/emea-en/support](http://www.HelenofTroy.com/emea-en/support) or find service contact information at the end of this owner's manual.

This warranty is applicable to Europe, Middle East and Africa only.

UK Only: This does not affect your consumer statutory rights.

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REF BUA4000EU

[www.BraunHealthcare.com](http://www.BraunHealthcare.com)

Certain trademarks used under license from  
The Procter & Gamble Company or its affiliates.

ExactFit™ is a trademark of Helen of Troy  
Limited and/or its affiliates.

Helen  
of Troy

 Kaz Europe Sàrl  
A Helen of Troy Company  
Q-Center, Chaux 4  
CH-1030 Bussigny - Switzerland

0297

Made and  
printed in China

© 2023 All rights reserved.  
P/N A007048R3  
13JUN23

**Brand:** Braun  
**Category:** BPM  
**Model:** **BUA4000EU**  
**Artwork Part #:** A007048R3  
**Die Line Part #:** N/A  
**Subject:** Owner's Manual  
**Region:** EMEA  
**Flat Size:** MM: W180 x H160  
**Folded Size:** MM: W90 x H160  
**Scale:** 1/1  
**Material:** 40# gloss or matte text  
Minimum 80# gloss or matte text (cover)  
**Page Count:** 136  
**Revision:** 1  
**Date:** **13JUN23**  
**Release Date:** **26JUN23**  
**Re-release Date:**  
**Re-release Date:**

**Colors:**

Dielines (Do not print)

Spot Colors

Cyan  
0%Magenta  
0%Yellow  
0%Black  
100%PMS  
XXXXPMS  
XXXX**Special Instructions:****Quality Requirement of Artwork and Quality Clarification Process of Artwork Printing:  
Meet Eng-QS-06&02**