BRAUN

ExactFit™ 2

BUA4050

Braun ExactFit™ 2
BUA4050
Upper arm blood pressure monitor

<input type="checkbox"/> GB	English.....	2
<input type="checkbox"/> CZ	Česky	11
<input type="checkbox"/> DE	Deutsch	20
<input type="checkbox"/> DK	Dansk.....	30
<input type="checkbox"/> ES	EspaÑol	39
<input type="checkbox"/> FI	Suomi	49
<input type="checkbox"/> FR	Français.....	58
<input type="checkbox"/> HR	Hrvatski.....	68
<input type="checkbox"/> IT	Italiano	78
<input type="checkbox"/> NL	Nederlands.....	88
<input type="checkbox"/> NO	Norsk.....	98
<input type="checkbox"/> PL	Polski.....	107
<input type="checkbox"/> RO	Română.....	117
<input type="checkbox"/> SA	اللغة العربية.....	135
<input type="checkbox"/> SE	Svenska.....	136
<input type="checkbox"/> SI	Slovenščina	145
<input type="checkbox"/> SK	Slovenčina	154
<input type="checkbox"/> TR	Türkçe.....	163

GB English

Product description

- 1** LCD display
- 2** Colour coded guidance
- 3** Power (start/stop) button (↓)
- 4** Memory button **M**
- 5** Hose port
- 6** Battery compartment
- 7** Connector
- 8** Arm cuff
- 9** Air hose

Intended use and contraindications

This device is intended to be used by adults (+18 years old) to measure people's blood pressure, that includes systolic, diastolic and heartbeat pulse rate. This device is to be used for intermittent measurements only. For home usage only.

This device has not been tested for and hence is not intended for neonatal and infant subjects, pregnant, including pre-eclamptic users.

This blood pressure monitor is not intended for clinical use in a professional environment and is for home usage only. The use of this blood pressure monitor is not intended as a substitute for a consultation with your doctor.

What you should know about blood pressure

Blood pressure constantly changes throughout the day. It rises in the early morning and declines during the late morning. Blood pressure rises again in the afternoon and finally drops to a low level at night. It may vary in a short period of time. Therefore, readings from successive measurements can fluctuate.

This device will display your blood pressure reading referring to two values: systolic and diastolic, through the principle of pressure oscillation. The systolic blood pressure (the upper number) indicates how much pressure your blood is exerting against your artery walls when the heart beats. The diastolic blood pressure (the lower number) indicates how much pressure your blood is exerting against your artery walls while the heart is resting between beats. The pulse value is also displayed after each measurement.

Blood pressure measured in a doctor's office only provides a momentary value. Repeated measurements at home better reflect one's actual blood pressure values under everyday conditions.

Moreover, many people have a different blood pressure when they measure at home, because they tend to be more relaxed than when in the doctor's office. Regular blood pressure measurements taken at home can provide your doctor with valuable information on your normal blood pressure values under «everyday» conditions.

The ESH (European Society of Hypertension) and WHO (World Health Organisation) have set up the following standard blood pressure values when measured with a resting pulse at home:

Blood pressure (mmHg)	Normal values	Grade 1 mild hypertension	Grade 2 moderate hypertension	Grade 3 severe hypertension
SYS = systole (upper value)	up to 134	135-159	160-179	≥180
DIA = diastole (lower value)	up to 84	85-99	100-109	≥110

¹ European Society of Hypertension

Safety information and important precautions

To ensure accurate measuring results, read the complete instructions for use carefully. Measurement errors may occur if the original cuff is not used.

People suffering from cardiac arrhythmia, vascular constriction, arteriosclerosis in the extremities, diabetes or users of cardiac pacemakers should consult their doctor before measuring their blood pressure themselves. Deviations in blood pressure values may occur in such cases.

If your blood pressure reading is in the severe hypertension range with values of 180 mmHg systolic and above, and/or 110 mmHg diastolic and above; and you find you have one or two readings in this range, please see your doctor straight away.

If you are under medical treatment or taking any medication, please consult your doctor first.

Avoid compression or restriction of the connection tubing. Continuous cuff pressure due to the connection tubing kinking can affect blood flow and cause possible injury to the user.

It is not recommended to use this device on the arm of the same side of the body on which the user had a mastectomy.

Measuring blood pressure too frequently can cause injury to the user due to blood flow interference.

It is not recommended to apply the cuff to a limb on which there is intravascular access or on which an arterio-venous (A-V) shunt is present. This can cause temporary interference of blood flow and potential injury to the user.


DO NOT apply the cuff over a wound as this can cause further injury.

It is not recommended to use the device while simultaneously using other medical electrical (ME) equipment on the same limb.

Inserting batteries

- Use alkaline batteries, type AA 1.5V as supplied with this product.
- Remove the battery compartment cover and insert four batteries with correct polarity (see symbol in the battery compartment).

 Only discard empty batteries. They should not be disposed of in the household waste, but at appropriate collection sites or at your retailer.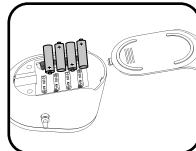

Key rules for accurate blood pressure measurement

Always take readings at the same time of day, under the same conditions.

Do not take a measurement within 30 minutes of smoking, drinking coffee or tea or any form of exertion. These factors will influence the measurement.

Ensure at least 5 minutes of quiet rest before measurements; if possible, 10 minutes of rest are recommended. Try to always use the same resting time before taking a measurement.

Always measure on the same arm. The left arm is recommended. Do not move or speak. Sit comfortably, keep your legs uncrossed and feet flat on the floor, back and arm supported.

Place the cuff in direct contact with the skin. Do not wrap the cover over clothing.

If you want to take multiple measurements, wait approximately 5 minutes before repeating.

Applying the arm cuff

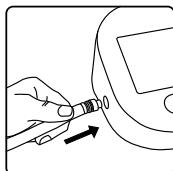
Fig. 1

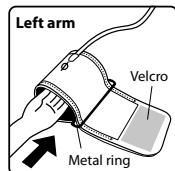
Fig. 2

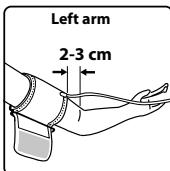
Fig. 3

1. Plug the air hose into the connector (Fig. 1).

2. The cuff is located correctly when the velcro is on the outside of the cuff and the metal ring does not touch the skin (Fig. 2).

3. Put your arm through the cuff loop. The bottom of the cuff should be approximately 2-3 cm above elbow (Fig. 3). The tube should lie over the brachial artery on the inside of the arm (Fig. 4).

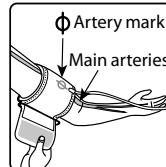
Fig. 4

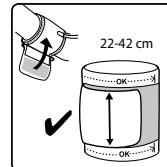
Fig. 5

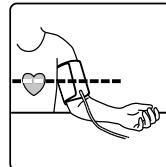
Fig. 6

4. Pull the cuff so that the top and bottom edges are tightened around your arm, make sure not to wrap the cuff very tightly. Allow space for one finger to fit between the cuff and the arm.
5. When the cuff is positioned properly, press the velcro firmly against the pile side of the cuff.
6. This cuff is ready for use if the arrow heads fall within the range on the other side of the cuff when the cuff is tightened around your arm (Fig. 5).
7. Sit on a chair and place your arm on the table so that the cuff is at the same level as your heart (Fig. 6).

Taking a measurement

8. Sit down with straight back with your feet flat on the floor.
9. Ensure the cuff is wrapped snugly (per the "Applying the arm cuff" section above, Fig. 3-6).
10. Do not move the unit during measurement, or the proper measurement will not be achieved.
11. Press power button  to begin. The last results will be shown on the screen (M reading). The measurement starts automatically. While measurement is in progress, do not move or speak.
- Note: If necessary to interrupt a measurement, press the power button  at any time. The device immediately cancels the measurement, lowers cuff pressure and automatically turns off.
12. The cuff will first inflate and then deflate. At the end of the measurement the reading is displayed:
 - Upper (systolic) value
 - Lower (diastolic) value
 - Pulse
13. Press the power button  to turn the product off. Otherwise the product will turn off automatically after 1 minute.
14. Test results will be saved into the memory automatically (see memory function below).

This device is not intended as a substitute for regular check-ups by your doctor, please continue to visit your doctor on a regular basis for a professional reading.

Irregular heartbeat detector

Talking, moving, shaking or an irregular pulse during the measurement can result in the appearance of this icon. Usually this is not a cause for concern, however if the symbol appears often, we recommend you seek medical advice. The device does not replace a cardiac examination, but it could help identifying pulse irregularities to be discussed with your doctor.

- ♡ This symbol at the right-hand side of pulse reading indicates a regular heartbeat.
- ⌚ This symbol at the right-hand side of pulse reading indicates that pulse irregularities were detected during the measurement. This irregular heartbeat can be caused simply by talking, moving or shaking during a measurement. In this case, the result may deviate from your normal blood pressure – repeat the measurement. Normally this is not cause for concern. However if the symbol appears frequently, and these guidelines are strictly followed, seek medical advice.

Colour coded guidance

The instant read colour indicator follows the ESH and WHO recommendations for interpreting readings and is described in the table below.

	Green: normal blood pressure	Your blood pressure reading is normal, and values are below 135 mmHg systolic and below 85 mmHg diastolic.
	Yellow: mild hypertension	Your blood pressure reading is in the mild hypertension range with values between 135-159 mmHg systolic and/or 85-99 mmHg diastolic.
	Orange: moderate hypertension	Your blood pressure reading is in the moderate hypertension range with values between 160-179 mmHg systolic and/or 100-109 mmHg diastolic.
	Red: severe hypertension	Warning Your blood pressure reading is in the severe hypertension range with values of 180 mmHg systolic and above and/or 110 mmHg diastolic and above. If you get readings in this range, consult your doctor straight away.

Blood pressure readings will be automatically saved in the device memory. Results are displayed for approx. 60 seconds, then device will automatically shut-off.

Memory function

Your blood pressure monitor can store the latest 30 readings.

Storing measurement data

After each blood pressure measurement, the systolic pressure, diastolic pressure, pulse rate will be automatically stored. Memory #01 is always the most recent one.

Once the memory is full, the oldest values will be overwritten.

Press memory button **M** to review the stored data. The last memory data (sys/dia/pul), IHB and WHO indicator will show on LCD as "01","02","03"..."10".


Press and hold memory button to scroll through previous stored data.

Press and hold power and memory button together for 3 seconds to clear the memory of the device of all pre-existing information. This device is not intended as a substitute for regular check-ups by your doctor, please continue to visit your doctor on a regular basis for a professional reading.

Battery charge indicator

Batteries discharged—replacements required

- ➊ When the batteries are fully discharged, the empty battery symbol is displayed. You cannot take any further measurement and must replace the batteries. Refer to the section "Inserting batteries".

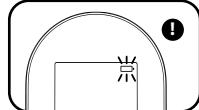Storage and cleaning

- Do not expose device to either extreme temperatures, humidity, dust or direct sunlight.
- The cuff contains a sensitive air-tight inflatable bladder. Handle this carefully and avoid all types of straining through twisting or buckling.
- Clean the device with a soft dry cloth. Stains on the cuff can be carefully removed with a damp cloth. Do not use gasoline, thinners or similar solvents. Do not wash or dry clean the cuff. Do not store the product in a place exposed to direct sunlight or high humidity.
- Do not drop the device or treat it roughly in any way. Avoid strong vibrations.
- NEVER open the device! This will make the manufacturer's warranty invalid and could adversely affect the device's accuracy!

Calibration

This product has been calibrated at the time of manufacture. If used according to the instructions for use, periodic recalibration is not required. If at any time you question the measuring accuracy, please contact our service representative (see last page for contact information).

What to do if ...

Error	Description	Potential cause and remedy
	Batteries are low	Get ready to replace batteries soon with new ones.
	Batteries are fully discharged	Insert new batteries or check that batteries are correctly inserted.
«Err0»	MeasuringError	The pulse signals on the cuff are too weak. Re-position the cuff and repeat the measurement.
«Err1»	PumpingError	The cuff is not fastened properly. Re-position the cuff and repeat the measurement.

Error	Description	Potential cause and remedy
«Err2»	Out of range resultsError	Relax for a while and repeat the measurement. Make sure the cuff is correctly positioned on your arm.
«Err3»	Pump timeError	The cuff is not fastened properly. Re-fit the cuff and take a measurement again.
«Err254»	Device internalError	Remove and then reinsert the batteries. If the problem persists, contact customer support.
«Err255»	CalibrationError	Remove and then reinsert the batteries. If the problem persists, contact customer support.
Readings from repeated measurements differ considerably.	Blood pressure is a fluctuating value. For healthy adults, deviations of 10 to 20 mmHg are possible.	Reminder: always use the same arm for measurements!
	Cuff is not properly fitted.	Ensure that the cuff is fitted according to the "Applying the arm cuff" section.
	Readings were not consistently taken at heart level.	For each measurement, make sure that the cuff is at heart level.
	Talking, coughing, laughing, moving etc. when measuring will influence the reading.	While measuring, relax, keep still, do not move or speak.
Blood pressure values measured at your doctor's office differ from your measurements.	Doctor visits can frequently cause anxiety.	Record the daily development of the measured values and consult your doctor.
Display remains blank, or is unusual, when instrument is switched on.	Batteries are not installed properly.	Check batteries for correct polarity.
Device frequently fails to measure blood pressure values, or values measured are too low (or too high).	Cuff may not be properly positioned.	Ensure that the cuff is fitted according to the "Applying the arm cuff" section.

Specifications

Model:	BUA4050
Operation principle:	Oscillometric method
Display:	Liquid crystal display
Range of measurement:	Rated Cuff pressure : 0 mmHg~280mmHg
Measurement pressure :	SYS: 60~240mmHg; DIA:40~190mmHg
Laboratory accuracy:	Pulse rate : Pulse: +/- 3 mmHg (cuff pressure) +/- 5% max
Pulse rate :	200 measurements
Battery life:	Approximately 1 minute of inactivity
Auto power off:	4 batteries, 1 arm cuff with tube, instruction manual
Accessories:	Automatic
Inflation:	4 batteries, type AA 1.5V
Power supply:	5°C to 40 °C (41 °F to 104 °F)
Operating temperature:	15% to 90% relative humidity, non-condensing
Operating humidity:	-25 °C to 55 °C (-13 °F to 131 °F)
Storage/transport temperature:	15% to 95% relative humidity, non-condensing
Storage/transport humidity:	70~106 kPa
Operating/storage/transport atmospheric pressure:	Universal size (22-42 cm)
Cuff:	5 years
Service life:	2 years
Warranty:	

If device is not used within specified temperature and humidity ranges the technical accuracy cannot be ensured.

IP21 Protected against solid foreign objects of 12.5 mm diameter and greater. Protected against vertically falling water drops.

Type BF applied part

+45°C
-40°C

See instructions for use

-25°C
+55°C

Caution

Manufacturer

Date of manufacture

REF Catalogue number

Batch code

Serial number

Medical Device

Authorized representative in the European Community

Artery mark

Subject to change without notice.

Supply connection: internally powered equipment.

Mode of operation: continuous operation.

This medical device carries the CE mark and is manufactured in conformity with RoHS Directive 2011/65/EU and other applicable directive and/or regulations as specified in the EU Declaration of Conformity.

MEDICAL ELECTRICAL EQUIPMENT needs special precautions regarding EMC. NOTE: Do not use this device in the presence of electromagnetic or other interference outside the normal range specified in EN 60601-1-2. For detailed description of EMC requirements please visit www.HelenofTroy.com/emea-en/ support or contact service center. Portable and mobile RF communications equipment can affect MEDICAL ELECTRICAL EQUIPMENT.

Inquiry

In Europe, any serious incident (e.g., death, life-threatening injury, surgical intervention, etc) that has occurred in relation to this blood pressure monitor should be reported to Kaz Europe (see Helen of Troy Consumer Service Numbers) and to the competent authority of your member state.

The LOT and SN of your device are printed on the rating label on the back of the product. The first 3 numeric digits of the LOT No. represents the day of the year of manufacture. The next 2 numeric digits represent the last two numbers of the calendar year of manufacture and the letters at the end designate the manufacturer of the product. E.g., LOT 156140nb, this product was made on day 156, year 2014, at manufacturer identifier onb.

Please do not dispose of the product in the household waste at the end of its useful life. Disposal can take place at your local retailer or at appropriate collection points provided in your country.

Warranty

Please read all instructions before attempting to use this device. Please retain the receipt as proof of and date of purchase. The receipt must be presented when making any claim within the relevant warranty period. Any claim under the warranty will not be valid without a proof of purchase.

Your device is warrantied for two years (2 years) from date of purchase.

This warranty covers defects in materials or workmanship that occur under normal use; defective devices meeting these criteria will be replaced free of charge.

The warranty DOES NOT cover defects or damage resulting from abuse or failure to follow the user instructions. The warranty becomes void if the device is opened, tampered with, or used with non-Braun branded parts or accessories, or if repairs are undertaken by unauthorised persons.

Accessories and consumables are excluded from any warranty.

For support requests, please visit www.HelenofTroy.com/emea-en/support/ or find service contact information at the end of this owner's manual.

This warranty is applicable to Europe, Middle East and Africa only.

UK Only: This does not affect your consumer statutory rights.

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REF BUA4050EU, BUA4050EUAM

www.BraunHealthcare.com

Certain trademarks used under license from
The Procter & Gamble Company or its affiliates.

ExactFit™ is a trademark of Helen of Troy
Limited and/or its affiliates.

**Helen
of Troy**

 Kaz Europe Sàrl
A Helen of Troy Company
Q-Center, Chaux 4
CH-1030 Bussigny - Switzerland

0297

Made and
printed in China

© 2023, All rights reserved.
P/N A007824R2
13JUN23

Brand:	Braun
Category:	BPM
Model:	BUA4050EU and BUA4050EUAM
Artwork Part #:	A007824R2
Die Line Part #:	N/A
Subject:	Owner's Manual
Region:	EMEA
Flat Size:	MM: W180 x H160
Folded Size:	MM: W90 x H160
Scale:	1/1
Material:	40# gloss or matte text Minimum 80# gloss or matte text (cover)
Page Count:	176
Revision:	1
Date:	13JUN23
Release Date:	26JUN23
Re-release Date:	
Re-release Date:	

Colors:

Dielines (Do not print)

Spot Colors

Cyan
0%Magenta
0%Yellow
0%Black
100%PMS
XXXXPMS
XXXX**Special Instructions:****Quality Requirement of Artwork and Quality Clarification Process of Artwork Printing:
Meet Eng-QS-06&02**