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rok zał. 1949

TECH - MED[®]
TECHNIKA MEDYCZNA

ISO POLSKA
13485 MARKA

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MMAD ≤ 5µm

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TM-NEB 100

CONTENTS

WORKING PRINCIPLE	3
PRODUCT IDENTIFICATION	3
IMPORTANT SAFEGUARDS	5
HOW TO FILL THE MEDICATION	9
HOW TO ASSEMBLE	10
HOW TO USE THE UNIT	12
MAINTENANCE	14
TROUBLESHOOTING PROBLEMS	17
SPECIFICATIONS	18
MANUFACTURER'S DECLARATION	19
DISPOSAL	22

Thank you for purchasing the TECH-MED Piston Nebulizer TM-NEB 100.

1943
TECH - MED[®]
TECHNIKA MEDYCZNA

has always been with you for over 70 years - we use our experience to continually improve our products, which incorporate all the latest technologies in diagnostics and are constantly subjected to quality control. Your choice is the best possible one. We wish you good health. We are always at your disposal.

Be sure to read this Instruction Manual before using the unit for you to use it safely and correctly.

WORKING PRINCIPLE


Compressor Nebulizer TM-NEB 100 is a compact medical device design to efficiently deliver physician prescribed medication to the bronchial lung passages. It is intend for use in the treatment of asthma, COPD and other respiratory ailments in with an aerosolized medication is required during therapy.

Working principle: The compressor forces air to the nebulizer. When the air enters the nebulizer, it converts the prescribed medication into an aerosol of microscopic droplets that can easily be inhaled.

PRODUCT IDENTIFICATION

Before first use, make sure you have all the items listed in the manual. If any item is missing, please contact the store where you purchased the product or the nearest TECH-MED dealer.

1. Compressor
(Main Unit)

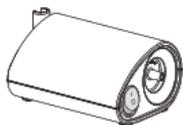2. Medicine Cup

3. Inhalation Mask
(Adult)

4. Inhalation Mask
(Children)

5. Mouthpiece

6. Nosepiece
(Optional)

7. Air Tube


8. Air Filters

9. DC Adaptor

Name of each part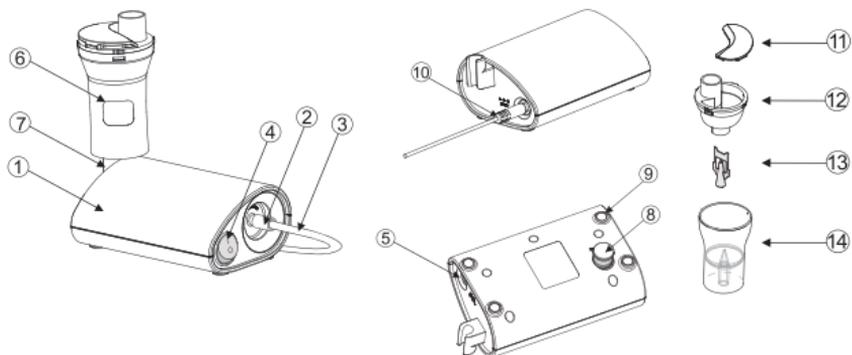

1. Compressor (Main Unit)

6. Medicine cup

11. Medicine cup cover

2. Air outlet	7. Medicine cup holder	12. Medicine cup upper shell
3. Air Tube	8. Air Filter Cover	13. Nebulizer slice
4. Power Switch	9. Footpad	14. Medicine cup lower shell
5. DC jack	10. DC adapter	

To start inhalation, you must assemble the unit.

Please read the „How to Assemble” chapter and understand the points for assembling before starting to use the unit.

IMPORTANT SAFEGUARDS

Warning: Be sure to read the notes on safety before using the unit

This unit is a medical instrument. Be sure to follow the instructions of a doctor and use the unit correctly. For the type, dose and regimen of medication, be sure to follow the instructions of a doctor.

The nebulization characteristics of this unit differ by the properties of the medication. Especially with the use of a medication having high surface activity or viscosity such as medication solubilizing agent or expectorant, the nebulization rate may be reduced. The nebulization rate may also be reduced when the temperature of medication is low.

Product caution:

1. For the type, dose and regimen of the medication, be sure to follow the Instructions of a doctor.
2. When you use the unit for the first time after purchasing it or after not using it for a long period of time, be sure to clean and disinfect Medicine Cup, Inhalation Mask Mouthpiece and Nosepiece (optional).
3. Clean and disinfect Medicine Cup, Inhalation Mask, and Mouthpiece after each use.
4. Clean and disinfect the Medicine Cup, Mouthpiece, Inhalation Mask and Nosepiece (optional) before using them for the first

time after purchase.

5. Be sure to dry the cleaned and disinfected parts promptly, and store them in a dean place. Do not place or attempt to dry the device, components or any of the nebulizer parts in a microwave oven.
6. Keep the device out of the reach of unsupervised infants and children. The device may contain small parts that can be swallowed, moreover strangulation due to cables and hoses may occur.
7. Do not wash the Main Unit and Power Plug with water or hot water.
8. Do not immerse the Main Unit in water or other liquid.
9. Do not cover the compressor with a blanket, towel, or any other type of cover during use. This could result in the compressor overheating or malfunctioning.
10. Do not use the device where the device may be exposed to flammable gas or vapors.
11. Pentamidine is not an approved medication for use with this device.
12. Always dispose of any remaining medication In the medicine cup after each use.
13. Do not leave the device or its parts where it will be exposed to extreme temperatures or changes in humidity, such as leaving the device in a vehicle during warm or hot months or where it will be exposed to direct sunlight.
14. Do not use or store the device where It may be exposed to noxious fumes or volatile substances.
15. Make sure that the Medicine Cup is clean before use.
16. Do not use in anaesthetic or ventilator breathing circuits.
17. Not suitable for use In the presence of flammable anesthetic mixture with air or oxygen or nitrous oxide.

Risk of Electrical Shock:

- Do not use the compressor (main unit) or Power Plug while they are wet.
- Do not plug or unplug the Power Plug Into the electrical outlet with wet hands.
- Do not use or store the device in humid locations, such as a

bathroom. Use the device within the operating temperature and humidity.

- Do not operate the device with a damaged power cord or plug.

Maintenance and Storage caution:

- Do not leave the cleaning solution in the nebulizer parts. Rinse the nebulizer parts with clean hot tap water after disinfecting.
- Store the device and the components in a clean, safe location.
- If you have any problems with this device, such as setting up, maintaining or using, please contact with service personnel to TECH-MED Technika Medyczna. Don't open or repair the device by yourself.

Caution:

1. Do not Inhale water in the Medicine Cup.
2. Do not spill water or other liquids on the compressor and Power Plug. These parts are not waterproof. If liquid spills on these parts, immediately unplug the Power Plug and wipe off the liquid with gauze or other soft absorbent material.
3. Do not drop or apply strong shock to the Main Unit or Medicine Cup.
4. Do not disassemble, repair or modify the unit.
5. Provide close supervision when this device is used by, on or near infants, children or compromised individuals.
6. If the device is used continuously, the service life of the device may be shortened.
7. Limit use to 25 minutes at a time, and allow a 40 minutes interval before using the device again.
8. Do not insert any object into the compressor.
9. Make sure the Air Fitter is clean. If the Air Filter has changed color or has been used for more than 60 days, replace it with a new one.
10. Make sure the Medicine Cup is correctly assembled, the Air Filter is properly installed, and the Air Tube is correctly connected to the compressor and the Medicine Cup. Air may leak from the Air Tube during use if not securely connected.
11. Do not use the device if the Air Tube is bent.

12. Do not add more than 8mL of medication to the Medicine Cup.
13. Do not operate the device at temperatures greater than +40°C (+104°F).
14. Do not tilt the Medicine Cup so the angle of the kit is greater than 45°. Medication may flow into the mouth.
15. Do not shake the Medicine Cup while using the device.
16. Use only TECH-MED Technika Medyczna authorized parts and accessories. Parts and accessories not approved for use with the device do not perform the expected specification or it may damage the unit. Changes or modifications not approved by TECH-MED Technika Medyczna will void the user warranty. To avoid injury to the nasal mucosa, do not squeeze the optional Nosepiece into the back of the nose.
17. When using this device, there will be some noise and vibration caused by the pump in the compressor.
18. There will also be some noise caused by the emission of compressed air from the Medicine Cup. This is normal and does not indicate a malfunction.
19. Do not use the device while sleeping or if drowsy.
20. Remove the Power Plug from the device after use.
21. Unplug the Power Plug from the electrical outlet before cleaning the device.
22. Do not leave the Medicine Cup with medication after use.
23. Do not store the Air Tube with moisture or medication remaining in the Air Tube. This could result in infection as a result of bacteria.
24. Do not wrap the Power Plug around the compressor (main unit).
25. People who are not trained with specified skills are not permitted to maintain the device.
26. If you have any problems with this device, such as setting up, maintaining or using, please contact service personnel to TECH-MED Technika Medyczna. Don't open or repair the device by yourself.

SYMBOLS MEANING

The warning signs and the sample icons shown here are listed for you to use this product safely and correctly as well as to prevent the risk and the damage to you and others from happening. The icons and meanings are as follows.

	Reading Instruction Book before use
	Type BF equipment
CE 0197	TUV NO.
	Manufacturer's name and address
	Some electrical and electrical equipments forbid to abandon and disposal at will
	Authorized representative in the European community
	Symbol for "SERIAL NUMBER"
	Attention consult accompanying documents
IPX1	Protection against ingress of water, it means the device could protected against vertically falling water drop 15°.


HOW TO FILL THE MEDICATION

Lift the Mouthpiece, or Nosepiece or Mask from the Medicine Cup, and unscrew the Nebulizer Upper Shell.

Caution: Be sure not to knock the baffler off to the ground.

The baffler is fixed on the Nebulizer Upper Shell by two hooks and can be pulled out With some power.

Fill in the correct amount of prescribed medication in the Nebulizer Bottom Shell.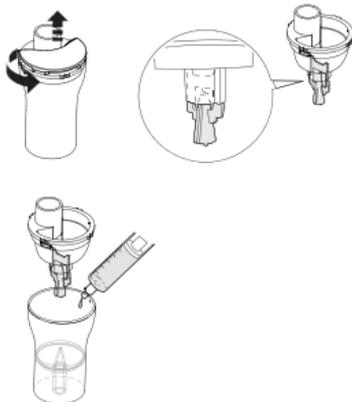

HOW TO ASSEMBLE

Caution: Make sure of that the mouthpiece, nosepiece, mask and air filter be clean before use.

- Clean and disinfect the Medicine Cup, mouthpiece, nosepiece or optional masks before using them for the first time after purchase.
- If the Unit has not been used for a long period of time, or if more than one person uses the same device, clean and disinfect the Medicine Cup, mouthpiece, nosepiece or optional masks before using them.
- Make sure the air filter is dean. If the air filter has changed color or has been used on average for more than 60 days, replace it with a new one.

1. Prepare the Power Source


Connect the DC adapter to the DC Jack of the Main Unit, and then to the power outlet as shown in the Figure.


⚠ Caution: Don't operate with wet hand!

2. Assemble the Medicine Cup


A. Screw the Nebulizer Upper Shell to the Nebulizer Bottom Shell clockwise until it is securely closed.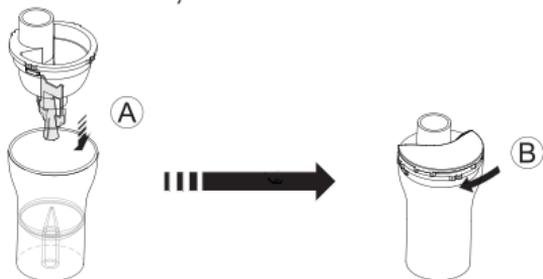

B. Assemble the desired inhalation accessories.

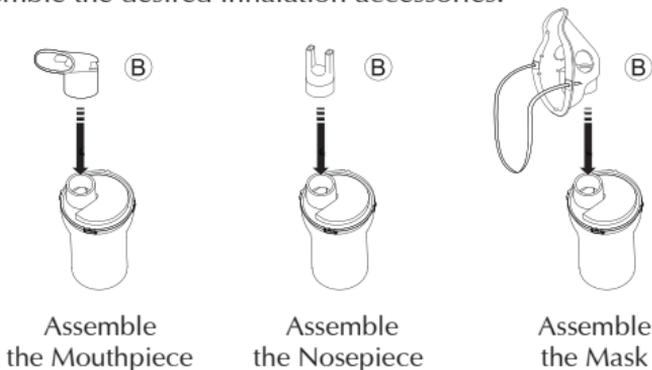
4. Assemble the Air Tube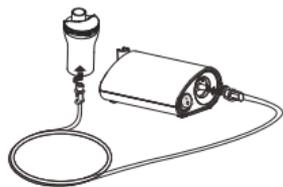

Step 1: Twist the Air Tube plug and push it firmly into the Air Tube Connector of the compressor.

Step 2: Twist the Air Tube plug slightly and push it firmly into the Air Tube connector on the bottom of the Medicine Cup.

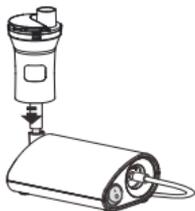Caution: Keep the Medicine Cup upright when assembling the Air Tube. Otherwise, medication may spill out of the Medicine Cup.

 **Caution:** Don't use the Unit if the Air Tube is bent.

Use the Medicine Cup Holder as a temporary holder for the Medicine Cup.

HOW TO USE THE UNIT

To start the inhaler and start nebulization, follow these steps:

1. Hold the Medicine Cup as shown in the Figure

Correct Posture

Caution: Do not tilt the Medicine Cup greater than 45°. Otherwise, medication may flow into the mouth.


2. Turn on the Unit

Turn the Power Switch to the ON position. As the compressor starts, nebulization begins.

3. Inhale medication as instructed by the Physician

Remember to stop inhaling when the medicine is used up.

The difference among the inhalation accessories as following.

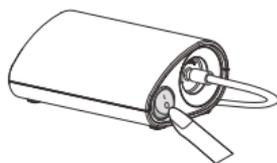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4. Turn off the Unit. Turn the Power Switch to the OFF position. As the compressor stops, nebulization stops.

5. Disconnect the Air Tube from the Medicine Cup

- Hold the Air Tube Plug and gently pull it down.
- Check the Air tube. No condensation or moisture should remain in the Air Tube.

If any condensation or moisture remains in the Air tube, remove the moisture from the air tube. Follow the directions below:

- Make sure the Air Tube is still connected to the air tube connector of the compressor.
- Turn on the Unit, and the compressor will start and pump air through the air tube to expel the moisture.
- Turn off the Unit.

6. Disconnect the air tube from the compressor

- Hold the Air Tube Plug and gently pull the Air Tube Plug off the Air Tube Connector on the upper side of the compressor.

7. Unplug the DC Adaptor from the power outlet

- Unplug the DC Adapter from the power outlet and DC Jack of the main unit.

MAINTENANCE

Be sure to clean and store the unit after inhalation.

1. Remove the inhalation accessory from the Medicine Cup.
2. Disconnect the Air Tube from the Medicine Cup.
3. Gently screw off the Medicine Cup Upper Shell.
4. Discard the remaining medication.
5. Wash the parts sufficiently in water.

⚠ Caution: Do not immerse the container in boiling water. Doing so may damage the container.

6. Hand dry or air dry in a clean environment using a soft, clean, lint-free cloth
7. Clean the casing of the main unit by using a soft cloth moistened with water or mild detergent.

⚠ Uwaga: Do not use abrasive cleaners.

8. Assemble the Medicine Cup and store it in a dry bag.

How to Disinfect

Disinfect the Medicine Cup and Mouthpiece or nosepiece after the last treatment of the day. - The method as described below:


- Use a commercially available disinfectant with the instructions provided by the disinfectant manufacturer, it is applicable to the masks' disinfection.
- Submerge the parts in the disinfectant solution for the specified period.
- Remove the parts and discard the solution.
- Rinse the parts with clean hot tap water, shake off excess water and allow to air dry in a clean environment.

⚠ Caution: Never dry the parts in the microwave oven.

⚠ Caution: Never use an autoclave, EOG gas disinfection or low temperature plasma sterilizer to disinfect the device.

How to clear the Air Filter

⚠ Caution: Do not wash or clean the Air Filter. It may cause a blockage. Do not use cotton or other material to substitute the Air Filter.

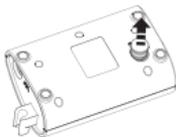
1. Pull the Air Filter cover to remove from the back side of the compressor.

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Caution: Before inserting the new Air Filter makes sure the Air Filter is clean. Do not operate the device without the Air Filter.

4. Put the Air Filter cover back to the compressor.

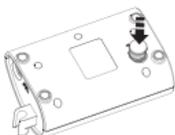⚠ NOTE:

- If the Air Filter has changed color or has been used for more than 60 days, replace it with a new one.
- If water or medication has spilled on the Air Filter, do not wash or clean the Air Filter. It may cause a blockage.
- Before inserting the new Air Filter makes sure the Air Filter is clean.
- Do not operate the device without the Air Filter. Use only the Air filters recommended by the manufacturer.

Product and operating Cautions:

- This unit is a medical instrument. Be sure to follow the instructions of a doctor and use the unit correctly.
- The nebulization characteristics of this unit differ by the properties of the medication. Especially with the use of a medication having high surface activity or viscosity such as medication solubilizing agent or expectorant, the nebulization rate may be reduced.

The nebulization rate may also be reduced when the temperature of medication is low. Do not use the device if it is damaged in any way. The continuous use of a damaged unit may cause injury, improper results, or serious risk. When not in use, store the device in a dry room and protect it against extreme moisture, heat, lint, dust and direct sunlight. Never place any heavy objects on the storage case.

- Provide close supervision when the device is used by, on, or near infants, children or compromised individuals.
- No modification of this equipment is allowed.
- Inhalation Mask is recommended when the main unit is used by a children less than 5 years old. Keep the device out of the reach of unsupervised infants and children. The mouthpiece or other small parts may be swallowed.
- The metallic sound may be generated, but it is not a trouble.
- The device shall be disconnected from the power source after use.
- The mouthpiece and inhalation mask contact the user directly, please keep it clean.
- Do not drop the device or subject it to other shocks or vibration. Never dry the parts in the microwave oven.
- Be noted that parts other than mouthpiece cannot be boiled!
- Do not carry or leave the nebulizer with medication in the medicine cup. If you have any problems with this device, such as setting up, maintaining or using, please contact service personnel to TECH-MED Technika Medyczna. Don't open or repair the device by yourself.

Storage:

- Do not store the device in locations exposed to high temperatures(over 50°C), low temperatures (below-20°C), high relative humidity(over 85%) or excessive amounts of dust.
- Never place any heavy objects on the storage case.
- Store the device and component in a clean, safe location.
- Keep the unit out of reach of small children.

TROUBLESHOOTING PROBLEMS

Trouble	Possible Cause	How to Correct
No power to the device when the Power Switch is on.	The DC adapter is not plugged into the main unit	Turn the Power Switch off Plug the DC adapter into the DC jack. Turn the device on.
No nebulization or low nebulization rate when the power is on.	No medication in the Medicine Cup. Too much or too little medication in the Medicine Cup.	Add the correct amount of prescribed medication to the Medicine Cup.
	The Baffler is not attached to the Medicine Cup Upper Shell or incorrectly positioned.	Make sure the Baffler is correctly attached to the Medicine Cup Upper Shell.
	The Medicine Cup is not correctly assembled.	Make sure the Medicine Cup is correctly assembled and the inhalation accessory is correctly attached.
	The nozzle is blocked.	Clean and disinfect the Medicine Cup to remove the blockage.
	The Medicine Cup is tilted at an incorrect angle.	Hold the Medicine Cup correctly. Do not tilt the Medicine Cup so the angle of the kit is greater than 45 degrees.
	The Air Tube is incorrectly attached.	Make sure the Air Tube is correctly attached to the compressor and the medicine Cup.
	The Air Tube is folded or damaged. The Air Tube is blocked.	Make sure the Air Tube is not folded, kinked or bent. Inspect the Air Tube for any damage. Replace the Air Tube if damaged.
	The Air Filter is dirty.	Replace the Air Filter with a new clean Air Filter.
The device is very hot.	The compressor is covered.	Do not cover the compressor with any type of cover during use.
	Operating continuously over 25 minutes.	Limit use to 25 minutes at a time and allow a 40 minutes interval before using the device again.

The device is abnormally loud.	The Air Tube Is Incorrectly attached.	Attach the Air Fitter cover correctly. Makes sure the Air Filter cover is not blocked.
--------------------------------	---------------------------------------	--

Note: Note: If the unit does not nebulize normally after taking the procedure mentioned above, contact the store where you purchased the unit or the nearest TECH-MED Technika Medyczna.

SPECIFICATIONS

Product Name	Piston nebulizer
Model	TM-NEB 100
Power source	AC 100V – 240V, 50Hz/60Hz
Working Environment	Temperature: 10 to 40 °C Humidity: 30 to 85% RH Atmospheric Pressure: 700 - 1060hPa
Storage Environment	Temperature: -20 to 50 °C Humidity: 30 to 85% RH Atmospheric Pressure: 700 - 1060hPa
Sound Level	≤ 52dB
Capacity of Medicine Cup	≤ 8 ml
Nebulization Rate	≥ 0.20 ml / min
Particle Size	MMAD approximately 5 um
Dimensions	144mm(L)*78mm(W)*67mm(H)
Weight (not including battery)	Approximately 260g (not including Medicine Cup and Air Tube)
Pollution	Degree 2
Voltage Category	CATEGORY II
Altitude Category	≤3000 m
Protection against harmful ingress of water or particulate matter	IP21
Free Flow Rate	4 l / min - 7 l / min

Nebulization Pressure	20 Kpa - 80 Kpa (2,9 psi-11,7 psi)
-----------------------	------------------------------------

- Nebulization rate is lowered at low temperature.
- Specifications and appearance may be changed without prior notice for improvement.

MANUFACTURER'S DECLARATION

1) This equipment needs to be installed and put into service in accordance with the information provided in the ACCOMPANYING DOCUMENTS;

2) IMMUNITY TEST LEVELS for basic safety and essential performance of ME equipment and ME systems should be chosen based on a high probability of maintaining basic safety and essential performance, and shall be according to the professional healthcare facility environment, home healthcare environment, and special environment, based on the locations of intended use.

3) HOME HEALTHCARE ENVIRONMENT is a dwelling place in which a patient lives or other places where patients are present, excluding professional healthcare facility environments where operators with medical training are continually available when patients are present. Such as schools, outdoors, domiciles, vehicle hotels and pensions.

4) Please discard the parts of the device once it is out of use according to local authorities.

5) MANUFACTURER will provide circuit diagrams, component part lists, descriptions, calibration instructions to assist to SERVICE PERSONNEL in parts repair.

A1 Electromagnetic Emissions-For all EQUIPMENT and SYSTEMS

Guidance and manufacture's declaration-electromagnetic emission		
The Piston nebulizer TM-NEB 100 is intended for use in the electromagnetic environment specified below; The customer or the user of the Piston nebulizer TM-NEB 100 should assure that it is used in such an environment.		
Emission test	Compliance	Electromagnetic environment-guidance

RF emissions CISPR 11	Group 1	The Piston nebulizer TM-NEB 100 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The Piston nebulizer TM-NEB 100 is suitable for use in all establishments, other than domestic and power supply network that supplies building used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	

A2 Electromagnetic Immunity -For Home Healthcare Environment EQUIPMENT and SYSTEMS

Guidance and manufacturer's declaration - electromagnetic Immunity

The Piston nebulizer TM-NEB 100 is intended for use in the electromagnetic environment specified below; The customer or the user of the Piston nebulizer TM-NEB 100, should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Poziom zgodności	Electromagnetic environment-guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	Floors should be wood, concrete or ceramic tile. If the floor is covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines	± 2 kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line to line ± 2 kV line to ground	± 1 kV line to line ± 2 kV line to ground	Mains power quality should be that of a typical commercial or hospital environment.

Pole magnetyczne o częstotliwości zasilania (50/60 Hz) IEC 61000-4-8	30A/m	30A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0%UT; 0,5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°	0%UT; 0,5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°	Mains power quality should be that of a typical commercial or hospital environment. If the user of the Piston nebulizer TM-NEB 100 requires continued operation during power mains interruptions, it is recommended that the Piston nebulizer TM-NEB 100 be powered from an uninterruptible power supply or a battery.
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms	The MANUFACTURER should consider reducing the minimum separation distance, based on RISK MANAGEMENT, and using higher IMMUNITY TEST LEVELS that are appropriate for the reduced minimum separation distance. Minimum separation distances for higher IMMUNITY TEST LEVELS shall be calculated using the following equation: [wzór] $E = \frac{6}{d} \sqrt{P}$ Where P is the maximum power in W, d is the minimum separation distance in m, and E is the IMMUNITY TEST LEVEL in V/m.
Radiated RF IEC 61000-4-3	10 V/m 80 MHz to 2.7 GHz	10 V/m	
NOTE: U_T is the a.c. mains voltage prior to the application of the test level.			

DISPOSAL

If a symbol visible next is affixed to the device, its ancillary equipment or the packaging, this means that such a product must not be disposed of with household waste. It must be disposed of at a recycling collection point for waste electrical and electronic equipment. In the European Union and other European countries, special systems are operating for the collection of waste electrical and electronic equipment. By disposing of your waste product properly, you are helping to prevent risks to the environment and human health. Recycling waste materials protects environmental resources. For more information on how to recycle this product, contact your local authority, waste disposal company or the retailer from whom you purchased it.

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Vapo Healthcare Co., Ltd.
Share Info GmbH