PL | EN | DE | IT | FR | ES | NL | UA |
LT | HR | CS | SK

AD-10

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EN

Dear customer!

Thank you for placing trust in us and choosing Overmax.

Thanks to high-quality materials and modern solutions, we provide you with a product perfect for everyday use. We are convinced that the product will satisfy your requirements as it has been manufactured with utmost care. Before you use the product, please read this manual carefully.

Should you have any remarks or questions concerning the product, please do not hesitate to contact us:

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PRECAUTIONS

1. Wait at least 15-20 minutes after drinking alcohol and smoking tobacco before testing. Remember to rinse your mouth with water.
2. For more accurate results, it is recommended to calibrate the device every 12 months or after 300 tests.
3. The device operates within a temperature range of 10 – 40°C, with optimal accuracy for tests conducted at room temperature (i.e., about 20 – 26°C).
4. A test result lower than 0.10 ‰ or 0.10 % BAC or 0.050 mg/L is natural if you have not consumed alcohol earlier. This is because, in some cases, it is possible to measure a substance other than alcohol in the exhaled air.

5. **Attention!** Do not use the device as a tool to determine and make decisions about the ability to drive vehicles. The displayed result of the amount of alcohol in the exhaled air is for informational purposes only. The manufacturer and seller disclaim all legal liability for the measured/estimated test value.
6. **Attention!** The result obtained using the estimated time to sobriety counter is approximate and is for informational purposes only.

DESCRIPTION (FIG. A)

1. Direction of blowing
2. Mouthpiece entry
3. Turn on and off / Start test / Display number of tests performed / Show last result
4. Display

BATTERIES

The breathalyzer requires two AAA batteries. To replace the batteries, press the cover located on the back of the device and slide it off the battery compartment. Remove the old batteries and insert new ones, paying special attention to their polarity. Close the battery compartment.

15 MINUTES BEFORE THE MEASUREMENT

Before proceeding with the test, wait at least 15 minutes after the last alcohol consumption. Rinse your mouth with water before performing the test.

During these fifteen minutes, do not consume alcohol or smoke cigarettes. If you consume alcohol or do not rinse your mouth, the test result may be artificially inflated due to the residue of alcohol. If you do not follow these recommendations, the test result may be incorrect, and you may also damage the sensor!

OPTIMAL TEMPERATURE FOR MEASUREMENT

It is recommended to perform tests and store the device at room temperature. Do not store the product in extremely humid places with extreme temperatures (e.g., in a car).

PERFORMING THE MEASUREMENT

1. Insert the mouthpiece into the appropriate place (2, fig. A).
2. **Caution** – tests performed without a mouthpiece may give inaccurate results. It is recommended to change the mouthpiece with automatic mode to an easier measurement mode. The product will automatically enter the automatic mode.
3. Press and hold the power button (B, fig. 1A) for 5 seconds. The sensor will begin to collect a sample (at the end of the about 20 seconds a manual mode will appear on the display). It is possible to perform the measurement in automatic or manual mode.
4. Choose the mode: automatic or manual.

AUTOMATIC MEASUREMENT DETECTION MODE

1. When the message  (abbreviation for „ready”) appears on the display and the device emits a short sound, take a deep breath and blow into the mouthpiece attached to the device. Make sure you exhale air long and steadily (not too fast, at a constant pace) for about 5 seconds.
2. The device will make a sound, and a message  will appear on the display. This means that the device has recognized a proper exhale.
3. Blow into the mouthpiece until you hear a double click sound, which indicates that the product has collected the sample needed for the measurement.
4. The breathalyzer will start analyzing the collected data. Wait a moment for the results to appear.

MANUAL MEASUREMENT START MODE

1. When the message  (abbreviation for „ready”) appears on the display and the device emits a short sound, press the power switch (3, fig. A) to start manual mode.
2. A message will appear on the display, and the device will emit a continuous sound signal.
3. Take a deep breath and blow continuously into the mouthpiece. Keep blowing until you hear a double click sound and a message  appears on the display. At this point, stop blowing air.
4. The device begins to analyze the blown air. Wait a moment, and the

result of the amount of alcohol in the exhaled air will be displayed.

DISPLAY FORMAT

The result of the amount of alcohol in the exhaled air will be displayed in any format (‰, %, mg/L) according to the BAC display format specific to a particular country. If you want to change the display format, press the power switch twice (3, fig. A).

TIME BETWEEN TESTS

After completing the first test and before conducting subsequent ones, etc., it is recommended to wait at least a few minutes (preferably 5-10 minutes) between tests to prevent the accumulation of alcohol residue on the sensor and to maintain the correct condition of the sensor.

ESTIMATED TIME TO SOBRIETY COUNTER

When the Blood Alcohol Content (BAC) is not zero, the device displays “BRT” (Best Rest Time), which represents the estimated time required to reduce the BAC level to 0.00, assuming no further alcohol consumption. BRT is displayed in hours and minutes format (H.MM). For example, a value of 2.45 means that it will take 2 hours and 45 minutes for the BAC level to drop to 0.00.

The device alternately displays the BAC and BRT results in several cycles, after which it turns off automatically.

The counter operates in the background, even when the device is turned off. When you turn the device back on, you will see how much time is

left for the BAC to drop to zero.

To reset the device, press and hold the power switch for 2 seconds until the device turns off. After turning the device back on, it will be ready for the next test.

Attention! The result obtained using the estimated time to sobriety counter is approximate and is for informational purposes only.

If the calculated optimal rest time is more than 9 hours 59 minutes, the timing continues in the background and will be displayed again as soon as it falls below 9 hours 59 minutes.

TEST RECORD MEMORY MODE

To check the total number of conducted tests for alcohol concentration in the exhaled air, follow these instructions:

1. After turning on the device, during the countdown, press the power switch to display the total number of conducted tests. This number also helps to decide when calibration is needed (see: Calibration).
2. Press the power switch again to display the result of the last conducted test.
3. Press again to display the results of previous tests.
4. To reset the counter, press the power switch multiple times.
5. It is recommended to recalibrate the device once a year or after 300 tests, whichever comes first.

SYSTEM MESSAGES

Message	Meaning of the Message
H	The device has detected a high concentration of alcohol (above 4.00‰ BAC / 0.400%BAC /2.00 mg/L).
FLo	Insufficient or uneven concentration of blown air detected. Another test should be performed.
Snr	The device's sensor is not ready. This happens when high concentrations of alcohol residues or contaminants are detected on the sensor from the last test. In such a case, turn off the device and wait a few minutes before conducting the next test.
rPt	The device could not complete the test. Repeat the test.
Out	The test was conducted at too low or too high a temperature. Return the device to normal room temperature. Before retesting, wait a few minutes for the sensor inside the device to stabilize to room temperature.
Out	The calculated optimal rest time is more than 9 hours 59 minutes. The timing continues in the background and will be displayed again as soon as it falls below 9 hours 59 minutes.
Lo	Low battery level. Insert new AAA alkaline batteries.

MAINTENANCE

To keep the device in good condition, it is recommended to use the device at least once a month.

EN

CALIBRATION

The device is factory calibrated.

The accuracy of the measurement can change after 6 to 12 months of normal use, depending on the number of tests performed and the conditions of use.

If the device provides inconsistent test results, no results, or exceptionally high or low results, it should be recalibrated by a qualified specialist. Contact service for calibration.

If the total number of tests performed is 300 or more, or you have used the device for more than a year since the last calibration (whichever comes first), it is recommended to send the device to a local distributor for recalibration.

The photos are for reference only, the actual look of the products may differ from the ones shown in the pictures.

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