Clear

Table of contents

Spis treści

4	Instrukcja obsługi	PL
9	User manual	EN
14	Bedienungsanleitung	DE
20	Návod k použití	CZ
25	Használati utasítás	HU
30	Používateľská príručka	SK
35	Vartotojo vadovas	LT
40	Lietotāja rokasgrāmata	LV
45	Kasutusjuhend	ET
50	Manual de utilizare	RO
55	Ръководство за употреба	BG
61	Manuel de l'utilisateur	FR
66	Manuale utente	IT
72	Manual del usuario	ES

A

B

C

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EN

USER MANUAL

Dear customer, thank you for choosing our brand.

The Xblitz Clear is a state-of-the-art breathalyser that features an advanced electrochemical sensor with high sensitivity, accuracy and repeatability of measurements. A large LCD screen and a mouthpiece with backflow protection ensure user comfort and ease of use. Features such as measurement history tracking, high precision and mobility make the device comfortable and safe to use and meet users' needs - in any situation. Before using the device, please read the following instructions.

CAUTION: Do not allow yourself or anyone else to drive after drinking alcohol.

PRECAUTIONS

1. Do not drop the device or shake it violently.
2. Test results may be inaccurate when battery capacity is low.
3. To ensure the accuracy of the measurement, perform the test 15 minutes after drinking alcohol.
4. If the device has been left unused for a long time, the first measurement taken may be inaccurate.
5. If you want to perform multiple tests, wait at least 1 minute between tests.
6. Do not blow tobacco smoke, liquids or pieces of food into the device.

This can permanently damage the sensor and lead to an incorrect measurement.

7. If the device is dirty, use a soft material to wipe it off. Do not use caustic cleaners, abrasive materials or wire brushes, as they may damage the housing of the device
8. The correlation between breath alcohol content and blood alcohol concentration depends on many variables, including environmental factors (such as wind, humidity, temperature, etc.) as well as the health of the person being tested.
9. Measurement results from this device do not constitute evidence in legal terms and cannot be used in court. The driver bears full responsibility for driving under the influence of alcohol.
10. The result of the test may vary depending on a number of factors. Getting a particular test result one time does not mean that the result will be the same even a few minutes later.
11. Do not allow yourself or anyone else to drive after drinking alcohol. Always use a taxi or other vehicle driven by a sober person when you or someone else has consumed alcohol.
12. Under normal use, the device needs to be re-calibrated approximately every 6 months or after 1000 tests performed. For accurate measurement results, contact your device distributor or service centre and ask them to recalibrate the device's sensor.

01. CONTENTS OF THE KIT

1. Breathalyser
2. AAA battery x2
3. Mouthpiece x5
4. User manual

02. DESCRIPTION OF THE DEVICE

SEE FIG. A

1. Mouthpiece/air outlet
2. Display
3. Buttons
4. Air outlet
5. Indicator light
6. Loudspeaker
7. Battery cover

03. DESCRIPTION OF THE DISPLAY

SEE FIG. B

1. Battery status
2. Measurement value
3. Measurement unit
4. Measurements from memory
5. Error
6. Blow
7. Wait

04. COMMUNICATIONS

The following are the messages that may appear on the display and their meaning.

1. **Message:** „8.8 8 8”
Meaning: Reading
2. **Message:** „1000 after switching on”
Meaning: Number of remaining measurements before necessary calibration
3. **Message:** Wait + „5 4 3 2 1”
Meaning: Sensor warm-up in progress, please wait
4. **Message:** „Wait + ---”
Meaning: analysis in progress
5. **Message:** „CAL”
Meaning: Calibration
6. **Message:** „Blow”
Meaning: Please blow
7. **Message:** „Blow+ Error +rp”
Meaning: error while blowing
8. **Message:** „Lob”
Meaning: Low battery level
9. **Message:** „RECORDS XX”
Meaning: Recorded measurement number XX

05. INSTRUMENT OPERATION

1. **Preparation for measurement and measurement**
 - a. Remove the battery compartment cover on the back of the device. Place 2 AAA 1.5V batteries inside. Make sure the battery terminals are facing the right way. Place the battery compartment cover back in place.
 - b. Place a clean mouthpiece in the device. If you are unsure where and how to place the mouthpiece, see the device diagram in the DEVICE DESCRIPTION section.
 - c. Press and hold the WATCH button in **FIG.C2** until you hear a short

beep to switch the device on. The instrument's display will turn on, indicate the number of remaining measurements before the necessary calibration and illuminate the indicator LED alternately in red, green and blue. It will then display a 5-second countdown to warm up the instrument's sensor.

NOTE: Do not blow into the mouthpiece until the countdown is complete. Premature use of the device may lead to an incorrect measurement of the breath alcohol content.

- d. When the display shows „BLOW“ and you hear a beep, take a deep breath and blow continuously into the mouthpiece, which will illuminate if a measurement is being taken. Blow until you hear a beep coming from the device. After a few seconds, the measurement result will be displayed on the screen.

NOTE: If the air flow is too low or is interrupted while blowing, the display will show „rP“ and display „BLOW“ and „ERROR“. Wait a few seconds before measuring again.

- e. If the measurement result is below the alarm threshold, the indicator LED will illuminate green. If the measurement result is at or above the alarm threshold, the LED will illuminate red and an alarm signal will sound from the unit.
- f. Press the **FIG.C2** button to turn off the result display and start the next measurement.
- g. The breathalyser will automatically switch off if there is prolonged inactivity to save battery power.
- h. If you wish to manually switch off the device, hold down the button for 3 seconds.

NOTE: If the test is repeated multiple times, replace the mouthpiece with a new one every 10 measurements or clean the mouthpiece of any water/saliva build-up. The presence of water/saliva in the mouthpiece can cause measurement errors.

2. Viewing stored measurements

With the unit switched on, press and hold the WATCH button in **FIG.C1** to review the history of measurements taken. The first record that appears on the screen is the last measurement taken. The unit can store up to 99 recently taken measurements. Use the buttons to browse through the measurements. Press the **FIG.C1** button to check the previous measurement or press the **FIG.C2** button to check the next measurement. Longer holding down the left button will take you back to the test interface.

3. Function settings

a. Enter settings mode:

With the unit switched on, press both the **PATCH FIG.C.1** and **FIG. C2** buttons simultaneously to enter the settings mode.

b. Changing the unit:

In setting mode, press the **PATCH FIG. C.1** button to change the flashing digit, then the **PATCH FIG. C.2** button to confirm the selection.

Enter 1111 to change the unit of measurement (default - ‰BAC).

Use the **PATCH FIG. C.1** and **FIG. C2** buttons to select your preferred unit. Once selected, press and hold the **PATCH FIG. C.2** button to confirm the selection or the **PATCH FIG. C.1** button to exit the setting without saving the changes.

c. Setting the alarm threshold:

In setup mode, press the **PATCH FIG. C.1** button to change the flashing digit, then the **PATCH FIG. C.2** button to confirm the selection.

Enter 1112 to change the alarm threshold (default - 0.200‰BAC). Press the **PATCH FIG. C.1** button to change the flashing digit, then the **PATCH FIG. C.2** button to confirm the selection. Once selected, press and hold the **PATCH FIG. C.2** button to confirm the selection or the **PATCH FIG. C.1** button to exit the setting without saving the changes.

4. Switching off the appliance

Hold down the **FIG. C.2** button to switch off the device.

06. SPECIFICATIONS

Sensor: Electrochemical

Measurement Range: 0~4,000‰BAC (0~2,000mg/L; 0~0.400%BAC; 0~4,000g/L)

Measurement resolution: 0.001‰BAC

Measurement error: ±0.050‰BAC for 0.500‰BAC

Units: mg/L, g/L, %BAC, ‰BAC

Sensor warm-up time: 5s

Operating conditions: 0°C - 40 °C humidity: ≤ 90% RH

Storage conditions: -10°C - 60°C, humidity: ≤90% RH

Display: LCD

Memory: Up to 99 measurements taken

Power supply: 2x AAA 1.5V batteries

Dimensions: 76x52x21.5mm

Weight: 53g (without battery)

07. WARRANTY CARD

The product is covered by a 60-month warranty. Warranty terms and conditions can be found at: <https://xblitz.pl/gwarancja>

Complaints should be made using the complaint form found at:

<https://reklamacje.kgktrend.pl/>

Details, contact and service address can be found at: www.xblitz.pl

Serial number.....

The warranty is only valid if the device is regularly serviced and calibrated at an authorised service centre. At least once a year. The device was calibrated before leaving the factory. Every breathalyser on the market requires periodic calibration, i.e. every 1,000 measurements, while at least every 12 months (every 6 months is recommended) the device must be reconfigured to correctly interpret the amount of alcohol in the air exhaled. The need for recalibration is not a defect in the device and is not subject to warranty. A device that is not regularly calibrated will show erroneous results.

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EN

The crossed out trash can symbol indicates that unusable electrical or electronic devices, its accessories (such as power supplies, cords) or components (for example batteries, if included) cannot be disposed of alongside with household waste. In order to dispose of the devices or its components (for example, batteries) deliver the device to the collection point, where it will be accepted free of charge. Disposal is subject to the recast version of the WEEE Directive (2012/19/ EU) and the Directive on batteries and accumulators (2006/66 / EC). Proper disposal of the device prevents degradation of the natural environment. Information about the collection points of the facilities is issued by the competent local authorities. Incorrect disposal of waste is subject to penalties provided for by the law in force in the given area.

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