07L

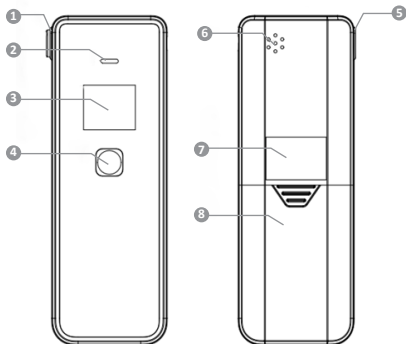
Table of contents

Spis treści

4	Instrukcja obsługi	PL
8	User manual	EN
12	Bedienungsanleitung	DE
16	Návod k použití	CZ
20	Használati utasítás	HU
24	Používateľská príručka	SK
28	Vartotojo vadovas	LT
32	Lietotāja rokasgrāmata	LV
36	Kasutusjuhend	ET
40	Ръководство за потребителя	BG
44	Manual de utilizare	RO
48	Manuel d'instructions	FR
52	Manuale di istruzioni	IT
57	Manual del usuario	ES

A

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Szanowny Kliencie,

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USER MANUAL

Dear Customer,

Thank you for choosing the Xblitz 07L. The device you have purchased is a solid-state breathalyser for measuring blood alcohol content from exhaled air. Please read the instructions below before use.

CAUTION: Do not allow yourself or anyone else to drive after drinking alcohol.

01. CONTENTS OF THE KIT

1. Breathalyser
2. AA battery x2
3. Mouthpiece x5
4. User manual

02. DESCRIPTION OF THE DEVICE

SEE FIG. A

1. Mouthpiece opening
2. Indicator light
3. Display
4. Button
5. Air outlet
6. Ventilation hole
7. Label
8. Battery cover

03. DESCRIPTION OF THE DISPLAY

SEE FIG. B

1. Battery indicator
2. Test result, messages
3. Unit

04. DISPLAY

The following are the messages that may appear on the display and their meaning.

1. Message: „C”

- Meaning:** Gas detected, please continue to blow
2. **Message:** „-“
Meaning: analysis in progress, please wait
3. **Message:** „Err“
Meaning: Sensor error
4. **Message:** „rP“
Meaning: Unstable exhalation or interrupted exhalation
5. **Message:** „blo“
Meaning: Start blowing into the mouthpiece of the device
6. **Message:** „lob“
Meaning: Low battery level

05. OPERATING THE DEVICE

1. Remove the battery box cover on the back of the unit and place two 1.5V AA batteries inside. Make sure the battery terminals are facing the correct way and reattach the container cover.
2. Fit the mouthpiece to the side of the device so that it is at a 90 degree angle to the rest of the device.
3. To start the device, hold down the button on the device for approximately 2 seconds. A beep will sound from the device, the LED will light up blue and the display will show a countdown from 20 to 0. Wait until the countdown is complete to ensure that the device sensor is ready for use.
NOTE: Do not blow into the mouthpiece until the countdown is complete. Premature use of the device may result in an inaccurate breath alcohol measurement. When the countdown is over, the display will show „blo“. Take a deep breath and start blowing into the mouthpiece. The display will show the letter „C“, continue blowing until a beep sounds from the device. After a few seconds, the measurement result will appear on the display. The result of the measurement is given in per mille (blood alcohol content).
NOTE: If during the measurement the flow of exhaled air is not constant or is interrupted, the display will show „rP“.
4. If the measured blood alcohol level is below 0.2 ‰, the LED under the display will light up green. If the measured blood alcohol level is above 0.2 ‰, the LED will light up red and an alarm signal will sound from the device.
5. If you want to manually switch off the device, hold down the button for 3 seconds. The device will automatically switch off a few dozen seconds after the last activity.
6. To change the unit of the result after the measurement has been completed, press the button. Each time the button is pressed, it will change to the next unit.

06. PRECAUTIONS

1. Do not drop the device or shake it violently.
2. Test results may be inaccurate when battery capacity is low.
3. To ensure the accuracy of the measurement, perform the test 15 minutes after drinking alcohol.
4. If the device has been left unused for a long time, the first measurement taken may be inaccurate.
5. If you want to perform multiple tests, wait at least 1 minute between tests.
6. Do not blow tobacco smoke, liquids or pieces of food into the device. This can permanently damage the sensors and lead to an incorrect measurement.
7. If the device is dirty, use a soft cloth to wipe it off. Do not use caustic cleaners or wire or hard cleaning instruments, they may damage the device housing. Do not use caustic cleaners or wire or hard cleaning utensils, as these could damage the appliance.
8. The correlation between breath alcohol content and blood alcohol concentration depends on many variables, including environmental factors (such as wind, humidity, temperature, etc.) as well as the health of the person being tested.
9. Measurement results from this device do not constitute legal evidence and cannot be used in court. The driver bears full responsibility for driving under the influence of alcohol.
10. The result of the test may vary depending on a number of factors. Getting a particular test result one time does not mean that the result will be the same a few minutes later.
11. Do not allow yourself or anyone else to drive after drinking alcohol. Always use a taxi or other vehicle driven by a sober person when you or someone else has consumed alcohol.
12. Under normal use, the device needs to be recalibrated approximately every 6 months or after 500 tests performed. For accurate measurement results, contact your device distributor or service centre and ask them to recalibrate the device's sensor.

07. FAQ

The following are the messages that may appear on the display and their meaning.

1. **Error:** Nothing is displayed on the LCD display
Possible reason: Incorrectly installed batteries
Solution: Correctly insert batteries in the device
Possible reason: Batteries nearly empty
Solution: Replace the batteries
Possible reason: Circuitry failure
Solution: Contact an authorised service centre

2. **Error:** No response to air being blown in
Possible cause: Unheated sensor
Solution: Wait for the countdown on the display to end
Possible cause: Circuitry failure
Solution: Contact an authorised service centre
3. **Error:** „Err” message appears and then the device switches off
Possible cause: Sensor fault
Solution: Contact an authorised service centre
4. **Error:** The device cannot be switched on
Possible reason: Dead batteries
Solution: Replace the batteries

08. SPECIFICATION

Sensor type: Solid-state

Measurement range: 0~ 2.00‰ BAC (0~ 1.00mg/ L; 0~ 2.00g/ L; 0~ 0.20% BAC; 0~ 1.00mg/ L).

Measurement resolution: 0.01BAC‰

Standby time: Approximately 35 s

Operating temperature: 0~ 50 °C

Permissible humidity: ≤ 93 %

Display: LCD

Batteries: 2x 1.5V AA

Dimensions: 124x44x19 mm

Weight: 49g (without batteries)

Error range: ± 0.1‰ BAC for 0.5BAC‰

09. WARRANTY CARD

The product is covered by a 60-month warranty. Warranty terms and conditions can be found at: <https://xblitz.pl/gwarancja>

Claims should be made using the claim form found at:

<https://reklamacje.kgktrend.pl/>

Details, contact and service address can be found at: www.xblitz.pl

Serial number.....

The warranty is only valid if the device is regularly serviced and calibrated at an authorised service centre at least once a year. The device has been correctly calibrated since new. Every breathalyser offered on the market requires periodic calibration, i.e. every 12 months (every 6 months is recommended) the device must be configured so that it correctly interprets the amount of alcohol in the exhaled air. The need for recalibration is not a defect in the device and is not subject to warranty. A device that is not regularly calibrated will show erroneous results.

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EN

The crossed out trash can symbol indicates that unusable electrical or electronic devices, its accessories (such as power supplies, cords) or components (for example batteries, if included) cannot be disposed of alongside with household waste. In order to dispose of the devices or its components (for example, batteries) deliver the device to the collection point, where it will be accepted free of charge. Disposal is subject to the recast version of the WEEE Directive (2012/19/ EU) and the Directive on batteries and accumulators (2006/66/ EC). Proper disposal of the device prevents degradation of the natural environment. Information about the collection points of the facilities is issued by the competent local authorities. Incorrect disposal of waste is subject to penalties provided for by the law in force in the given area.

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