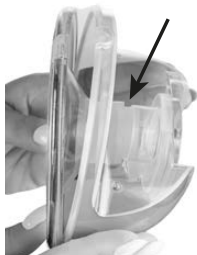
4.1

4.2a

4.2b

4.2c

4.3

4.4

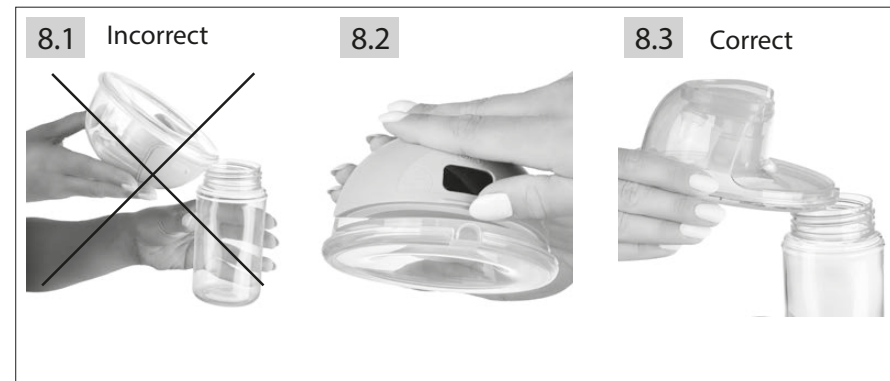
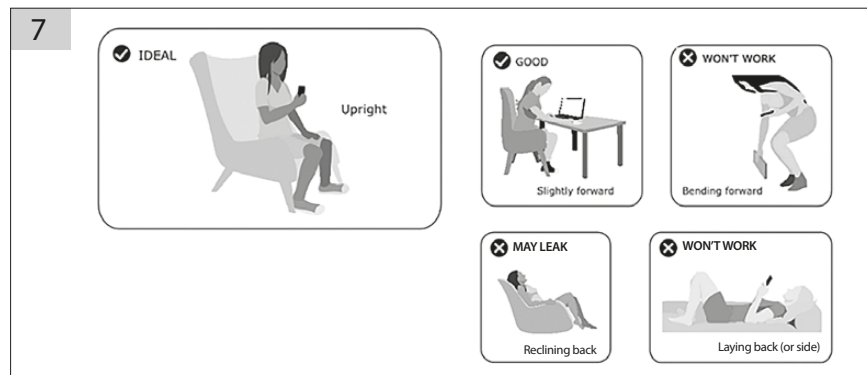
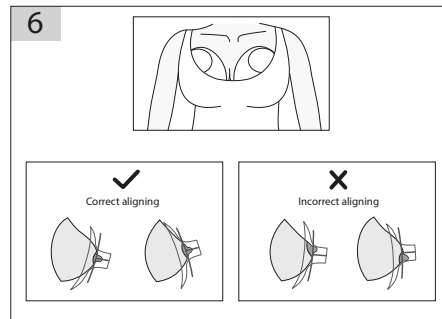
4.5

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Canpol babies Intelligent In-bra Breast Pump SmartSense
Model: 20/115, 20/116

SAFETY INFORMATION

Read all instructions carefully before using the Pump. This product is intended for home use only.
KEEP THIS MANUAL BECAUSE IT CONTAINS IMPORTANT INFORMATION. THE BATCH NUMBER INDICATED ON THE PACKAGING.

Important safety notes

Always follow basic safety rules when using electrical appliances, especially when children are around. Read all accompanying instructions before using this product. Follow the instructions for use.

Failure to do so may result in injury or damage to the breast pump.

WARNING

- Make sure you use a charger that is compatible with the breast pump specifications.
- Keep the charger away from sources of heat.
- Do not use the breast pump while the battery is charging!
- When unplugging the charger, first unplug it from the socket and then from the breast pump.
- Avoid using and storing the breast pump in places where it can easily fall or fall into the bathtub or sink.
- You can disinfect breast pump parts by boiling them. The breast pump body is not heat resistant, so keep it away from heat sources and open flames.
- Do not use your electric breast pump near flammable materials.
- Do not leave the breast pump switched on unattended.
- Do not use the breast pump if the charger or cords are damaged.
- Do not use the breast pump if it has fallen, become damaged or is not working properly. If you notice any signs of damage, stop using the breast pump immediately and contact: serwis@canpolbabies.com.
- Do not use an electric breast pump that has come into contact with water or other liquids. If the device has fallen into water or other liquid, do not touch it and immediately disconnect the charger from the socket.
- Do not soak the breast pump or immerse it in water or other liquids.
- Do not pour water over the breast pump.
- Do not use the breast pump in the shower or bath.
- Do not expose the breast pump to high temperatures.
- Remember that a short circuit may occur if the USB Type C port or

the device battery comes into contact with metal or liquid.

- Do not service or repair the product yourself. Do not modify the device.
- Do not use a damaged breast pump. Immediately replace damaged or worn accessories with new ones.
- Use the breast pump only for the purposes described in this manual.
- Do not use the breast pump while sleeping or when you are very sleepy.
- Do not use the breast pump while driving.
- Check all parts of the breast pump before EVER using it.
- Expressing breastmilk can cause uterine contractions. Do not express breast milk before giving birth. If you have become pregnant and are still breastfeeding or pumping, consult your doctor before using the machine again.
- Do not boil or heat pumped milk in the microwave. Breast milk heated in the microwave becomes very hot and can cause serious burns to the baby's mouth. Heating in the microwave can also change the composition of the milk.
- Use only items recommended by Canpol babies together with Canpol breast pumps.
- Take extreme care when using the breast pump or its parts near children.
- The breast pump is not intended for use by persons with physical, sensory or mental impairments or by persons without experience and knowledge, including children under 16 years of age. Children under the age of 16 should only use the breast pump if they understand the hazards of its use and have received instructions on safe use or under adult supervision.
- Keep all unused parts of the breast pump out of the reach of children to avoid accidental ingestion or damage. Remember that the machine and its parts are not toys.
- The plastic milk container and its parts become brittle when frozen and may break due to a fall or impact.
- Do not use milk that is in a damaged container or other damaged product for storage or feeding.
- If you are having trouble expressing or if expressing causes pain, contact your lactation consultant. This product is for personal use only. Use by more than one person poses a health risk.
- Portable radio-communication devices used in the vicinity of the breast pump may generate interference, causing it to malfunction.

INSTRUCTIONS FOR USE

Canpol babies smart shell breast pump SmartSense has been developed for lovers of the hands-free concept. It is the perfect solution when lactation is already stabilized, you are a KPI mum or you want to return to work. Ideal for the bra - the discreet and silent breast pump combines convenience with mobility, allowing you to pump hands-free at home during your daily routine, at work or on the go. We hope that you will find our breast pump very satisfying and pleasing. Before using the product, read the user manual carefully. Keep it for future reference as it contains important information.

BATTERY MAINTENANCE

The breast pump contains a built-in (non-replaceable), rechargeable lithium-ion battery.

WARNING:

- If the battery is completely discharged, do not use the breast pump immediately after connecting it to power. Leave the breast pump plugged in for at least 20 minutes before attempting to turn it on.
 - To keep the breast pump battery in good condition, check that the charger is compatible with the voltage and DC indicated in the product specification (5V 1A) before using it.
- To maintain the correct battery life cycle:
- Store the breast pump in a cool place.
 - Charge the battery before it completely discharges. This is helpful in maintaining the efficiency of the battery.
 - Fully charge the battery before putting the breast pump away for storage.
 - If the breast pump has been stored in a hot place, it should not be used immediately. To restore normal battery operation, wait one hour for the breast pump to cool down. During this time, the breast pump can be charged using the charger plugged into an outlet.
 - If the breast pump is stored for a long time without being used, it will slowly self-discharge the battery - in this case, it is a good idea to periodically connect the device to the charger.

Charging notes

- Strictly adhere to the following sequence of operations.
- Always use a DC5V 1A charger. The charger is not included in the kit.
- Make sure the charger matches the voltage of the socket.
- Connect the USB type C cable of the charger to the device. fig. 1.1
- Make sure the USB cable is not damaged.
- Insert the charger plug into a contact. The device will not start while charging. To switch the device on, disconnect it from the mains. fig. 1.2

MONTAGING

WARNING

- Before touching any breast pump or breast components, wash your hands thoroughly with water and soap (for at least one minute) and dry them with a clean towel or disposable paper towel.
- Remember that milk containers and their components can be damaged by improper use (e.g. by falling, tightening too much or tipping over).
- Be careful when using the milk containers and their components.
- Use only original electric breast pump parts. The use of substitutes may cause irreparable damage.
- Have the battery replaced by a specialized service center. The replacement battery must meet the same specifications as the battery being replaced and the current EU regulations.
- Ensure all components are completely dry before fitting.
- Assemble all components carefully, step by step, according to the instructions. Otherwise, the breast pump may not generate adequate vacuum.

COMPONENTS OF THE SET FIG.2.1

1. funnel size reducer 17 mm, 19 mm, 21 mm,
2. silicone flange size 24 mm,
3. valve,
4. control device,
5. membrane,
6. milk collecting container,
7. power cord type C (charger not included),
8. lid/stand,
9. bra strap extension 1 pc. (2 pcs. are included in the set with the double breast pump cat. no. 12/116).

To find out how to choose the right size of the funnel reducer, go to:

DESCRIPTION OF FUNCTIONS ON THE CONTROL PANEL FIG.3.1

- A - on/off/change modes,
B - decrease,
C - increase,
D - display,
E - heart icon - 2-in-1 mode,

- F - hand icon - breast stimulation mode,
G - battery level,
H - breast icon - breast pumping mode,
I - letter 'A' - automatic mode.

4 breast pump modes – adjust to your needs.

The breast pump offers four different modes that you can easily adjust to your stage of feeding, lactation level and your own preferences. Each mode has 12 levels of suction power - the breast pump remembers the last used settings for each mode.

2in1 mode - heart icon (E).

Stimulation and pumping in one cycle. This mode combines a gentle start to pumping with a transition to stronger suction movements - the machine quadruples in strength in one cycle. This is a good option to start with, but for some mums it may not be sufficient as the only mode. Switch to pumping mode if necessary - breast icon (H).

Stimulation mode - hand icon (F).

Stimulate milk flow. A short and fast sucking rhythm to stimulate the breast to start the flow of milk. Usually used for a few minutes before the actual pumping.

Recommended: at the start of each pumping session - especially for less sensitive breasts or when there are problems with outflow.

Extraction mode - breast icon (H).

Deep milk extraction. The suction rhythm in this mode is slower but deeper and more effective. Intended for the main part of the pumping session, after prior stimulation.

Recommended: after a few minutes of stimulation - as the main mode for pumping.

Automatic mode - A (I) icon.

Intelligent pumping without having to change modes. The breast pump switches from stimulation to pumping mode by itself: 5 cycles of stimulation + 10 cycles of pumping. This mode is designed for mums with flat or concave nipples, but will also work well for other mums.

Recommended: for beginners, for mums with lactation difficulties or when you want the convenience of a 'no click' mode.

HOW TO START?

It is best to start with the stimulation mode (hand icon - F), which gently stimulates the breasts and facilitates the flow of milk. After a few minutes, switch to pumping mode (breast icon - H) to effectively collect the milk.

USE

- Wash your hands before pumping and it is recommended to dry them with a paper towel.
- Placing the footrest under the legs and elevating the feet in an upright position helps to maintain a correct sitting posture and avoid pressure on the lower back.

- Place the funnel on the breast so that the nipple is in its central part and press lightly against the skin.

- Operation in all modes:

Press the switch for a long moment to switch the device on or off. On first start-up, the device will default to the stimulation level (heart icon), level 1.

On the next start-up, the device will start with the previously used mode and the previously used suction power. 2 seconds after start-up, the screen will display the device's operating time, pressing any button activates the display. You can adjust the mode, suction power and also switch between modes using the buttons on the control unit (button A, B and C). In each mode, you can adjust the suction power from 1 to 12 using the plus or minus button.

- The default operating time of this appliance is 20 minutes; the appliance will switch off automatically after this time.

- To avoid overfilling the milk container, it is recommended to stop operation of the machine when the volume of milk approaches 180 ml.

CLEANING

USB charging cable and breast pump control unit:

- Do not wash or disinfect in water.
- Wipe the surface with a paper towel or dry soft cloth.
- If the device is accidentally dropped in water or other liquid residue, discontinue use and contact customer service.

Other parts of the breast pump:

- Can be washed by hand with a mild detergent.
- Can be sterilised with steam, and boiling water. After boiling water, turn off the heat source and wait three minutes before disinfecting.
- Allow parts to dry.

ASSEMBLING

1. Wash your hands thoroughly. Gently place the non-return valve in the opening of the milk collection container. fig. 4.1 Make sure that the edges of the valve fit tightly into the opening.
 2. Fit the silicone funnel into the container. Start assembly by fitting the neck of the silicone funnel into the opening of the milk container by pressing it gently fig. 4.2A so that the top of the funnel fits snugly into the opening in the shell fig. 4.2B. Press the edges of the funnel against the shell until you feel resistance. Make sure the flange/edges of the funnel fit snugly against the edge of the container. fig. 4.2C.
 - 3 Turn the whole thing over and then place the silicone membrane, convex side inwards, over the opening in the container. fig. 4.3 Make sure the membrane fits tightly into the container.
 4. Fit the control unit starting from the bottom. Make sure it fits tightly into the container. fig. 4
 5. When not expressing milk, you can use the plastic cover of the funnel as a stand and put it down safely. Turn the stand over and place the breast pump in the cavity and the machine will wait until you are ready to continue pumping. fig. 4.5.
- ATTENTION!!!** All components must be fitted accurately, otherwise the operation of the unit will be affected and milk may leak. Immediately after use, disassemble the control unit by grasping the protruding part under the display. fig. 5. Then disassemble it and clean the parts that come into contact with the breast and breast milk.

PROPER APPLICATION OF THE MUSLE

Wash your hands thoroughly. Remember that the skin on the breast and the shell funnel must be dry before use. Insert the assembled breast pump into the bra so it is in the correct position with the nipple inside the drainage channel and the nipple protruding through the round opening. See fig. 6 You can use the bra extensions provided if you need to.

When expressing milk, remember not to tilt your chest backwards or forwards as this can cause milk to leak out. fig. 7.

RECOMMENDATION:

We recommend that you use the device with a well-fitting bra, for example a sports bra, to avoid the shell sliding inside the cup.

When pumping, try to keep your upper body upright. Lean forward or backward a maximum of 45°. IF YOU LEAN TOO FAR FORWARD OR ARE IN MOTION AND YOU HAVE ALREADY EXPRESSED A LOT OF MILK, MILK MAY LEAK OUT OF THE CONTAINER THROUGH THE VENT TO THE OUTSIDE.

WHEN YOU NOTICE THAT MILK IS DRIPPING FROM UNDER THE BREAST PUMP, EMPTY THE CONTAINER AND CONTINUE PUMPING.

Note: Wearing a nursing bra or breast pump for long periods of time can cause discomfort in the nipple glands. Remove the containers

when you feel pain or discomfort. Control how often and how long you use the breast pump. When you have finished pumping, unhook your nursing bra and slowly remove the breast pump. Then detach the control unit.

FLOWING THE MILK

With the breast pump still running, lean forward slightly and make sure that the milk in the tunnel of the silicone funnel has completely flowed into the milk container. Switch off the device and gently, without turning it over, remove it from the breast. **CAUTION!!! Never pour milk before removing the control panel.** It may be flooded. Detach the control unit from the milk collection container by grasping the protruding part above the display fig. 8.1, 2, 3.

Prepare a bottle or container to hold the milk and pour the milk through the opening at the top of the container, following the correct method shown in fig. 9. After use, wash all parts of the breast pump according to the cleaning instructions. Do not use the shell as a milk storage container.

MILK STORAGE

- Due to the active biochemical and microbiological processes taking place in milk, try to store milk at a constant temperature.
- Expressed milk can be stored in bottles, sterile pouches or sealed containers with lids (which should be disinfected before each use).
- Label each container of pumped milk with the date.
- Only freeze freshly expressed milk. Do not fill the container to the brim, as frozen milk increases in volume.
- Expressed and frozen milk can be defrosted in the same way as other foods except for the use of the microwave oven. (The use of a microwave oven can deprive the milk of valuable values and antibodies).
- Frozen milk can be thawed overnight in the refrigerator and then reheated in stages in a warmer or in hot water.
- After thawing, shake or stir the container to combine the separated fat particles with the remaining volume of milk.
- DO NOT store milk on the shelves located on the refrigerator door.
- NEVER refreeze milk that has previously been thawed.
- DO NOT combine freshly expressed milk with thawed milk.
- DO NOT combine freshly pumped milk with previously frozen milk.
- NEVER heat expressed milk in the microwave or boil it.
- ALWAYS check the temperature of the milk before serving.
- Discard unused milk.

Type of breast milk	Temperature (°C)	Recommended storing time
Fresh milk	+4°C	up to 96 hours
	from +4°C to +15°C	maximum up to hours
	+19°C to +25°C (room temperature)	optimally up to 4 hours
Frozen milk	from +25°C to +37°C	maximum up to 4 hours
	-20°C	optimally up to 3-6 months
Thawed milk	+4°C	up to 24 hours (since total thaw)
	from +25°C to +37°C	up to 4 hours

TROUBLESHOOTING

Problem	Solution
Compatibility	The flange self size is 24mm and with extra 21mm 19mm 17mm funnel. Use Nipple measure card to choose the suitable funnel or flange.
If the outside of Breast pump gets wet	Disconnect, turn off and stop using. Allow the outside of the breast pump to dry.
Small or no pump strength	Check all connections to the breast pump and accessories are firm. Disassemble the breast pump and check all parts are clean and undamaged. If any element is broken, replace a new one. Assemble the breast pump as the instruction. Ensure the flange fits perfectly while pumping.
If there is breast milk spillage	Turn off the breast pump switch. Wipe the breast pump with a damp(not soggy) cloth. When all components are dry, reassemble it.
Not "Lactating" or not pumping milk	Make sure that the breast pump accessories are properly assembled and you can feel the suction. -If no milk comes out after two consecutive breast pumps, relax and rest for 10-15mins. -If no milk comes out, consult a doctor or breastfeeding specialist.

TECHNICAL SPECIFICATIONS

Model: 20/115, 20/116
weight: 220 g, with lid 245 g (±5g)
charger parameters: 5V/1A DC via USB-C port
built-in 3.7V 1000mAh battery
vacuum generated 250mmHg ±20% /34kPa ±3kPa
charging time: approximately 2h
operating time: approximately 2h

If you have any doubts about the safety of the product, contact your supplier.

The information contained in this manual may be subject to technical changes.

This symbol indicates that the product must not be included in household waste. By disposing of the product waste in a legal manner, you will help to avoid possible negative consequences for the environment and human health, which could otherwise arise due to improper handling. For further information regarding the recycling of waste from this product, contact the municipal authorities in your area, the relevant waste disposal service or the supplier of the product. This product and the accompanying information comply with the requirements of the WEEE Directive 2012/19/EU.

The manufacturer declares the conformity of the product with European Union directives and harmonized standards. The product has been labelled as a result of a conformity assessment procedure.
The product meets the requirements of the Electromagnetic Compatibility (EMC) Directive 2014/30/EU and the RoHS Directive 2011/65/EU.

The product is manufactured in protection class III.

NOTE!

The product is not a toy. Keep all its parts out of the reach of children. Keep the instructions, as they contain important information. Serial number indicated on the packaging.

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DECLARATION OF CONFORMITY EU No. 01/25

Manufacturer: **Canpol Sp. z o.o.**
Puławska Street 430, 02-884 Warsaw, Poland

Product name: **Intelligent In-bra Breast Pump SmartSense**
Brand: **Canpol babies**
Model: **20/115**

In relation to the described product, the manufacturer declare under the sole responsibility that the product to which this declaration refers is in line with the following European Union directives:

- 2014/30/EU** Electromagnetic compatibility (EMC)
- 2011/65/EU** Restrictions of the use of certain hazardous substances in electrical and electronic equipment (RoHS) and Commission Implementing Regulation (EU) **2015/864**

The product is in line with harmonized standards and other technical standards:

- EN 55014-1:2017 + A11:2020
- EN 55014-2:1997 + A1:2001 + A2:2008 + AC:1997 (EN 55014-2:2019)
- IEC 61000-4-2:2008
- IEC 61000-4-3:2020
- EN IEC 63000:2018

The last two digits of the year in which the marking was applied

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Warsaw, 04.06.2025

(Place and date of issuing the declaration)

(name, surname, position and signature of the authorized person)

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