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Tel: 86-755-33825988

Fax: 86-755-33825989

Main Site: No.25 1st Industry Zone, Fenghuang Road,
Xikeng Village, Henggang Town, Longgang District,
Shenzhen, Guangdong China

Additional site 1: 2-4 Floor, No.5 Shanzhuang Rd., Xikeng
Village, Henggang Town, Longgang District, Shenzhen City,
Guangdong Province, China

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I. WARNINGS

1. The patients cannot diagnose the disease and get treatment by themselves on the base of measurement result, they must follow the instruction of doctors.
2. The product is only used for human body temperature measurement. Please do not use this product for purposes other than body temperature measurement.
3. Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
4. This product contains sensitive electronic components, and should not be used directly in an environment with strong electromagnetic interference (such as near mobile phones, microwave etc.) to prevent temporary impact on its accuracy.
5. Please do not disassemble or repair this device by yourself including changing device.
6. Please do not bend or stretch the device hard.
7. It is forbidden to collide, drop or mix this product with sharp objects.
8. Measurement without use of disease diagnosis; it cannot be used for emergency and continuous measurement in surgery.
9. Children under the age of 12 are prohibited from using it on their own.
10. This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance.

11. This device cannot be used with HF surgical equipment.
12. Please do not use this product for these people who suffer from otitis external, tympanitis and other ear diseases.
13. It is forbidden to immerse the infrared thermometer in any liquid, and it is forbidden to use it in an excessively high or low temperature environment for a long time.

If you use or store this product beyond the range of specified temperature and humidity, maybe it can not reach original performance specification:

- Use environment:
temperature: +10-+40°C, humidity: 15-93%RH
- Storage environment:
temperature: -25-+55°C, humidity: 0-93%RH


II. TECHNICAL DATA

Model	TR15 (PG-IRT1615)
Voltage	2 batterie AAA/1,5V (not supplied)
Dimensions	36 x 166 x 78 mm
Net weight	~86 g (without batteries)
Accuracy	• (35,0 °C ~ 42,0 °C) ±0,2 °C • (95,0 °F ~ 107,6 °F) ±0,4°F • other temperature ±0,3°C
Measuring range	34-43°C (93.2-109.4°F)
Memory	32 memories
Classification	Type BF


III. PRODUCT DESCRIPTION

Intended use: Infrared Ear/Forehead Thermometer is intended to measure human body temperature by measuring ear canal or forehead.

Scope of application: It is suitable for displaying the body temperature of the measured object by measuring the heat radiation in the forehead.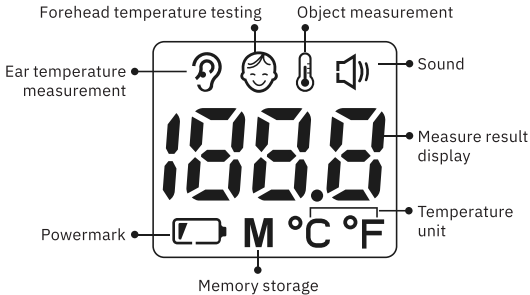

IV. PRODUCT LAYOUT

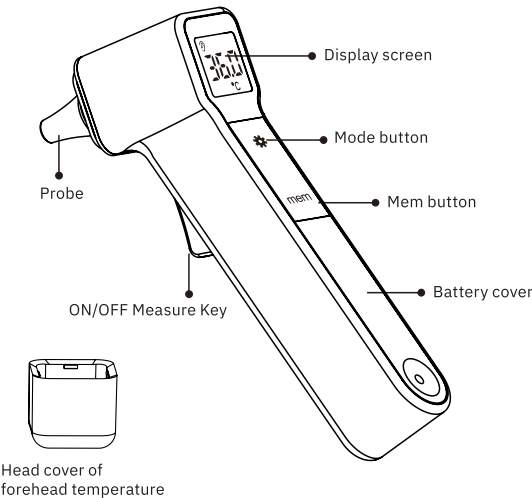
V. FUNCTIONS

1. Non-touching Klasyfikacjæ infrared measurement of ear/ forehead temperature.
2. Multiple colors and backlight display: Green, Orange and Red.
3. 32 sets of memory values.
4. The switch of degree Fahrenheit F and degree Celsius °C (original setting is degree Celsius °C)
5. Instant measurement within 1 second.
6. The design is convenient and economical without earmuff, which can save subsequent use costs.
7. It has the function of sound on/off.
8. The machine idle time of 15 seconds, turn off power automatically.

VI. BATTERY INSTALLATION

1. Press battery cover, it will bounce automatically.
2. Prepare two pieces of battery with the module of 1.5V AM (number seven battery) batteries (It is recommended to use alkaline batteries), please install it into base of battery according to correct positive and negative poles.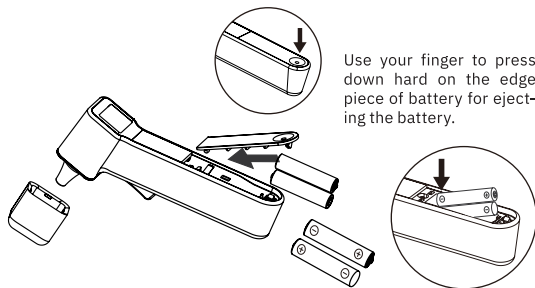

Insufficient power caution:

When battery voltage is lack, LCD shows  sign and battery mark is always on, which means you should replace the battery.

Warm prompts:

- If you do not use this product for a long time, please take out battery to assure its longevity. The liquid leakage of battery will harm the product; as well as pollute our environment.
- It is recommended to use alkaline batteries.
- The way to handle scrapped battery should match the requirements of local country government and environmental protection organ.

VII. WARNINGS BEFORE USE

1. It is very dangerous for patients to judge and treat on their own only by measurement results, so please be sure to follow the doctor's instructions.
2. Please do not touch with your hands or blow infrared sensor with your mouth.
 - When the infrared sensor is damaged or dirty, it may cause abnormal measurement results.
3. If there is a temperature difference between the storage site and the measurement site, please place it at room temperature (measurement site) for about 30 minutes before next measurement.
4. Please keep this product out of the reach of children.
 - When the child sticks to measure by himself, his ears may get hurt. If he swallows the battery or the transparent cover accidentally , please contact your doctor immediately.
5. A notice to the user and/or patient that any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.
6. When measuring body temperature, please do not get close to air conditioning.
7. Before and after use every time, use a cotton swab dipped 70% alcohol to wipe the probe surface. (If you see stains, fog or water on the infrared sensor glass, please use a cotton swab dipped 75% alcohol for wiping the infrared sensor glass gently.)
 - If you wipe it with toilet paper or facial tissue, it will scratch the infrared sensor resulting in incorrect measurement result.
 - Avoid cross-infection of ear disease and affect the accuracy of the measurement.
8. If the product suffers mechanical damage there is possibility of wrong measurement.

- The product touches water or immerses water accidentally, please fully dry before use, especially the water on the surface of the sensor should be clean by using cotton swab.
- Our aim is to avoid causing safety accidents and affecting measurement accuracy.

Caution:

- Do not use this product for these people who suffer from otitis external, tympanitis and other ear diseases. It is possible to worsen the affected area.
- Please do not use this product after swimming or bath or wet ears because it is possible to have low measurement result.
- Do not place waste battery to danger zone.
 - The battery may break.
- During ear measurement method, the product must be operated in the ear temperature mode. Result in inaccurate measurement result.
- If there is allergic reaction after using the device, immediately stop using it and call your doctor.


Suggestions:

- Please do not allow liquid(such as alcohol, water-drop, hot water and so on) to enter the product body because of this product without water resistance.
- This product should be stored in a dust-free, dry place, please avoid direct sunlight; Do not store in places with high temperatures, humidity, dust, or corrosive gases.
- If you find any problems, please contact the sales, you can not repair the product by yourself.
- Please deal with the waste and residue of this product at the end of the service life according to local laws and regulations.

VIII. PARAMETER SETTINGS

MODEL CALIBRATION

Turn on the machine to automatically measure the first temperature, and then press  to enter the calibration measurement mode.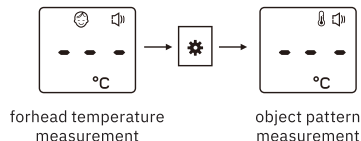

SOUND FUNCTION


- In the power-on state, press **MEM** key to set the sound on or off.
- Press **MEM** button, LCD screen will show  it means the sound works, meanwhile we will hear a short beep sound. Press **MEM** button again to turn off the sound.

THE SWITCH OF °F OR °C

In shutdown state, press and hold the **MEM** button for 6 seconds, when °F or °C flashing, press **MEM** button again to change unit to °C or °F. Wait for 8 seconds to turn off automatically or press **Measurement** to turn off.

MEMORY STORAGE FUNCTION

In off status, press **MEM** button, can read and save 32 sets of measurement values in order (as below picture shows). It will turn off automatically without operating for 15 seconds.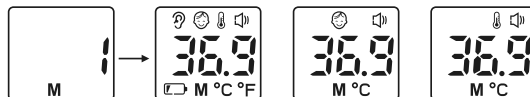

BACK LIGHT STATUS INSTRUCTIONS

Measure result	Display
<34°C	LO
	red backlight
34°C-37.1°C	green backlight
37.2°C-38.1°C	orange backlight
38.2°C-43.0°C	red backlight
>43.0°C	HI
	red backlight


IX. TEMPERATURE MEASUREMENT

Children within 1 y.o.:
pull ear back


Children over 1 y.o.
and adults:
pull ear above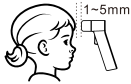

Forehead temperature:
the distance between the
thermometer and forehead
should be about 5mm

EAR TEMPERATURE MEASUREMENT


1. Remove the head cover to measure ear temperature.
2. Press **ON/OFF** button to enter the ear temperature measurement mode, display will show last measurement value.
3. Put the thermometer in your ear, if you hear the beep sound the measurement is finished .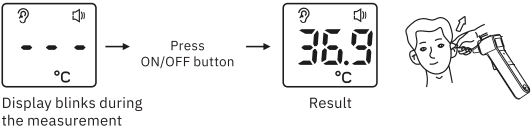

Warning: Don't remove the probe from ear until you hear the sound (If you it off, there won't be any sound).

FOREHEAD TEMPERATURE MEASUREMENT

1. Install head cover, which can measure forehead temperature.
2. Press **ON/OFF** button to enter the forehead temperature measurement mode, LCD will display last measurement value.
3. The distance between thermometer and forehead should be 1-5mm.
4. After 1 second you can see measurement result.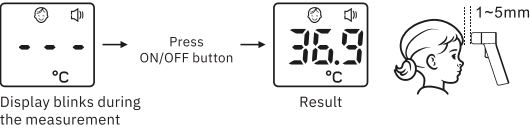


Warning: If you did not hear the beep sound, the measurement has not been completed yet. Please do not remove the thermometer probe from ear canal at this time. (If you turned off the sound indication, there will be not any sound).

OBJECT PATTERN MEASUREMENT

Installation of head cover has no influence on measurement result.

1. Just press **ON/OFF** button.
2. After 1 second you can see measurement result.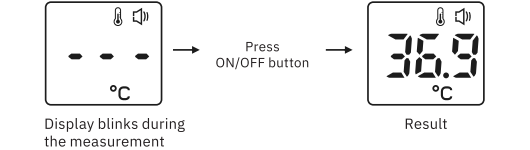


Warning: If you did not hear the beep sound, the measurement has not been completed yet. Please do not remove the thermometer probe from ear canal at this time. (If you turned off the sound indication, there will be not any sound).

X. CLEAN AND MAINTENANCE

In order to ensure the precise measurements, it recommended to clean the device after each use.

Please wipe the thermometer’s probe with a cotton swab dipped in 70% ethanol to remove residues and dirt, we recommend that you clean the probe for at least 60 seconds. And the rest part of the thermometer for at least 40 seconds. After cleaning, you should make sure that there is no visible dirt or residues on the thermometer.

The device can be cleaned for maximum 40000 times.

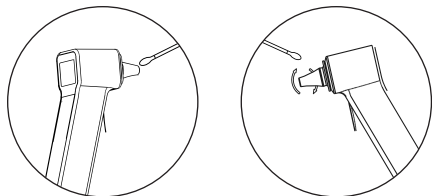Cleaning part: probe, body

Cleaner: 70% ethanol

Cleaning time: probe (60s), body (40s)

Drying time: 10 min

Warm prompts

Why do you need to clean the device after each use?

Because infrared temperature adopts highly sensitive technique to detect the temperature of the target object, not only will any earwax and dust paste affect the measurement accuracy, more likely to cause bacteria infection. So we suggest that you should do cleaning job well as the picture shows after you use every time.

XI. ERRORS ON SCREEN

Error	Meaning	Cause and solution
HI	Measurement result is higher than range (>43°C)	When the measurement probe is not properly placed in the ear canal or if the measurement distance is too far during the measurement process, the measurement result maybe be low.
Lo	Measurement result is lower than range (<34°C)	When measurement probe is dirty, the measurement value may be low; we should use a cotton swab to dip alcohol to wash the measurement probe properly.
Er.H	Environment temperature is higher than 40°C	Environment temperature cannot be higher than 40°C.
Er.L	Environment temperature is lower than 10°C	Environment temperature cannot be lower than 10°C.
Err	Operating temperature changed too fast by about 5°C	Place this product and keep steady in current environment for 30 minutes before next measurement.
	Operation is incorrect.	If the sound is turned on, it will read the value and remind at this moment (a short beep will be heard).

XII. TROUBLESHOOTING

Problem	Cause	Solution
Screen does not display even with power turn on.	Dead battery.	Replace the new battery.
	Batteries are placed incorrectly (check the polarity).	The battery polarity is the same as the battery case.
Measurement result is too low.	The measurement position is not correct.	Measure the temperature correctly according to the instructions.
	Sensor, probe or ear canal is dirty.	Please clear the dirt before the measurement.
Big temperature fluctuations with continuous measurement.	The measurement interval is too small.	The interval for each measurement should be above 10 seconds.

XIII. WARRANTY CONDITIONS

1. The warranty is granted for a period of 24 months from the date of sale. Defects discovered during this time will be removed free of charge in Poland by the service facility within 14 business days from the date of delivery to the service facility. The condition for granting the warranty is that the user read the user manual and use the equipment in accordance with this manual.

2. Warranty repair is understood as the performance of specialized activities by the service facility, appropriate to remove the defect covered by the warranty.

3. If you find a malfunctioning device, the fastest and most convenient way will be to fill out the online form available at: webber.com.pl/reklamacja.html

4. If any incorrect operation of the device is found, please contact the Manufacturer's Service Department: via the website: www.webber.com.pl/reklamacje or by phone: 61 1020 4922

5. The complaining party has the right to replace the equipment with a new one if:

a) during the warranty period, the service facility will make 4 repairs and the equipment will show defects that prevent it from being used as intended,

b) the authorized service facility states in writing that it is impossible to remove the defect,

c) the repair is not completed within the deadline specified in point. 1 or on another date agreed in writing with the complaining party.

6. The costs of repair or replacement, including in particular the costs of postage, transportation, labor and materials, are borne by the service facility.

7. The warranty does not exclude, limit or suspend the buyer's rights arising from the provisions of the Act of May 30, 2014. on consumer rights and other generally applicable legal provisions regulating the sale and liability of the manufacturer/seller for the goods.

A faulty device with a detailed description of the fault should be sent to the service address: Metrex – Service, ul. Grunwaldzka 229, 60-179 Poznań, PL; tel. 61 1020 492.

Disposal of used electrical and electronic equipment (applies to European Union countries and other European countries with separate waste collection systems).

This symbol on the product or its packaging means that the product should not be treated as household waste. It should be handed over to the appropriate point that collects and recycles electrical and electronic equipment. Correct disposal of the product will prevent potential negative consequences for the environment and human health that could otherwise be caused by inappropriate disposal of the product.

Recycling materials helps preserve natural resources. For detailed information about recycling this product, please contact your local authority, waste disposal service or the store where you purchased the product.

Lotus NL B.V.
Address: Koningin Julianaplein 10, 1e Verd,
2595AA, The Hague, Netherlands.
Tel: +31644168999

Shenzhen Pango Medical Electronics Co., Ltd
Tel: 86-755-33825988
Fax: 86-755-33825989
Main Site: No.25 1st Industry Zone, Fenghuang Road,
Xikeng Village, Henggang Town, Longgang District,
Shenzhen, Guangdong China
Additional site 1: 2-4 Floor, No.5 Shanzhuang Rd., Xikeng
Village, Henggang Town, Longgang District, Shenzhen City,
Guangdong Province, China

Importer:

METREX
Grunwaldzka 229
60-179 Poznań, PL
www.webber.com.pl

LEGENDS

	This icon is located next to the content of the order/warning		
	Warning		Read the instructions
	This icon is displayed next to a prohibition		
	Disassembly prohibition		
	Type BF		Medical device
	Read instructions before using product		
	The package of this unit can comply with the requirements of green environmental protection		
	The product/material is made of recyclable material		
	No discard it at will is allowed		
IP22	IP classification		Use by date
	Storage limit		Fragile, handle with care
	This way up		Protect from moisture
	Manufacturer		Unique device identifier
	Indicates the Authorised representative in the European Union		
	This symbol certifies that a product has met European Union consumer safety, health, or environmental requirements.		

METREX

ul. Grunwaldzka 229
60-179 Poznań, PL
[**www.webber.com.pl**](http://www.webber.com.pl)