
lionelo  
care

User manual  
Instrukcja obsługi  
Bedienungsanleitung  
Инструкция по эксплуатации  
Manuale d'uso  
Manuel de l'Utilisateur  
Manual de usuario  
Handleiding  
Naudojimo instrukcija  
Návod k obsluze

Használati utasítás  
Manual utilizare  
Bruksanvisning  
Bruksanvisning  
Brugsvejledning  
Käyttöopas  
Manual de instruções  
Οδηγίες χρήσης  
Kasutusjuhend  
Lietošanas instrukcija

# BabyAir Flow

Nasal aspirator  
Aspirator do nosa  
Nasensauger  
Назальный aspirатор  
Aspiratore per naso  
Aspirateur nasal  
Aspirador nasal  
Neusaspirator  
Nosies aspiratorius  
Aspirátor do nosu  
Aspirátor az orrhoz  
Aspirator nazal  
Aspirator för näsan  
Neseaspirator  
Aspirator til næsen  
Nenäaspiraattori  
Aspirador nasal  
Αναρροφητής μύτης  
Ninaaspirator  
Deguna aspirators

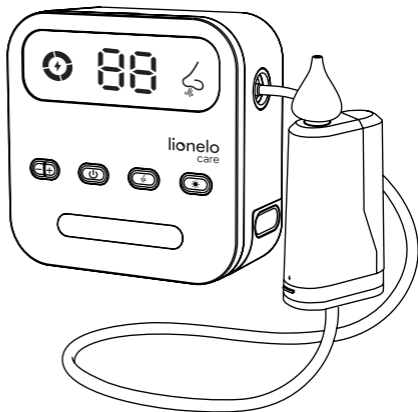1

2

3


4


5

6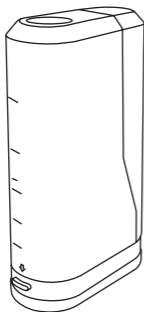


7

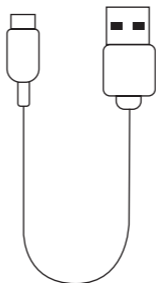
8



A

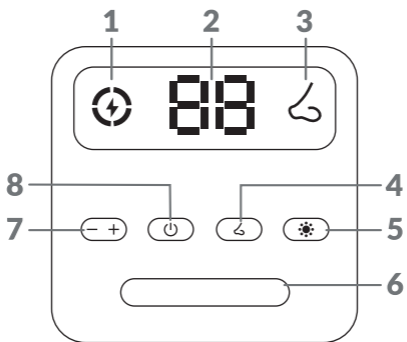


B


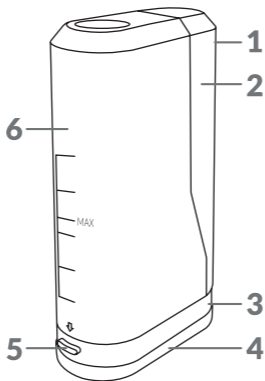
C

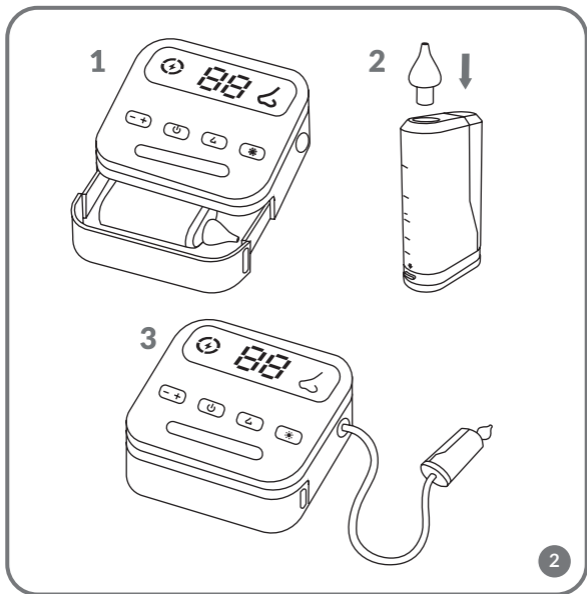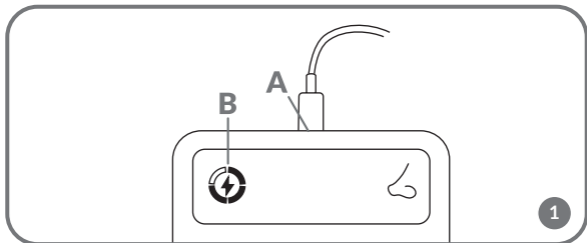
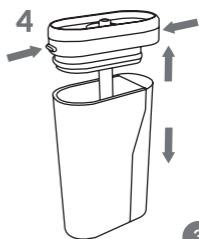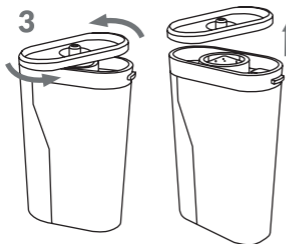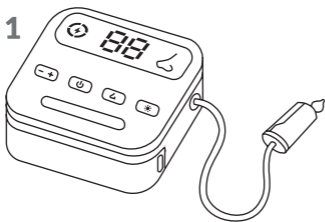
**3**

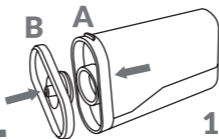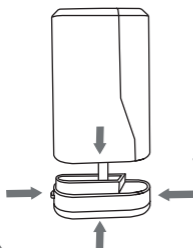**1**

**2**

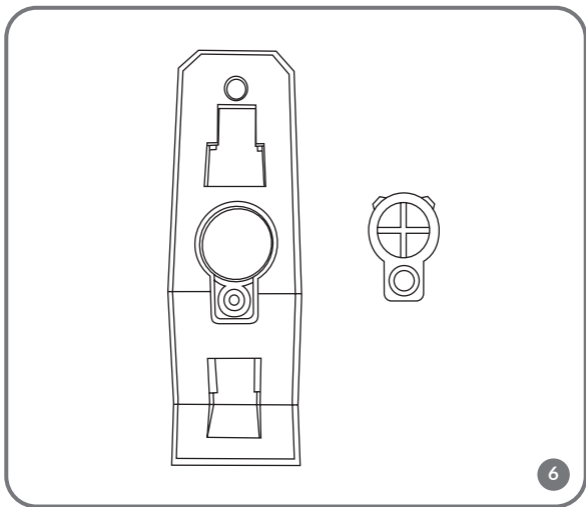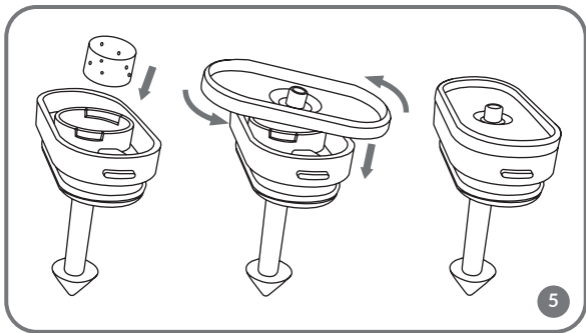- If necessary, clean the main unit with a damp cloth. Never submerge the main unit in water or any other liquids.
- Before reassembling the device, check to make sure that all parts are clean and dry.
- The aspirator tips and the secretion tank can be sterilised by steam or UV radiation.

*The photos are for illustrative purposes only, the actual appearance of the products may differ from that shown in the photos.*

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**EN:** The product complies with the requirements of European Union directives. In accordance with Directive 2012/19/EU, this product is subject to separate collection. The product should not be disposed of with municipal waste as it may pose a risk to the environment and human health. Return the used product to the recycling centre for electrical and electronic equipment.

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**EN:** The symbol placed on the product indicates that the batteries or accumulators used in it should not be disposed of with household waste. Such waste must be properly sorted in accordance with applicable regulations. Used batteries and accumulators must not be discarded in unsorted waste, as they may pose a risk to the environment and human health. Users should utilize available selective collection points, which enable their return, recycling, and safe disposal. For detailed information on local battery and accumulator recycling regulations, please contact your municipal office, local waste management facility, or landfill.

**Note:** Lithium batteries may pose a fire or explosion hazard, especially if damaged, overheated, or improperly disposed of. The chemicals contained within them may also be harmful to health and the environment.

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Detailed warranty conditions are available on the website:

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